



Ophthalmic Instruments



800.221.2166 | www.sklarcorp.com



TABLE OF CONTENTS

- Speculum 3-9
- Retractors & Expressors 10-12
- Fixation Rings 13
- Cutting Instruments 14-16
- Rongeurs 17
- Curettes..... 18
- Spatulas 19-21
- Hooks 22-24
- Scoops, Loops & Spoons 25-26
- Choppers & Manipulators ... 27-28
- Scissors 29-30
- Forceps 40-59
- Cannulas 60-65
- Foreign Body Instruments .. 66-67
- Probes & Dilators 68-69
- Needleholders 70-72
- Instrument Sets..... 73-75
- Accessories & Diagnostics 76-77
- Care & Cleaning..... 78-92

© Copyright 2016.
 Sklar Corporation, West Chester, PA. All rights reserved. No portion of this catalog may be reproduced without written permission of Sklar.



DISCLAIMERS:

Sklar offers a full line of quality surgical products and services to meet your needs. Product information within this catalog is intended to provide general information about Sklar, its products and services and makes no representation either expressed or implied to the contrary. It is not meant to provide specialist advice or instructions concerning the products available from Sklar. For additional information and special inquiries concerning our products and services, please contact Sklar directly. In an effort to continually improve our product line, Sklar reserves the right to change patterns without notice. Contact your sales representative to verify availability.

In no event is Sklar liable for any incidental, special or exemplary damages or loss of profits in connection with the use of a Sklar Instrument or Product.

Convenience Kits (FDA recognized term) illustrated in this catalog are representative of Sklar's assembly capability. The final design and approval of the kit is the responsibility of the customer and will be managed in accordance with the Sklar Design and Development process.

Sklar Corporation
889 S. Matlack Street
West Chester, PA 19382

Phone: 800.221.2166
Fax: 610.696.9007
Email: Surgi@sklarcorp.com

ORDERS

- Sklar's full line of Instrument Products are available through any authorized Sklar Distributor. Contact our Customer Service Department at 800.221.2166 to locate a Sales Representative in your area.
- Customers with established accounts and in good standing are subject to Net 30 days from date of invoice. Poor credit conditions can result in denial, delay or cancellation of orders.

SHIPMENTS

- Pricing is FOB West Chester PA unless otherwise noted and may be subject to change. Shipping schedules are approximate and subject to change without notice. Same day shipping is available. Standard orders will be shipped via the carrier of Sklar's choice. Special shipping requests will be accepted but additional handling charges are applicable.
- All Sklar instruments and product are carefully inspected prior to shipment. Customers shall inspect all items upon receipt and file a claim within (10) days of receipt of shipment.


WARRANTY

- Instruments and Products are recommended for their intended use only, and require continuous and proper care. Misuse can damage instruments and products and void the warranty.
- All Sklar instruments are guaranteed to be free from defect in material and workmanship. Defective instruments may be returned for repair or replacement at no charge. Repair or replacement is at the discretion of Sklar. Warranty is void on any instrument serviced by any company other than Sklar Instrument Repair Services.
- Sklar offers a variety of supplementary surgical products. These products are held to high standards for material and craftsmanship and some are subject to a limited warranty (vary by product).

RETURNS

- Unused instruments returned in the original packaging will be accepted within 30 days of purchase. Standard sterile kits may be returned within 30 days provided sterility of the inner packs have not been compromised, package is unopened and resalable. Most cleaning products are non-returnable.
- Product that was special (engraved/etched) or custom ordered, discontinued, marked or altered in any way, including product or package labeling, will not be accepted for return or credit.
- A Returned Goods Authorization (RGA) number must be obtained prior to returning products. Contact Customer Service at 800.221.2166 for authorization. Credit, repair and/or replacement are at Sklar's discretion. See www.sklarcorp.com for complete return policy and procedures.




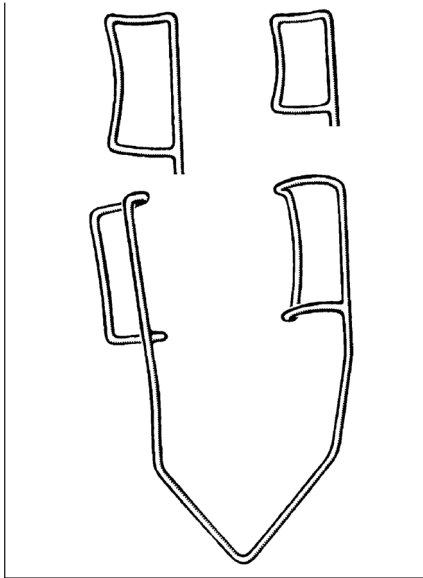

BARRAQUER WIRE SPECULUM

1½" LONG


65-1740	10mm loop	Child	
65-1742	15mm loop	Adult	
65-1744	15mm loop	Adult	Heavy Wire
65-1746	8mm loop	Infant	
65-1748	4mm loop	Infant	Delicate

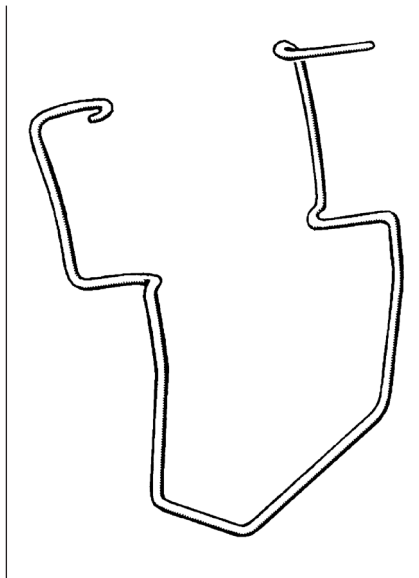

KRATZ-BARRAQUER WIRE EYE SPECULUM

BLADE WIDTH 14MM


65-1750	18mm Blade Spread	Adult
65-1752	15mm Blade Spread	Child
65-1754	10mm Blade Spread	Infant

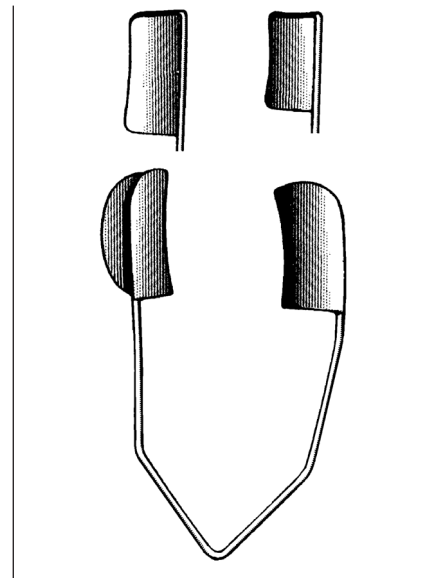

BARRAQUER WIRE SPECULUM

1½" LONG


65-1756	10mm Solid Blade	Child
65-1758	15mm Solid Blade	Adult

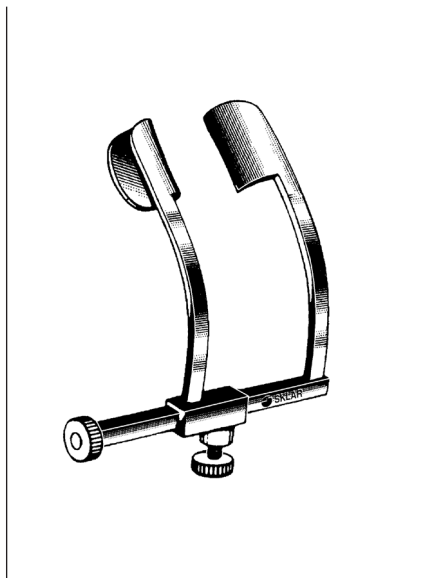

COOK EYE SPECULUM

2" LONG

65-1765	9x3mm Solid Blade Spread	25mm, Infant
65-1766	11x3mm Solid Blade Spread	27mm, Adult

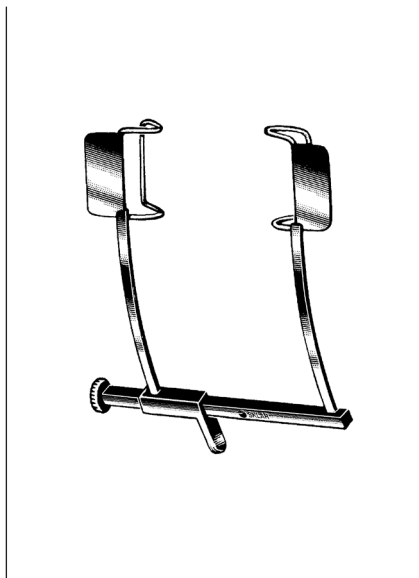

WIENER EYE SPECULUM

2¼"


65-1790	15mm Blades with Extensions to	Cover Eye Lashes, Chrome
---------	--------------------------------	--------------------------




MURDOCK EYE SPECULUM

2¼"


65-1540	15mm Spread
---------	-------------

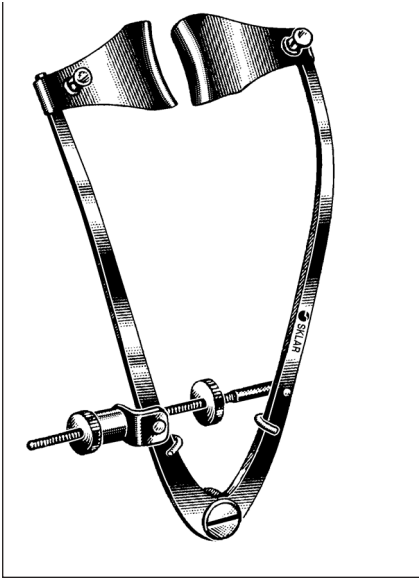


MCPHERSON EYE SPECULUM
65-7360 3¼" Blades 13mm Wide
with Suture Posts

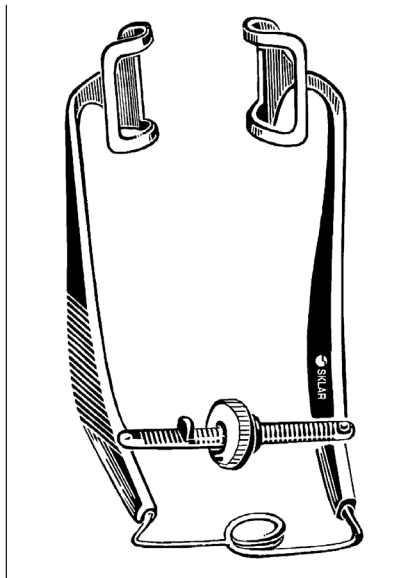


ZIEGLER SPECULUM
65-1522 2¼"

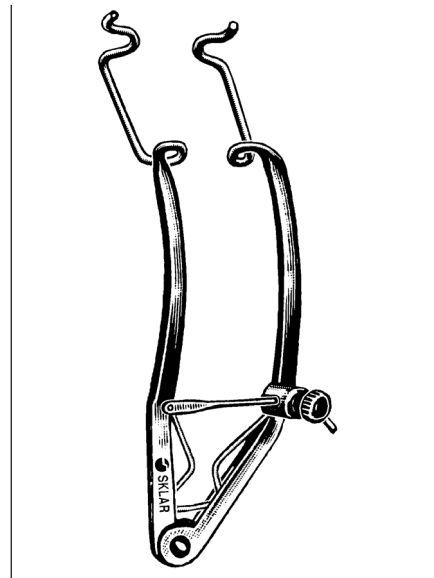


KNAPP EYE SPECULUM
65-1545 Non-Magnetic

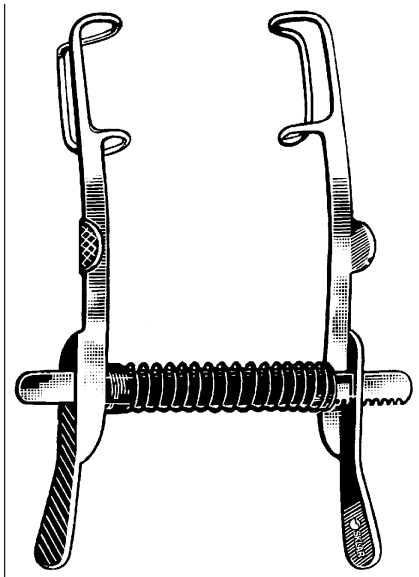


MELLINGER SPECULUM
65-1426 2¾" Solid Blades
65-1427 2¾" Fenestrated Blades

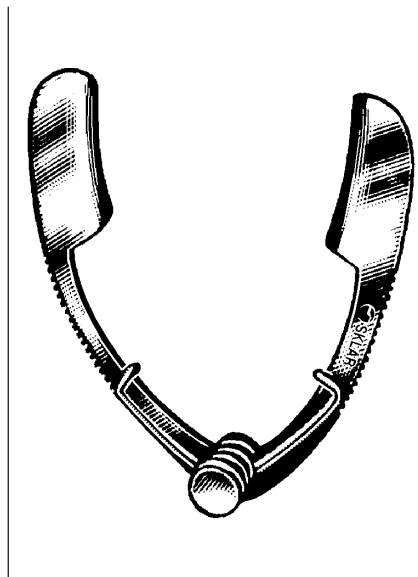


SAUER EYE SPECULUM
INFANT
65-1012 1¼" 10mm

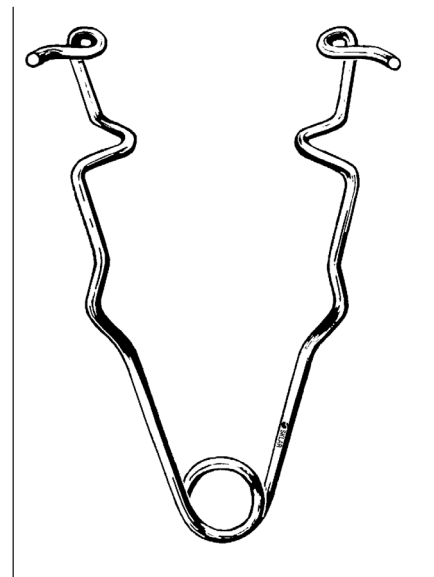


PLAIN WIRE SPECULUM
65-1127 2¾" Adult

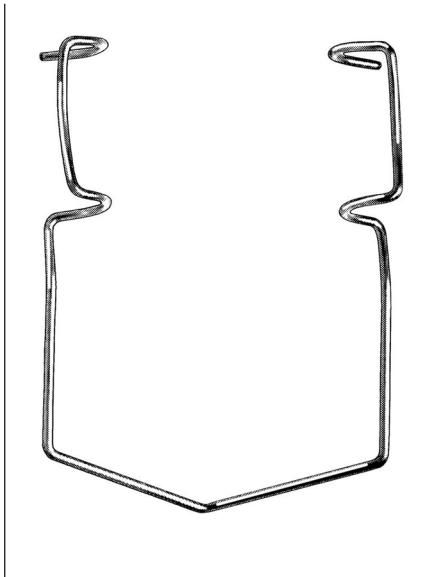


LEHNER SPECULUM
ANGLED UP 45 DEGREES
 65-1800 5" 14mm Blades

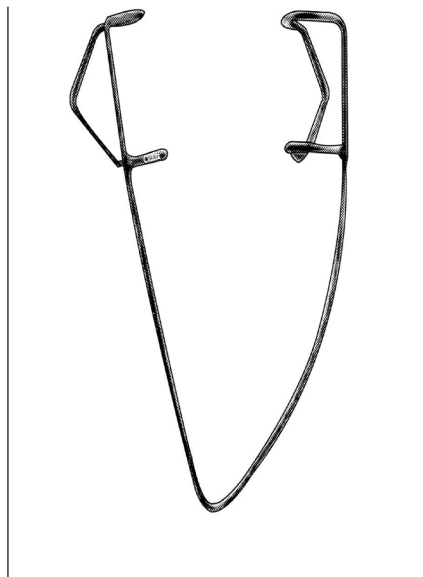


MCINTYRE WIRE SPECULUM
CLOSED WIRE
 65-7370 13mm V Shaped Blades

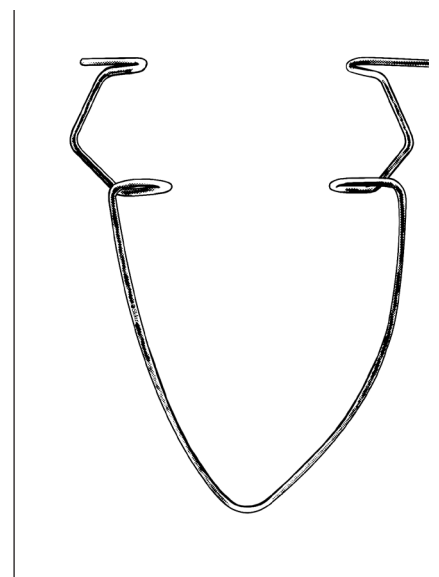


MCINTYRE WIRE SPECULUM
OPEN WIRE
 65-7371 13mm V Shaped Blades

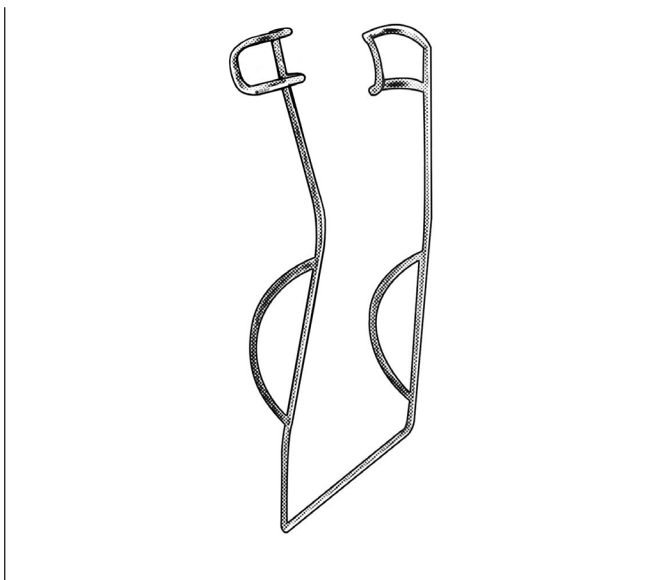

WIRE SPECULUM
 65-7685 3mm/5mm Interlacing

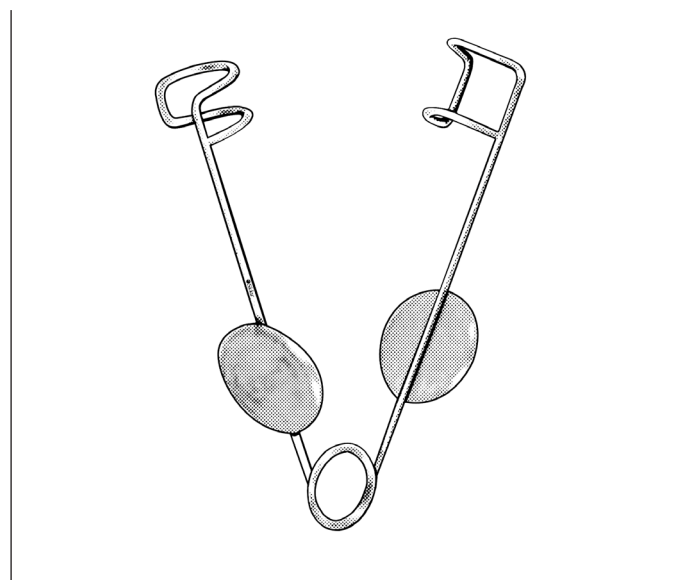


ALFONSO INFANT SPECULUM
 65-1538 25mm 5x8mm Blades
 Finger Grips

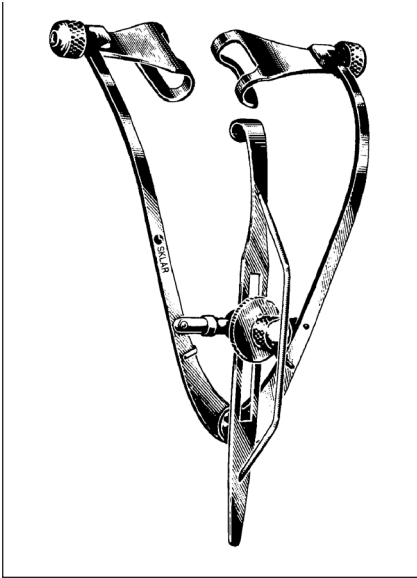


**PARK-MAUMENEE
EYE SPECULUM**

65-1528 3/4" Non-Magnetic Guard Over
Locking Screw

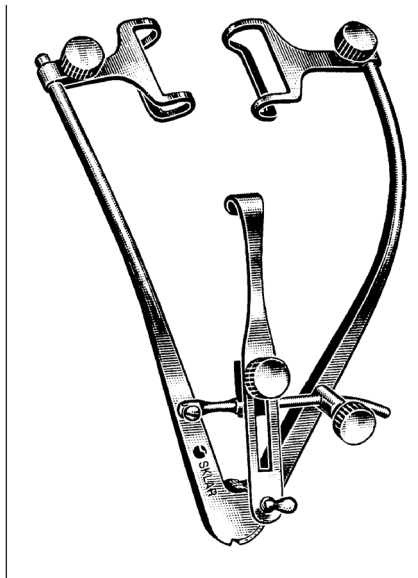

PARK-GUYTON EYE SPECULUM

65-1530 3"

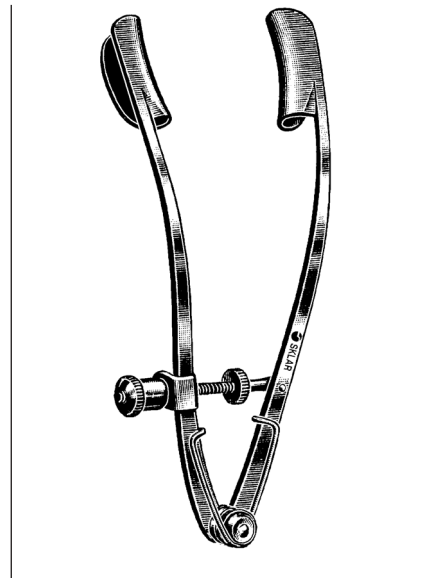


LANCASTER EYE SPECULUM

65-1532 3/4"




CASTROVIEJO SPECULUM

65-1535 3 1/2" 15x5mm Blades
65-1536 4" 15x5mm Blades

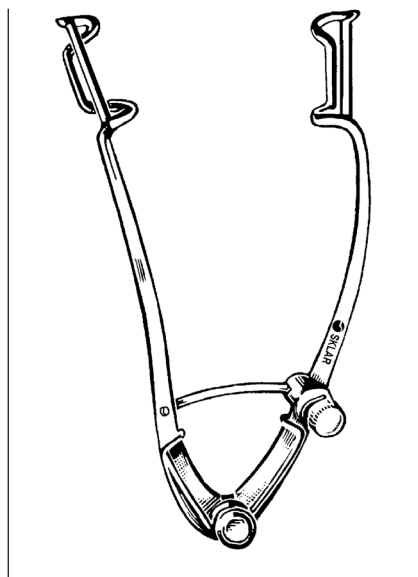


WILLIAMS SPECULUM

65-1227 2 3/4" Child
65-1332 3 1/4" Adult

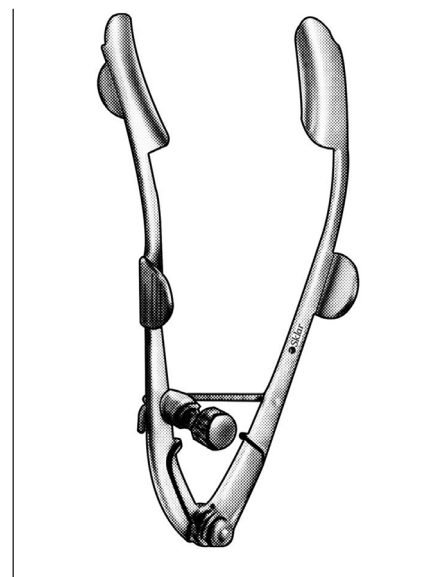

**LESTER-BURCH
EYE SPECULUM**

ADJUSTABLE, WITH LOCK
65-7680 Solid Blades with Flanges

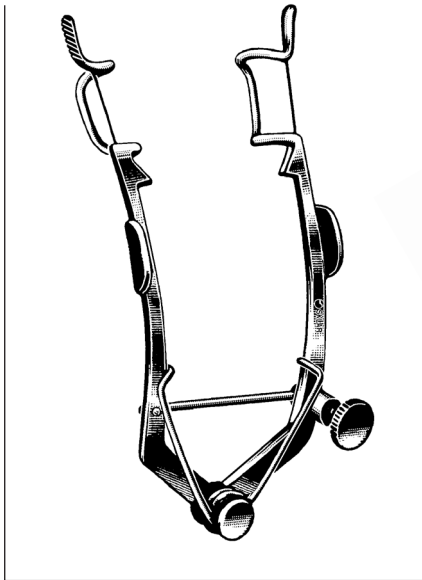


WEEKS EYE SPECULUM
65-1730 3¼", 15x5mm loop




**SPECULUM
ADJUSTABLE**
65-7681 15mm Thin Blades




**LIEBERMAN EYE SPECULUM
SOLID BLADES**
65-7678 10mm Child
65-7679 15mm Adult

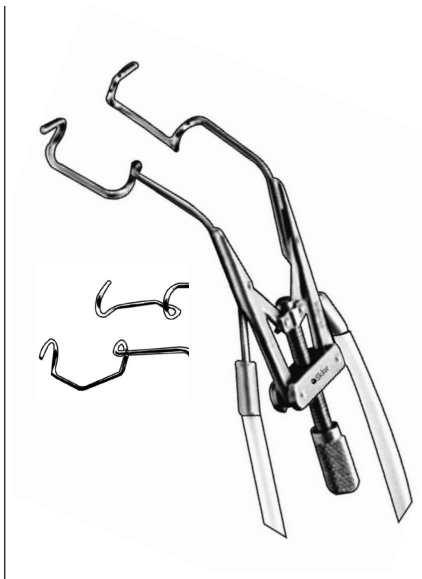


**LIEBERMAN EYE SPECULUM
ADJUSTABLE**
65-7375 15mm Open Blades
With Luer Lock and
Silicone Tubing
65-7385 15mm Open V Blades
With Luer Lock and
Silicone Tubing

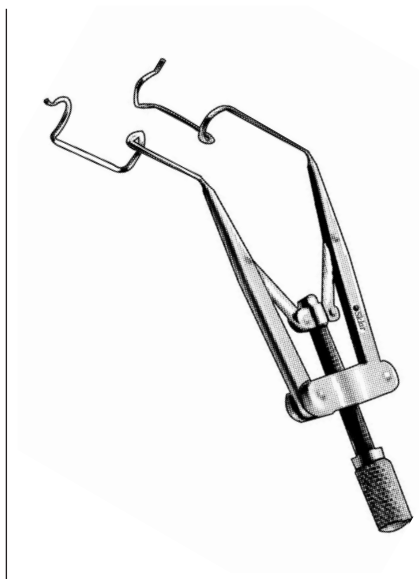

**LIEBERMAN ASPIRATING
EYE SPECULUM**
65-7677 15mm Open Blades

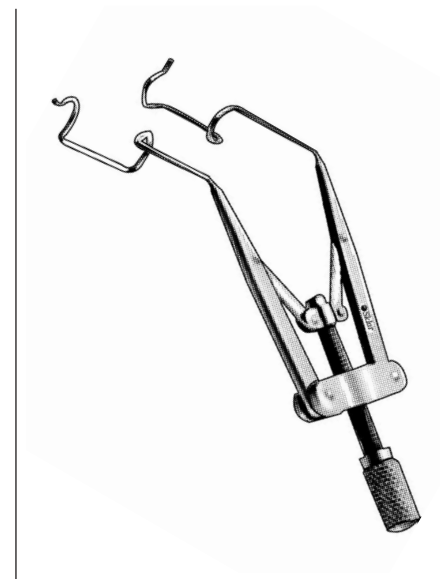


**LIEBERMAN EYE SPECULUM
KRATZ STYLE**
65-7684 10mm Child

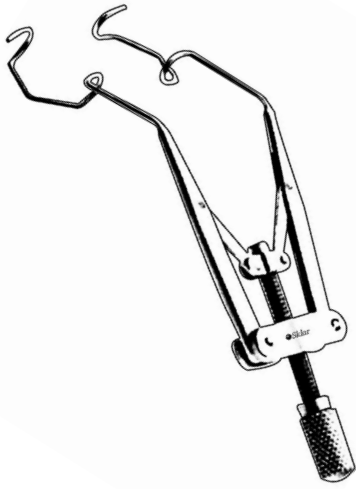

LIEBERMAN EYE SPECULUM

V SHAPED


65-7688 15mm Adult

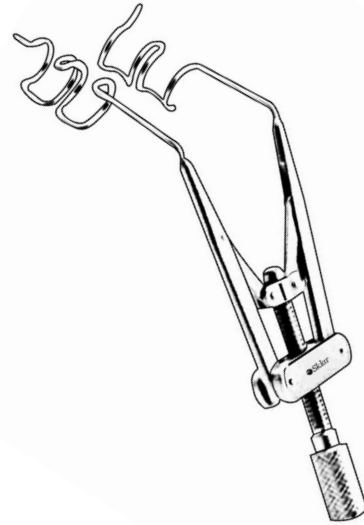

LIEBERMAN EYE SPECULUM

TRIPLE POST


65-7690 13mm

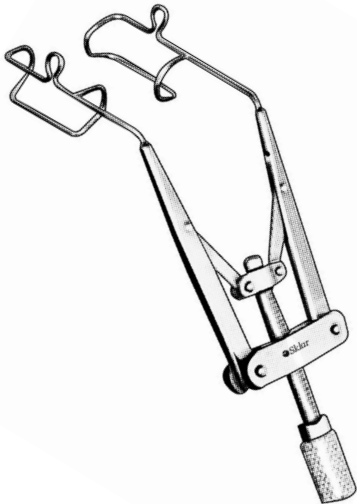

TANNA-LIEBERMAN SPECULUM

ADJUSTABLE WIRE BLADES

65-7682 15mm Adult

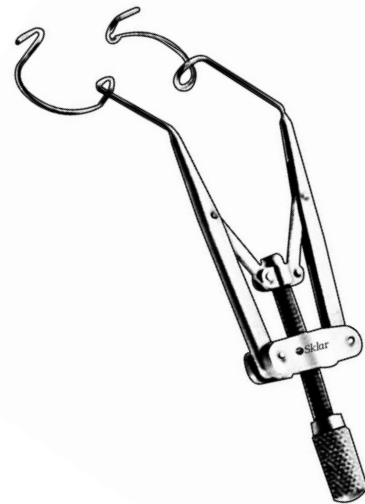


THORLAKSON EYE SPECULUM

ADJUSTABLE


65-7683 11mm Round Wire Blades




GINSBURG EYE SPECULUM
ADJUSTABLE
 65-1529 15mm Flared Upper Blades

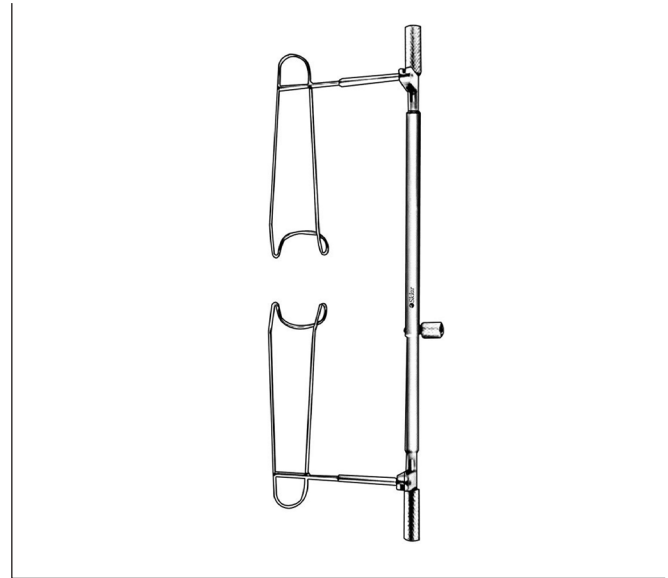

SCHOTT EYELID SPECULUM
ROTATABLE FOR BLADE ELEVATION
 65-7695 16mm Adult
 65-7696 11mm Adult

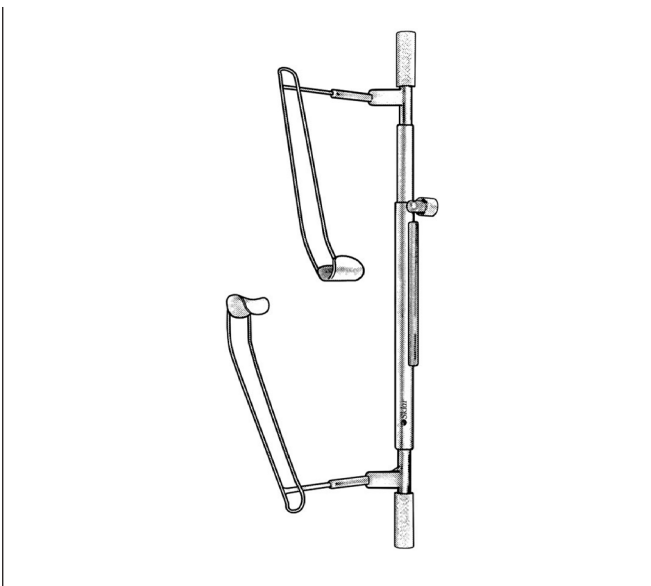

SCHOTT EYELID SPECULUM
SOLID, ROTATABLE FOR BLADE ELEVATION
 65-7697 16mm Adult
 65-7698 11mm Adult

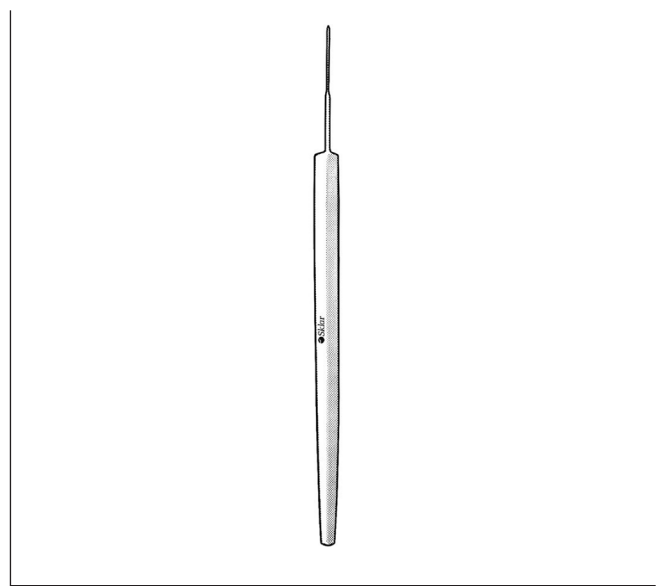

KERSHNER EYE SPECULUM
FENESTRATED
 65-1550




DESMARRES LID RETRACTORS

65-1841	#1	5½"	12mm
65-1842	#2	5½"	14mm
65-1843	#3	5½"	16mm
65-1844	#4	5½"	18mm




JAEGER LID PLATE

65-1835 4¼" Stainless

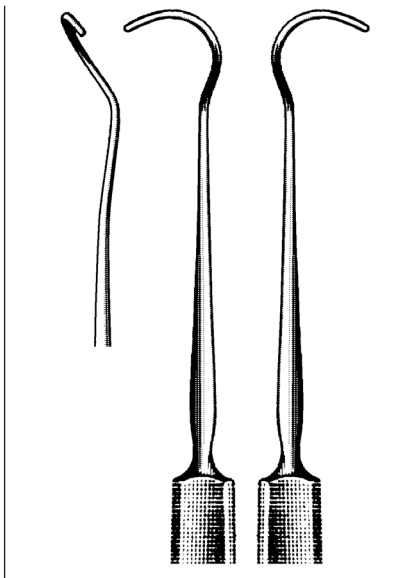


ARRUGA LENS EXPRESSORS

65-8350 Left
65-8352 Right

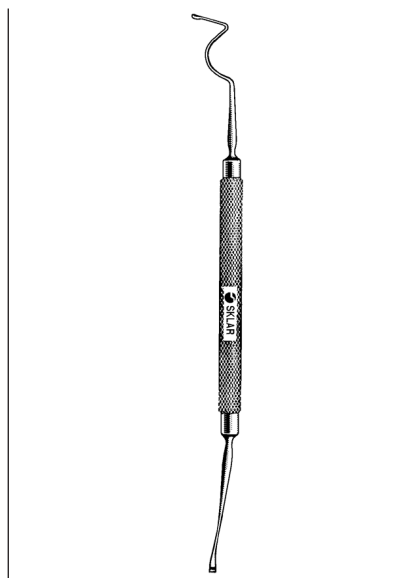


RIZZUTI IRIS RETRACTOR & LENS EXPRESSOR

5¾"
65-8354 2.5mm Blade

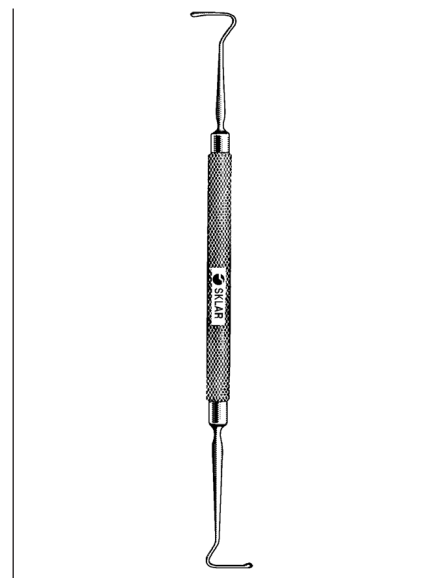


KIRBY MUSCLE HOOK & LENS EXPRESSOR


5¾"
65-9085 Double Ended
10x2mm Hooks

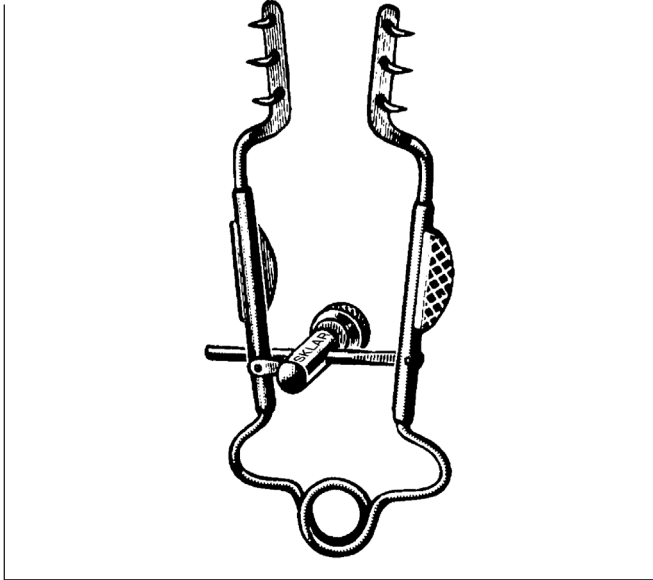


MELLER RETRACTOR
65-1830 3"

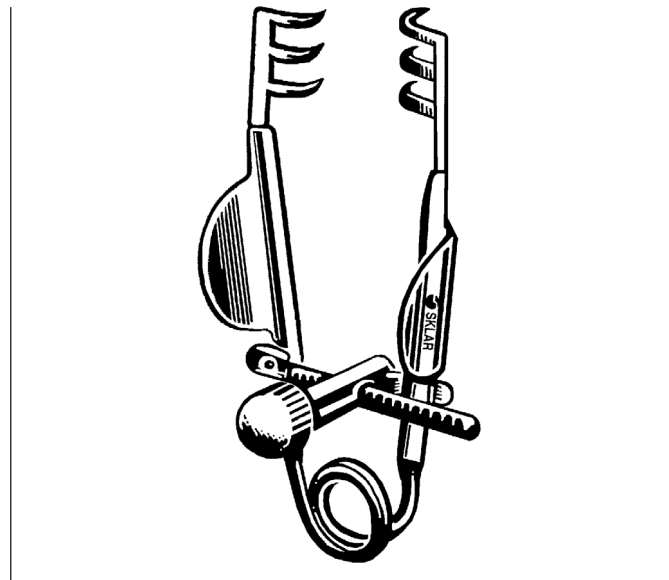


**MELLER RETRACTOR
DELICATE**
65-1815 1½"

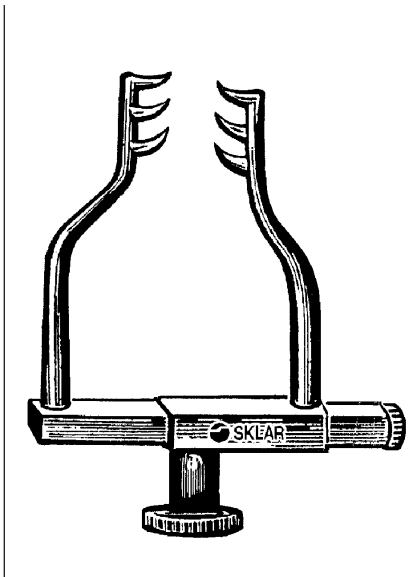


GOLDSTEIN RETRACTOR
47-2913 1½"

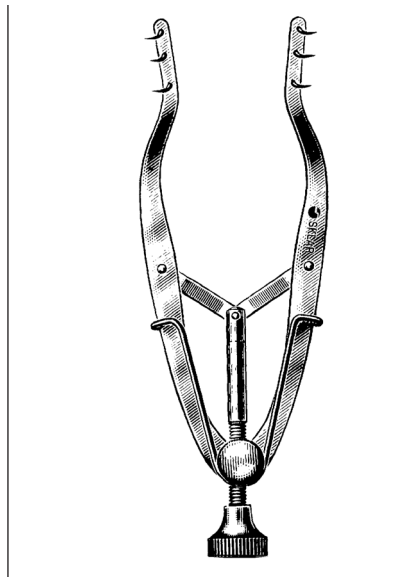

STEVENSON RETRACTOR
65-1831 3¼"

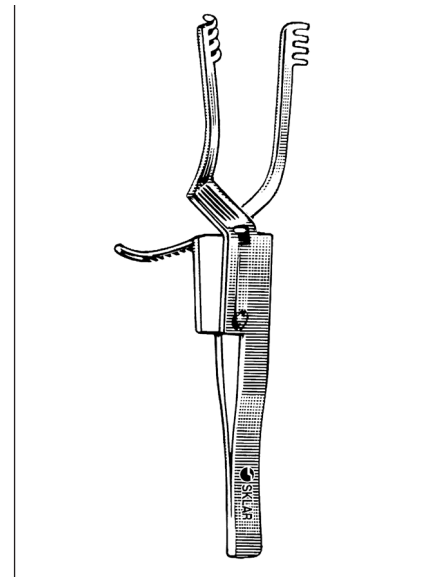

HOLZHEIMER RETRACTORS
22-8920 4" Sharp 4x4 Teeth
22-8940 4" Blunt 4x4 Teeth

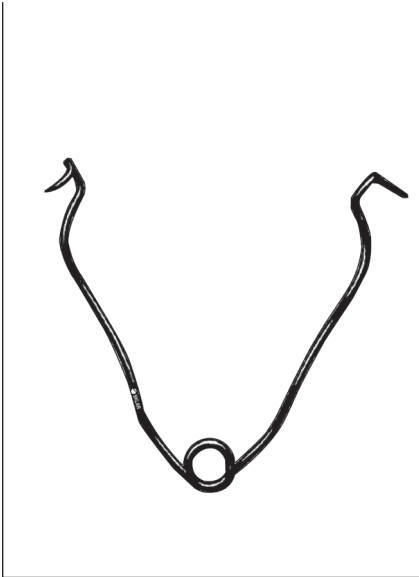

FARR RETRACTORS

SHARP TIPS

22-8835	1½"	22-8838	2½"
22-8836	2"	22-8839	3"
22-8837	2¼"		


FLAT TIPS

22-8842	2¼"		
---------	-----	--	--




SCAPENS FORKED ORBITAL RETRACTOR

65-1870	Flat Handle		
---------	-------------	--	--

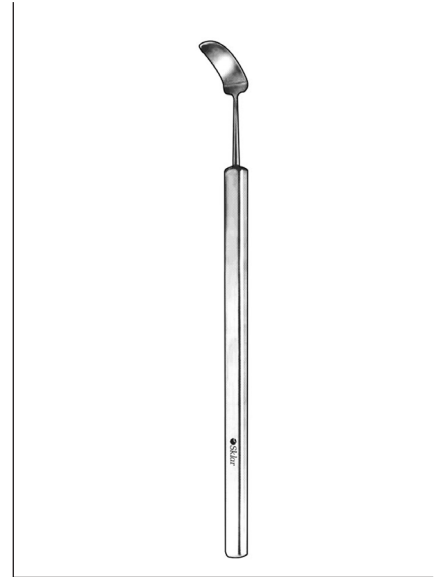

HELVESTON TISSUE RETRACTORS


65-1873	7mm	Barbie	
65-1874	9mm	Big Barbie	
65-1875	11mm	Great Big Barbie	

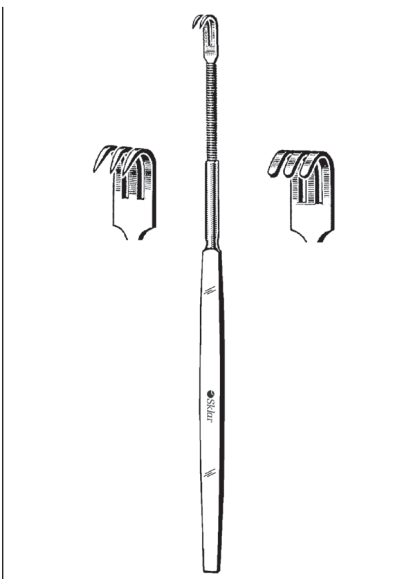

PLASTIC SURGERY RETRACTORS

FLEXIBLE SHANK


22-9143	6"	Blunt, 3 Prongs	
22-9133	6"	Sharp, 3 Prongs	

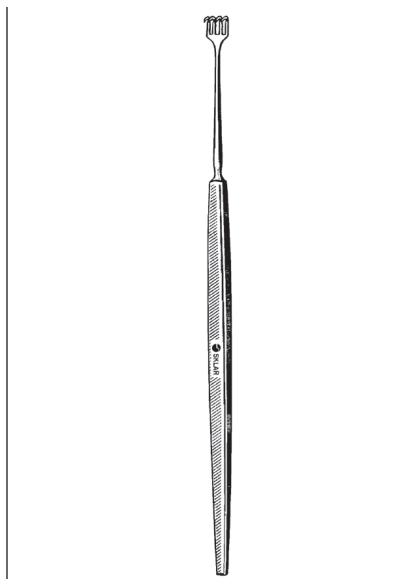

BLAIR-ROLLET RETRACTORS

DELICATE


47-2655	5½"	Sharp, 4 Prongs	
47-2752	5½"	Blunt, 4 Prongs	

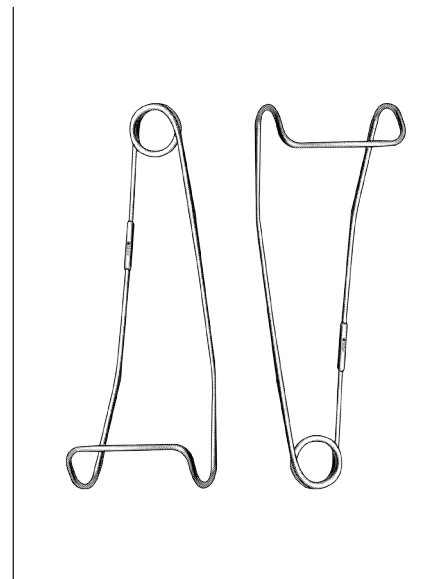

JAFFE LID RETRACTORS


WIRE

65-1871	14x7mm	Pair	Large
65-1872	10x7mm	Pair	Small

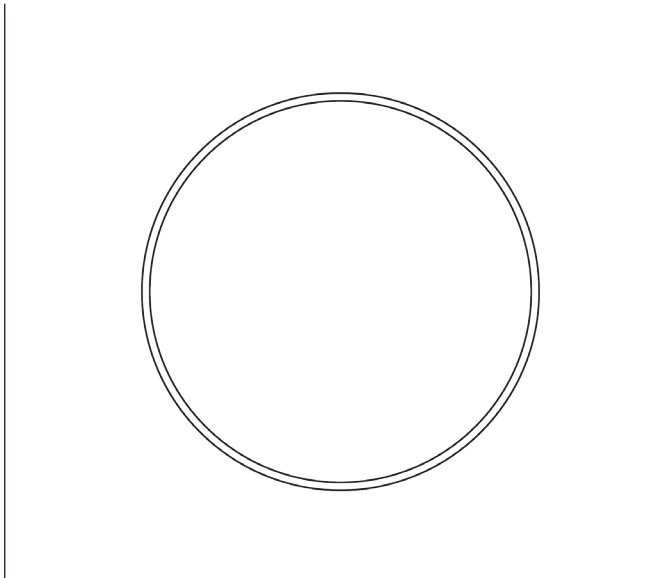

FLIERINGA FIXATION RINGS

- 66-7814 14mm Diameter
- 66-7816 16mm Diameter
- 66-7818 18mm Diameter
- 66-7820 20mm Diameter
- 66-7822 22mm Diameter
- 66-7824 24mm Diameter

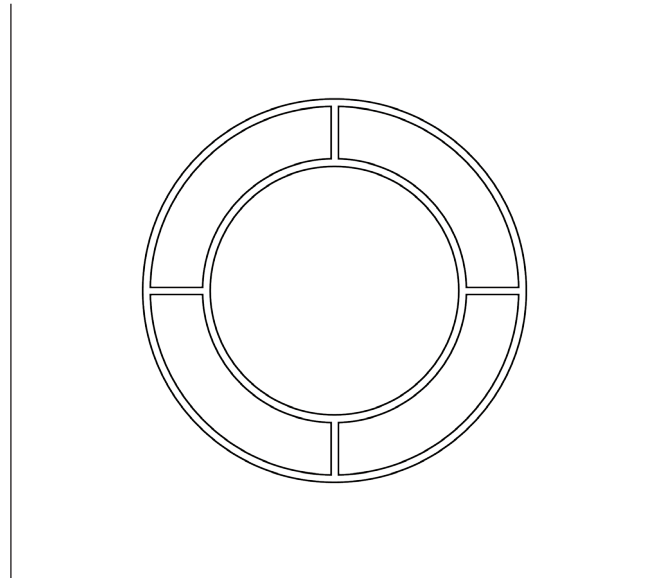


FLIERINGA-LEGRAND FIXATION RING


- 66-7850 17x24mm Diameter

FOX ALUMINUM EYE SHIELDS


Sklar's Fox Eye Shield is a perforated aluminum shield used to cover and protect an injured or postoperative eye. This product is a malleable, oval cupped shape device slotted on the outer edges so it can be fitted to various face/eye sizes. The shield also has a variety of holes around the shield to provide air circulation and ventilation.

SKU	Unit of Measure
65-7634	12/Pk
65-7635	1/Pk

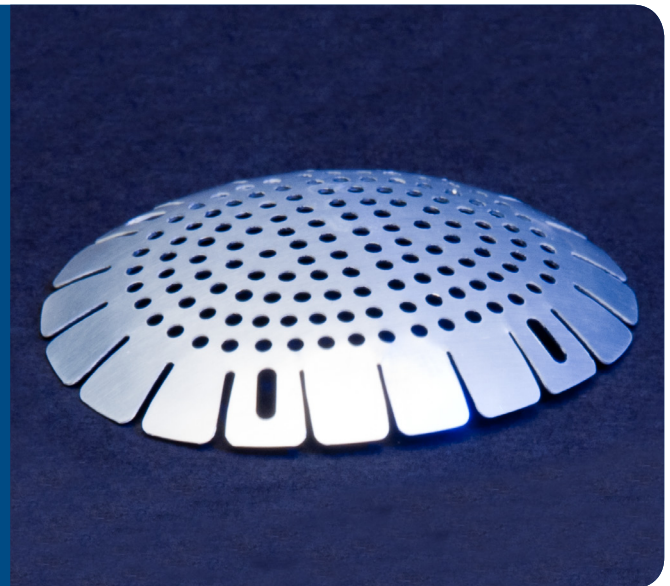


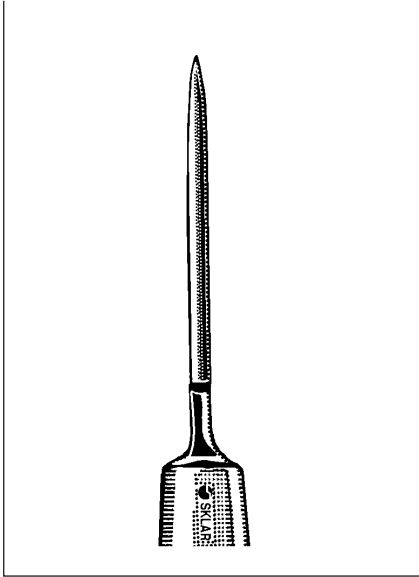


GRAEFE CATARACT KNIFE
65-3001 Small Blade 1.7x27mm

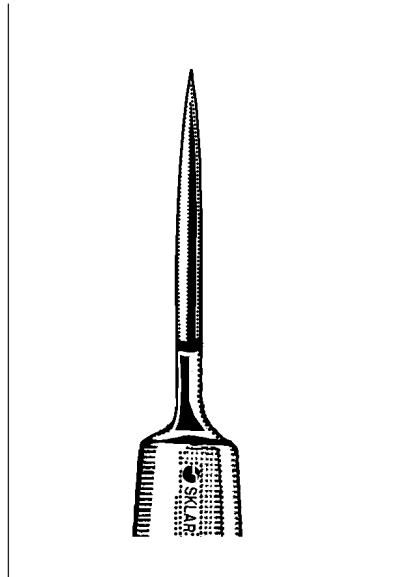


WHEELER DISCISSION KNIFE
65-3010 1mm

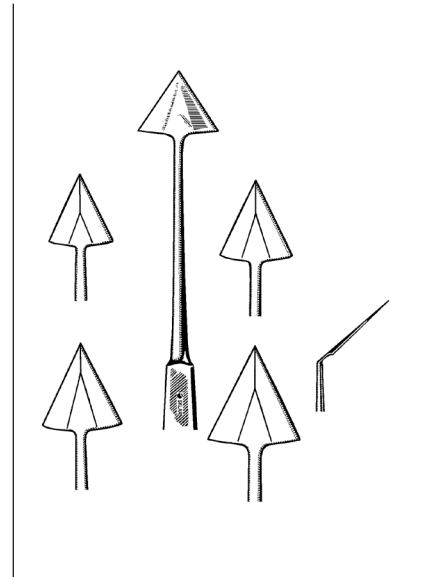


GRIESHABER TYPE KERATOMES
ANGULAR BLADE WITH BEVELED BACK
65-3050 Extra Small Size 0
65-3051 Small Size 1
65-3052 Medium Size 2
65-3053 Large Size 3

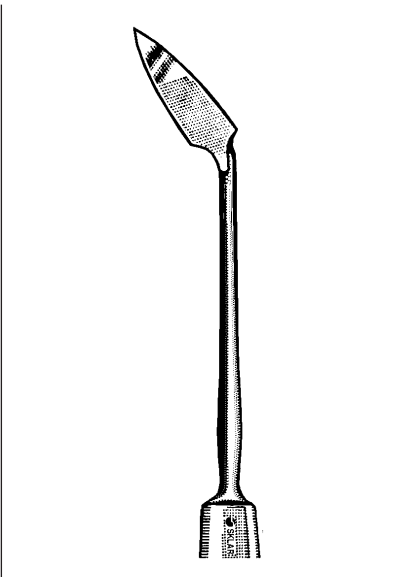


CASTROVIEJO KERATOMES
65-3030 Angled

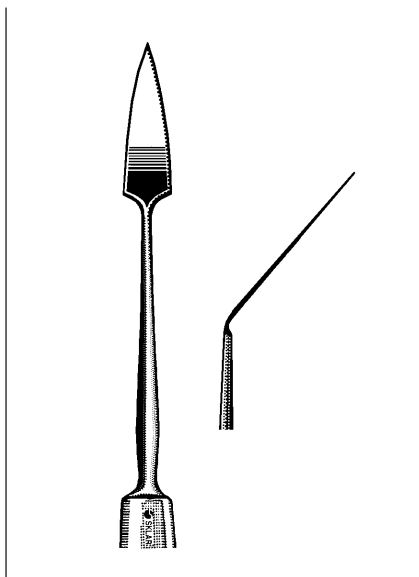

BERENS KERATOME
65-3060 19x3.25mm

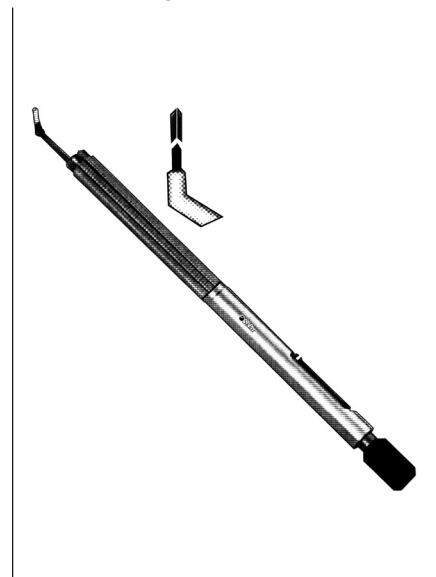


DIAMOND KERATOME
65-3044 2.8mm Angled
65-3046 3mm Angled
65-3048 3.2mm Angled

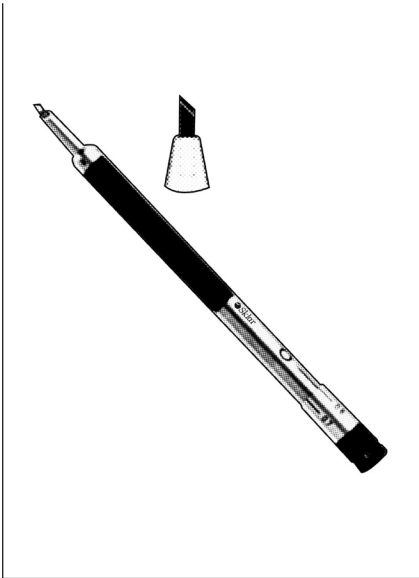

DIAMOND STEP KNIFE
66-8025 Single Edge 1mm




DIAMOND KNIFE
66-8026 Tri-Facet 1mm

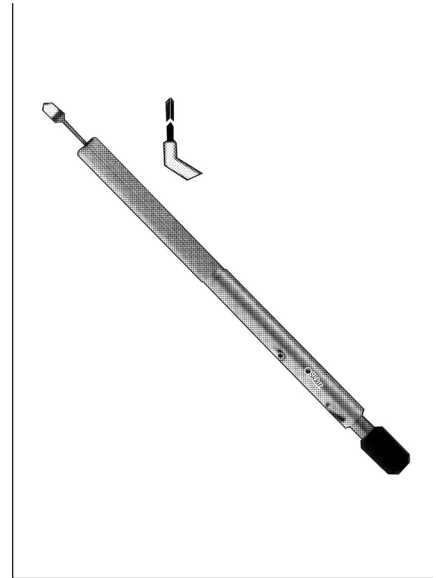


PHACO DIAMOND KNIFE
66-8027 2.5mm Angled
66-8028 2.8mm Angled
66-8029 3mm Angled
66-8030 3.2mm Angled




PHACO DIAMOND KNIFE
66-8031 Trapezoid

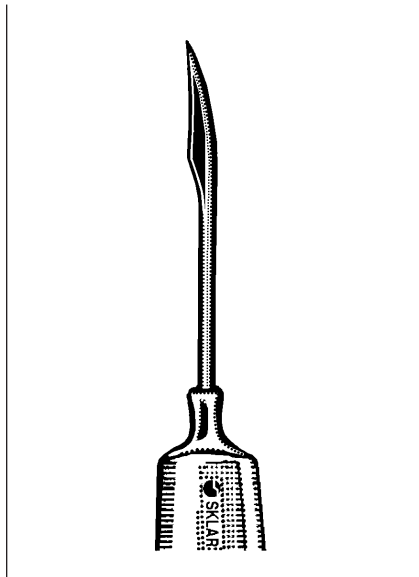


ZIEGLER KNIFE-NEEDLE
ANGULAR BLADE, 4½"
65-3094 Size 0 3mm Blade

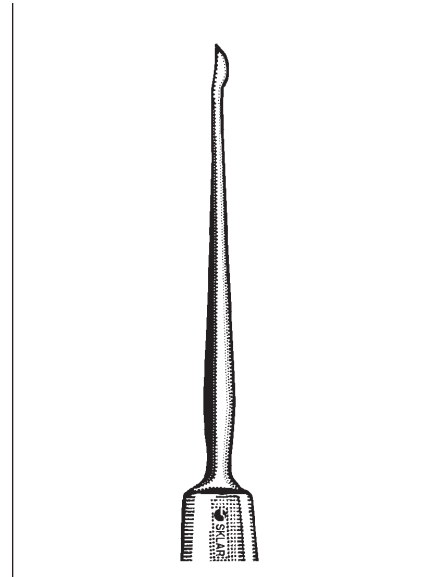


BARKAN GONIOTOMY KNIFE
65-3022 4½"

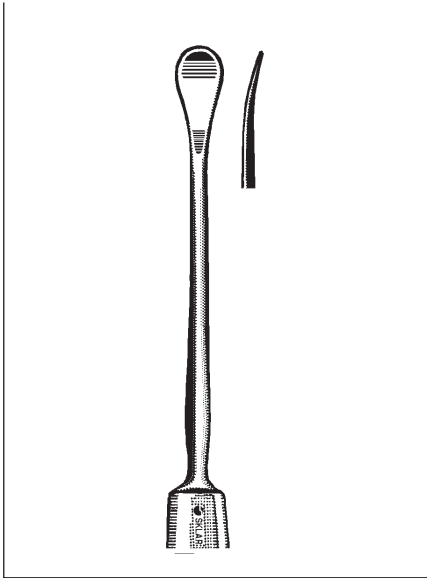


GILL CORNEAL KNIFE
65-3025 4¾" Blade is Slightly Curved on Flat

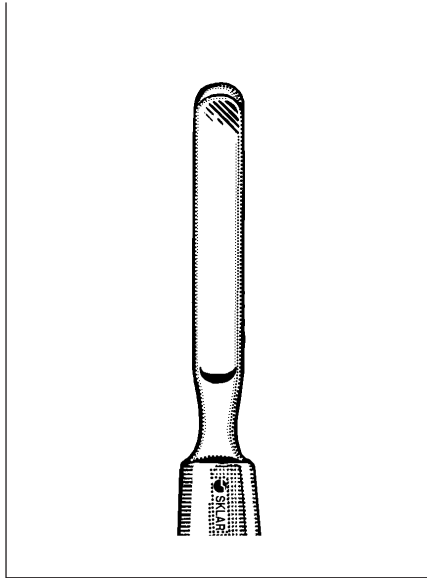

TOOKE CORNEAL KNIFE
65-3028 4½"

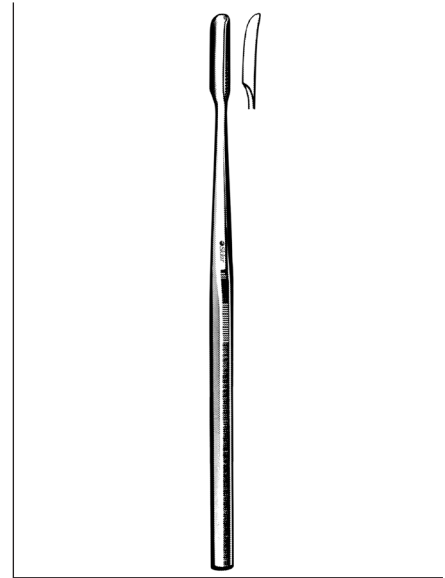


WEST GOUGE
40-6871 6¼" 4mm

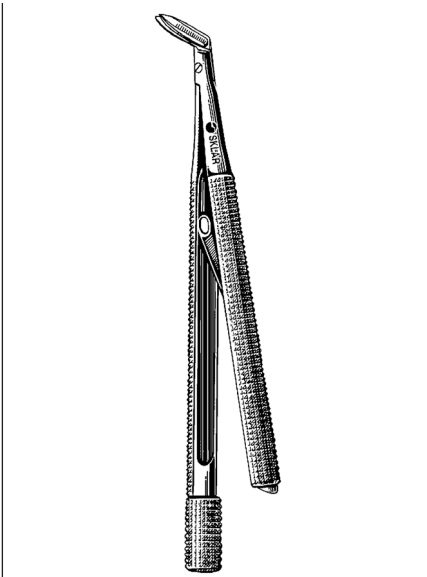

TROUTMAN BLADE BREAKER
65-2960 3½"



CASTROVIEJO BLADE BREAKER
65-2950 5"

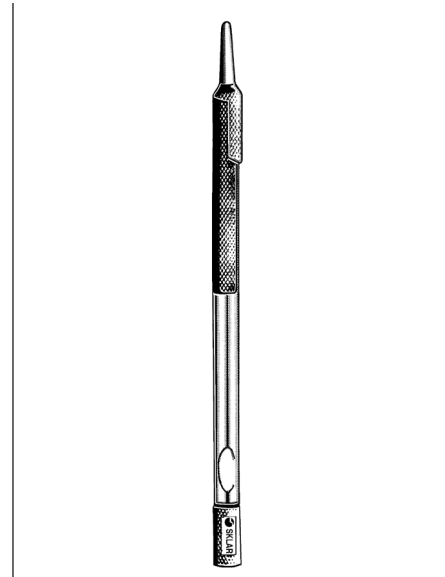


SWISS MODEL BLADE BREAKER
65-2958 4¾"



KERRISON RONGEURS

65-7400	3½" Shaft	3mm
65-7401	3½" Shaft	4mm
65-7402	3½" Shaft	5mm
65-7403	3½" Shaft	6mm

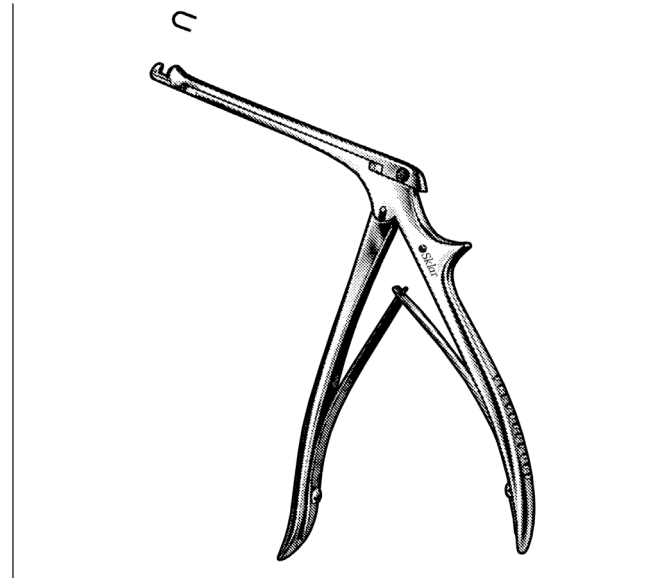

BEYER MASTOID PUNCH RONGEUR

65-7408	3½" Shaft	2mm
---------	-----------	-----




CITELLI RONGEURS

65-7409	2½" Shaft	1mm
65-7410	2½" Shaft	2mm
65-7411	2½" Shaft	3mm
65-7412	2½" Shaft	4mm



MEYHOEFER CHALAZION CURETTES

- 65-3173 #3 2.5mm
- 65-3174 #4 3mm

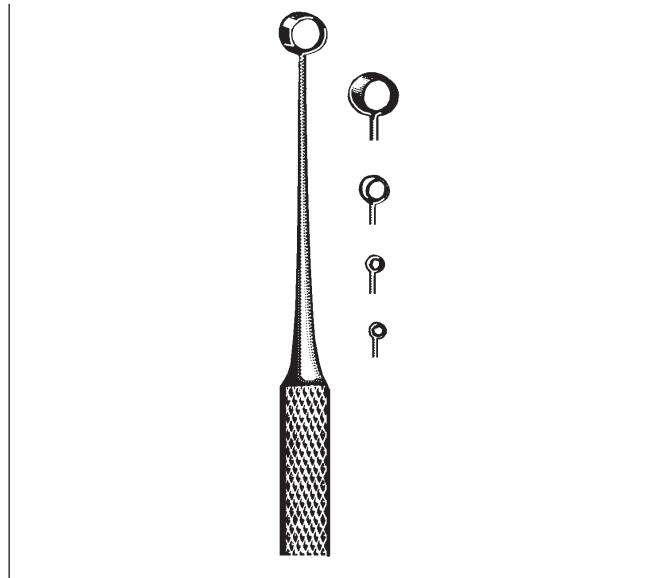


HEATH CHALAZION CURETTES
ONE SHARP EDGE AND ONE DULL EDGE

- 65-3180 #0 .5mm
- 65-3181 #1 1mm
- 65-3182 #2 2mm
- 65-3183 #3 3mm




SKEELE CHALAZION CURETTE

- 65-3162 Serrated, 2mm Blade, Stainless steel

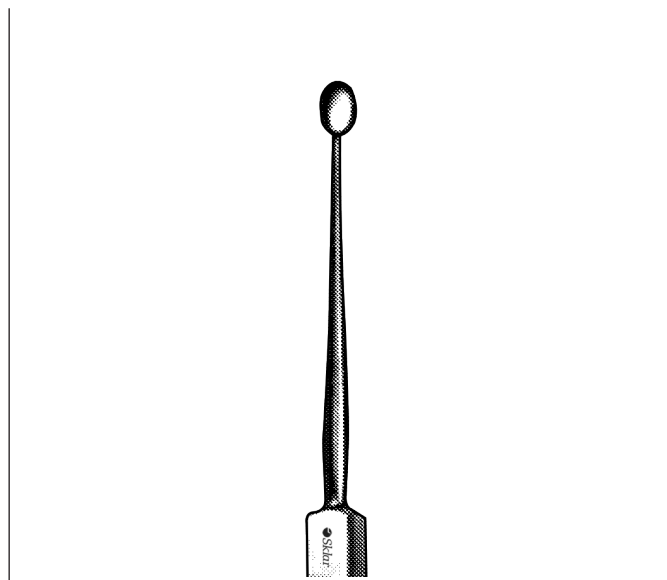


HEBRA CHALAZION CURETTES

- 65-3187 2.5mm
- 65-3188 3mm
- 65-3189 3.5mm

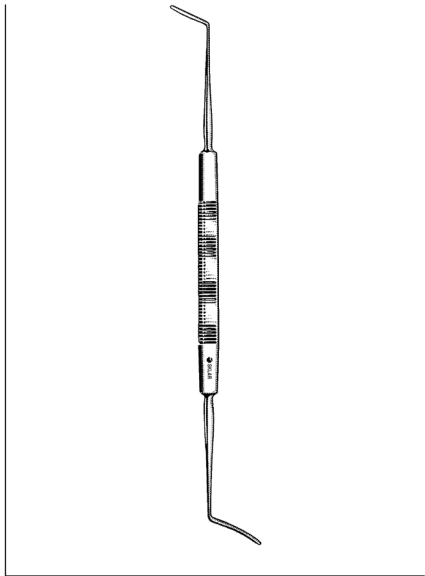


**CASTROVIEJO
CYCLODIALYSIS SPATULAS**

10 AND 15MM LENGTHS
65-4062 .5mm Wide
65-4064 .75mm Wide
65-4066 1mm Wide

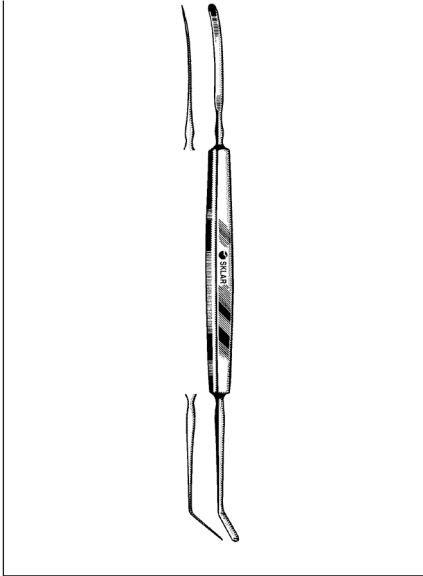


**CASTROVIEJO
CYCLODIALYSIS SPATULAS**

65-4070 5¼" Blade 10x1mm
65-4072 5¼" Blade 15x1mm

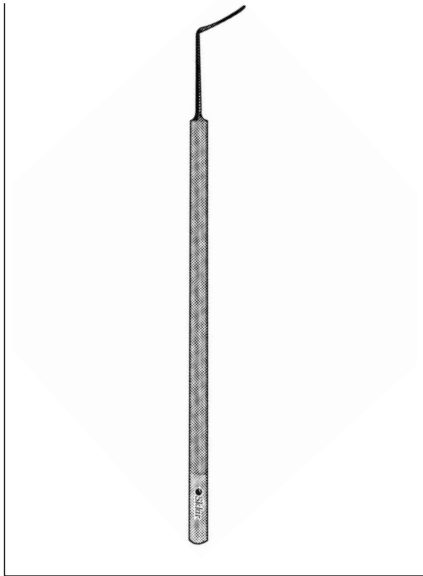


**BARRAQUER
CYCLODIALYSIS SPATULAS**

65-4044 .25x10mm Diameter
65-4045 .5x10mm Diameter

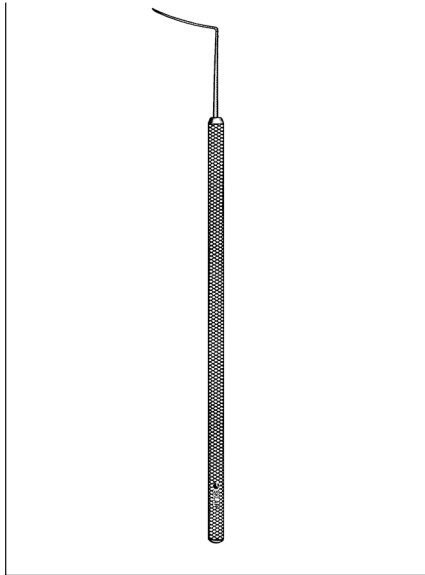


**BARRAQUER
CYCLODIALYSIS SPATULA**

65-4068 4¾" .5mm Diameter Round
15mm Angled Blade,
Smooth Pointed Tip

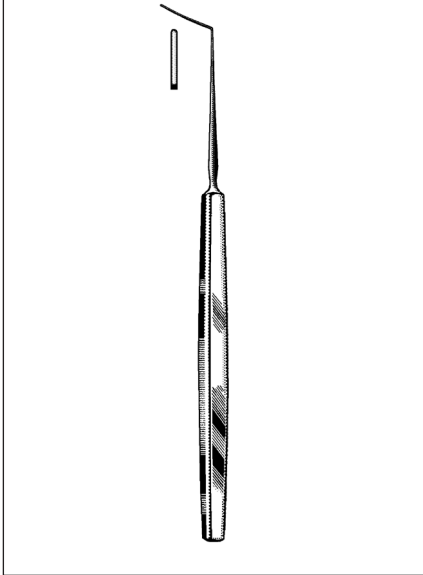

**ELSCHMIG CYCLODIALYSIS
SPATULA**

65-4050 5" 10x1mm

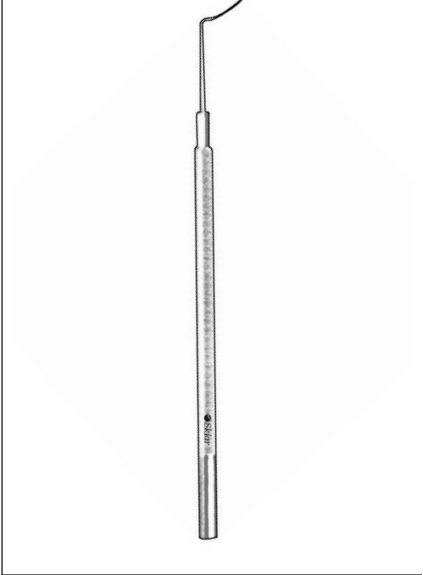


GIRARD SYNECHIAE SPATULA


65-4042 4½" .55mm Wide
with Semi-Sharp Tip




JAFFE LENS SPATULA
FLAT
 65-4049 .5mm Wide

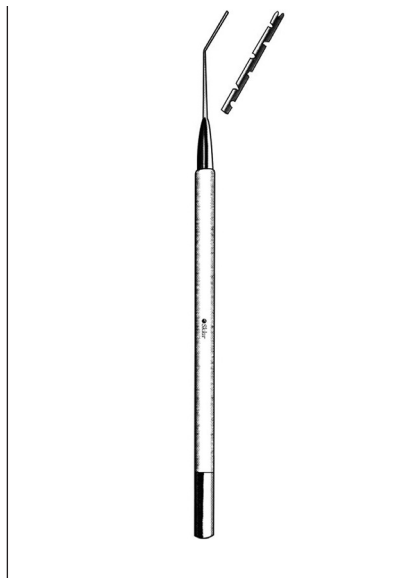


HIRSCHMAN LENS SPATULA
WITH SMALL NOTCHES
 65-4046 .5mm

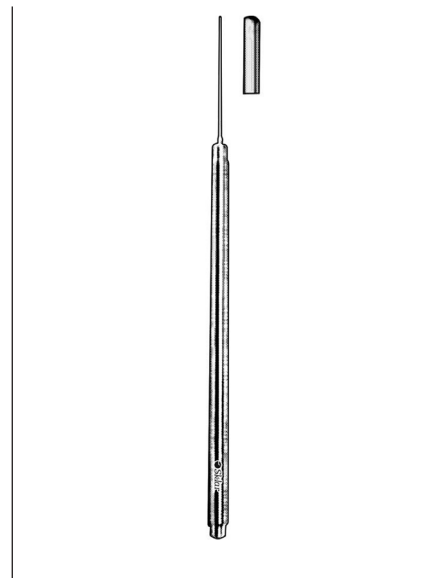


KNOLLE LENS NUCLEUS SPATULA
FLAT, MALLEABLE
 65-4052 .5mm

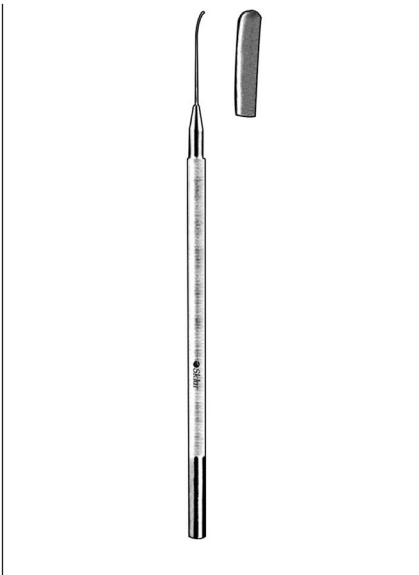

KOCH LENS NUCLEUS SPATULAS
GENTLY CURVED DUCKBILL TIP
 65-4040
GENTLY CURVED DUCKBILL TIP WITH NOTCHES
 65-4041

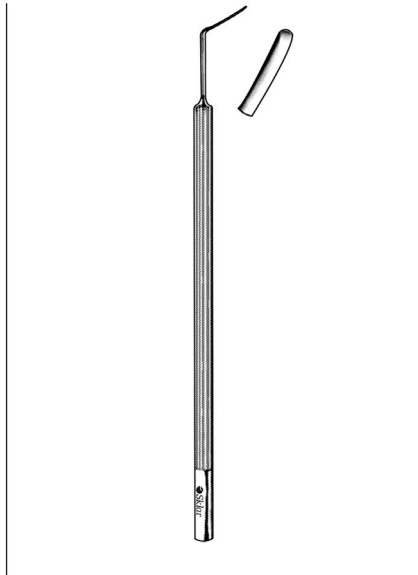


FUKASAKA LASIK SPATULA
 65-4035 Rounded Anterior and Flat Posterior Surface

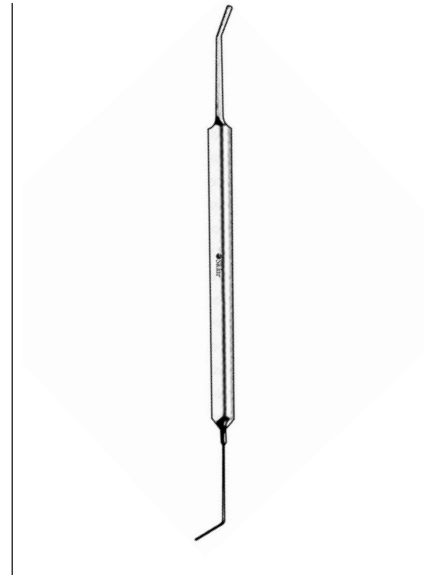


MADDOX LASIK SPATULA
FLAT & CYLINDRICAL
 65-4043 4½" .5mm Wide



GREEN SPATULA
SLIGHTLY CURVED
65-4074 1mm Wide

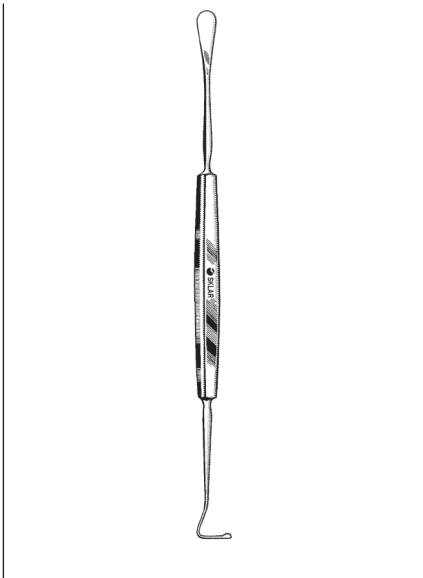

MCREYNOLDS SPATULA AND HOOK
65-4076 5 3/4"

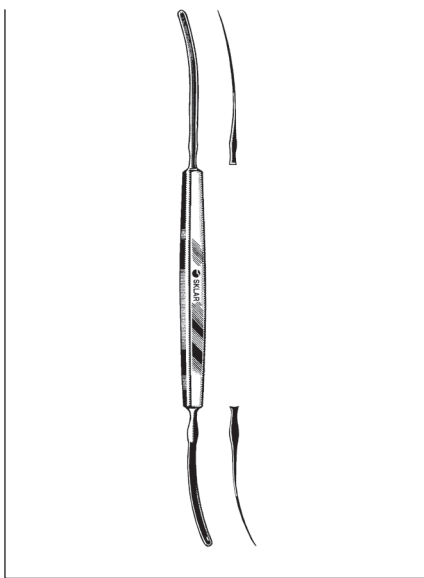


SMITH-FISHER IRIS SPATULA
ONE FLAT END, ONE SPOON END
65-4078 5"



CULLER IRIS SPATULAS
65-4047 1mm Wide
65-4048 2mm Wide




GIMBLE LENS NUCLEUS SPATULA
ANGLED 30 DEGREES
65-4038 Round, Smooth

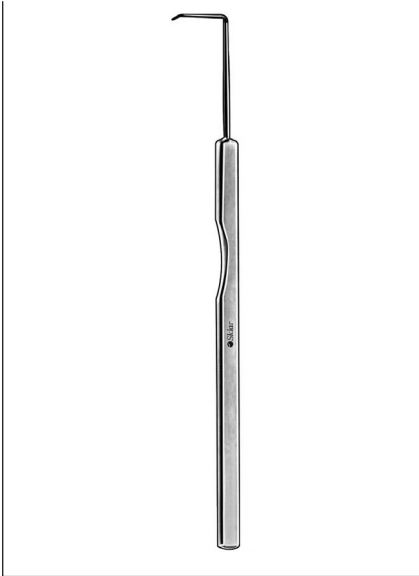


HELVESTON HOOKS

65-3236 8mm Small
65-3235 12mm Large

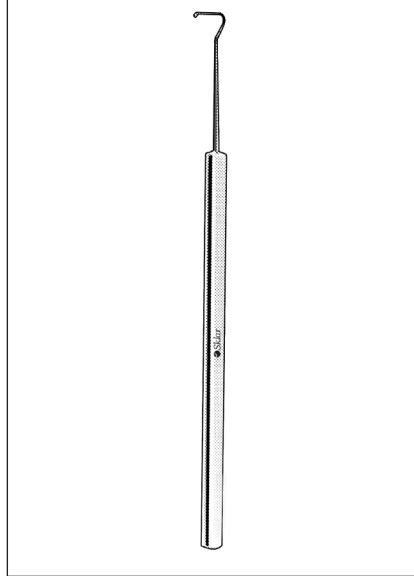


HELVESTON STRABISMUS HOOKS

65-3201 8mm
65-3202 10mm
65-3203 12mm

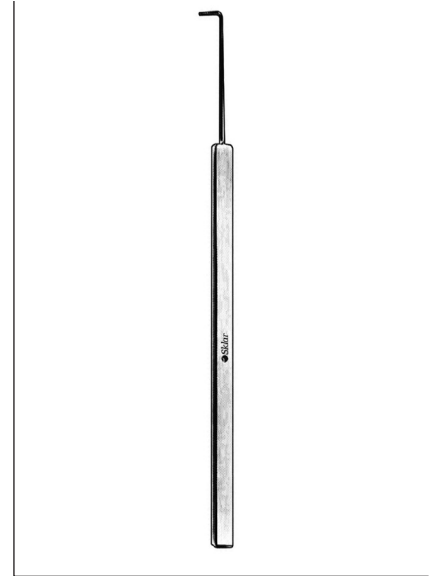


HELVESTON TEASER HOOK

65-3210 6mm Extra Delicate

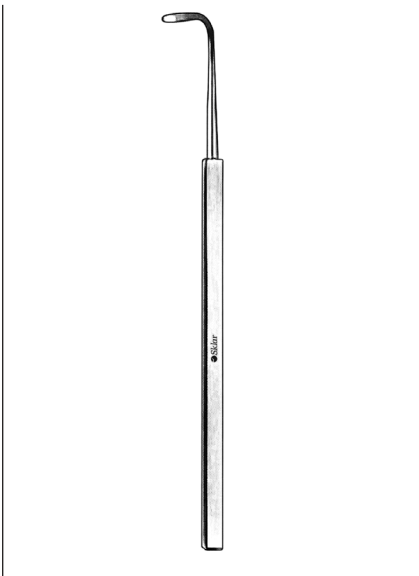

GASS RETINAL DETACHMENT HOOK

65-3533

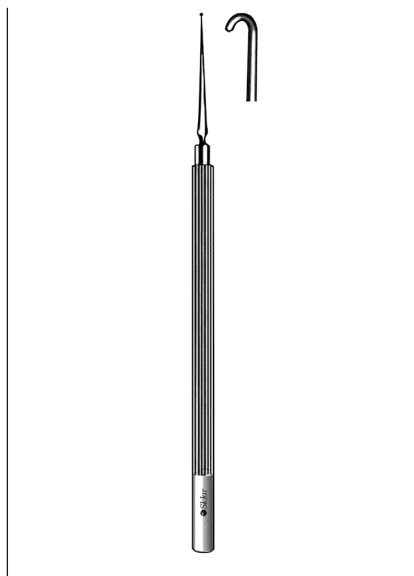


BONN HOOK ROUND HANDLE

66-7755 Extra Delicate




CRAWFORD RETRIEVING HOOK

65-3251 8¼"

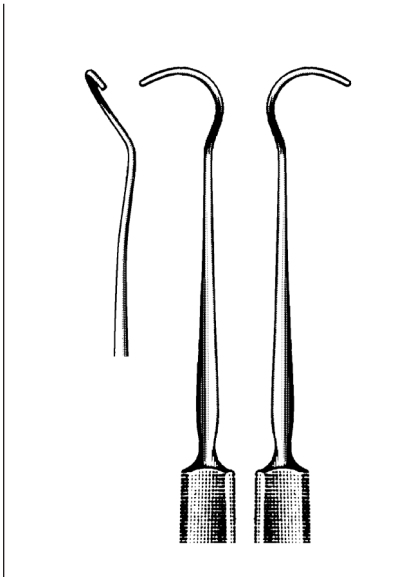


ARRUGA LENS EXPRESSORS

65-8350 Left
65-8352 Right

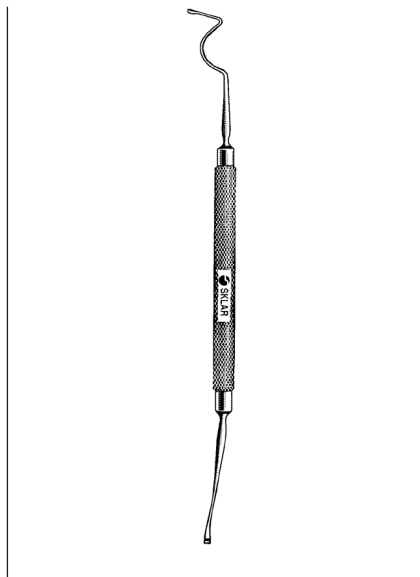


RIZZUTI IRIS RETRACTOR & LENS EXPRESSOR

5 3/4"
65-8354 2.5mm Blade

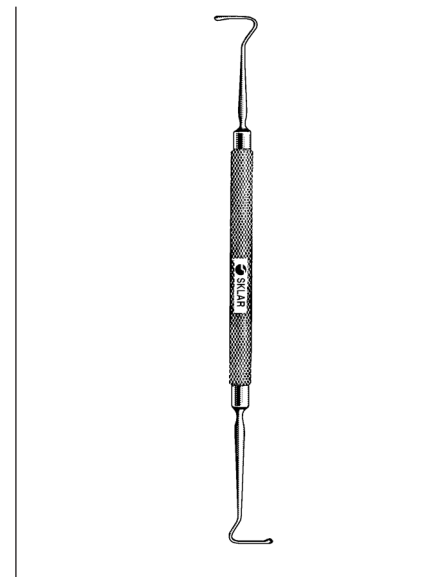


KIRBY MUSCLE HOOK & LENS EXPRESSOR

5 3/4"
65-9085 Double Ended
10x2mm Hooks

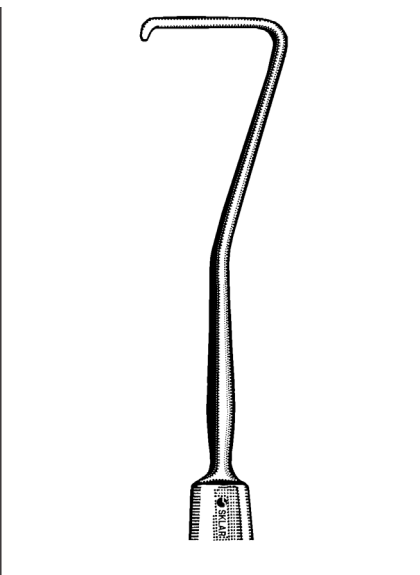


GREEN STRABISMUS HOOK

65-3200 5" 7mm

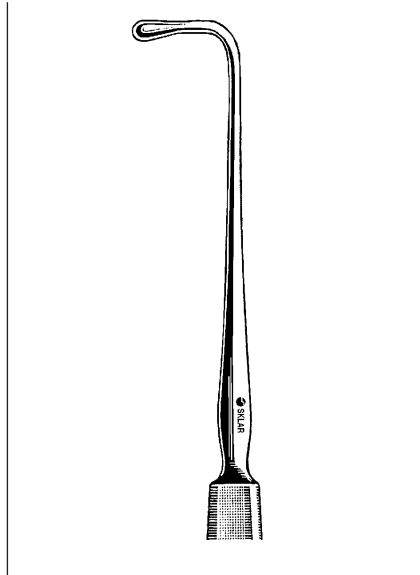


GRAEFE STRABISMUS HOOKS

65-3215 5 1/4" Small
65-3216 5 1/2" Medium
65-3217 5 3/4" Large

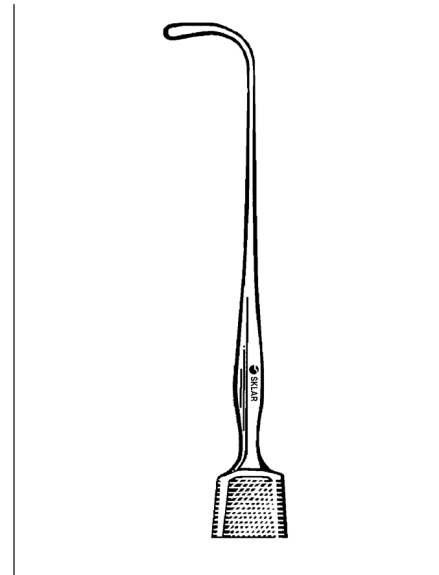


STEVENS TENOTOMY HOOK


65-3191 4 3/4"

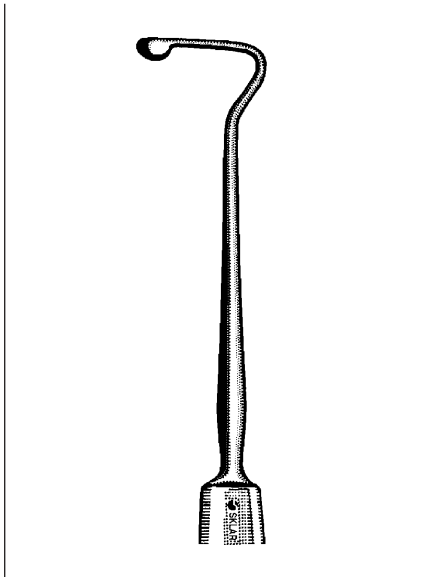


JAMISON MUSCLE HOOK
65-3195 5" 10mm

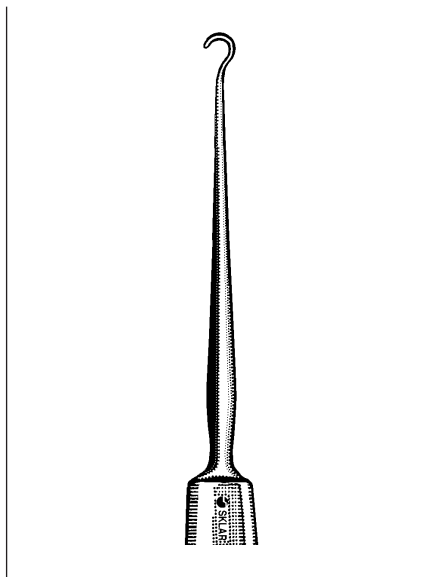


TYRELL IRIS HOOKS
65-3205 5" Blunt
65-3206 5" Sharp

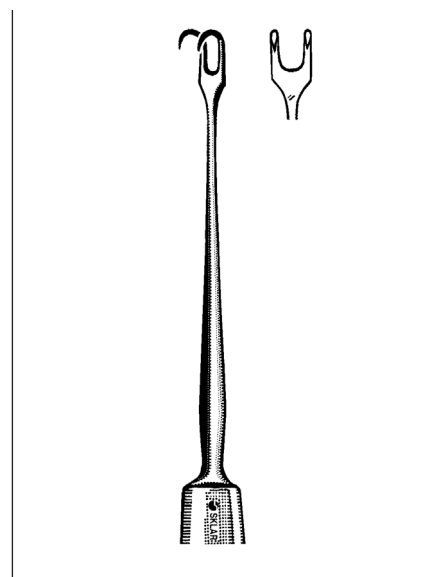


GUTHRIE FIXATION HOOKS
47-2650 5" Two Large Prongs
47-2651 5" Two Small Prongs

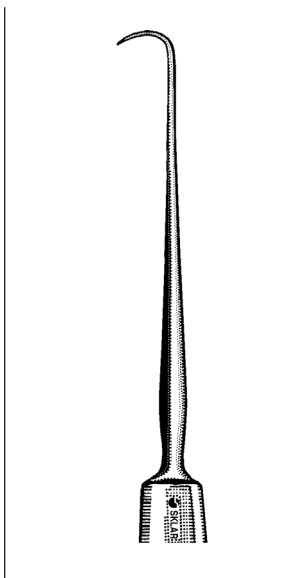


WIENER CORNEAL HOOKS
65-3250 5" Sharp
65-3248 5" Blunt

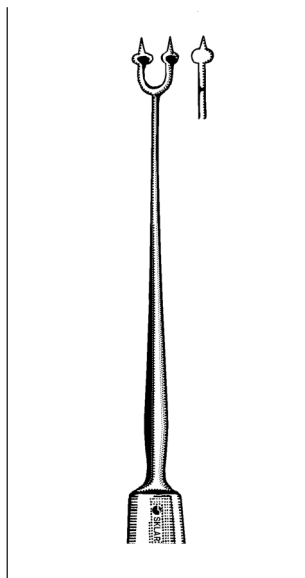


BURCH FIXATION PICK
65-3255 5¼" Stainless
Two Prongs with Stops to Limit Depth of Penetration

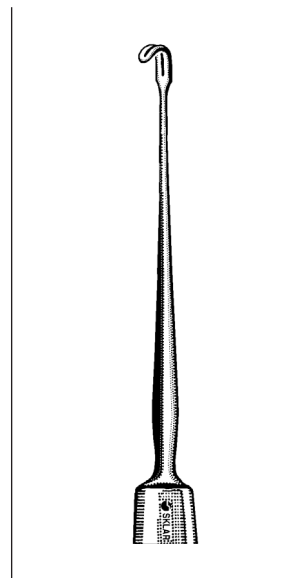

MCGANNON IRIS RETRACTOR
65-3258 5¼"

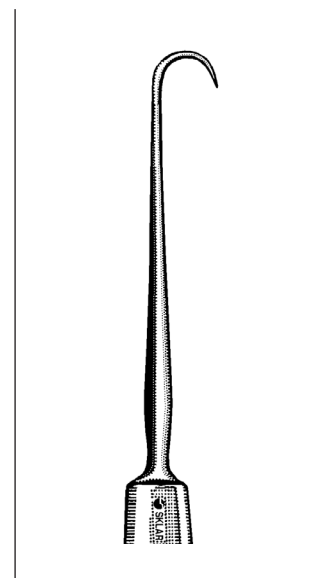

O'CONNOR HOOK
65-3265 5" Sharp

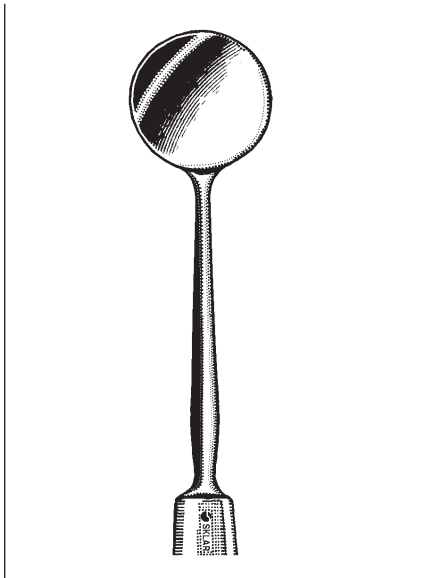

BUNGE EVISCERATION SPOONS
65-8151 Small
65-8155 Large



WELLS ENUCLEATION SPOON
65-8159




DAVIEL CHALAZION CURETTE
65-3140 5" 3mm spoon



CULLER LENS SPOON
65-8157



KNAPP LENS SCOOP
65-8195 5" 5mm spoon

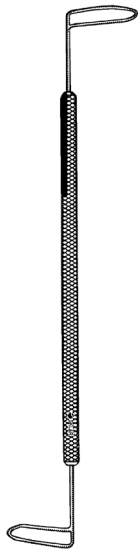

SIMCOE NUCLEUS LOOP

65-3295 5¼" Double ended, serrated loops, 4x20mm



WILDER LENS LOOP

5¼"
65-8215 Serrated
65-8216 Smooth




LEWIS LENS LOOP

5¼"
65-8206 Smooth




NEW ORLEANS LENS LOOP

5¼"
65-8185 5mm loop

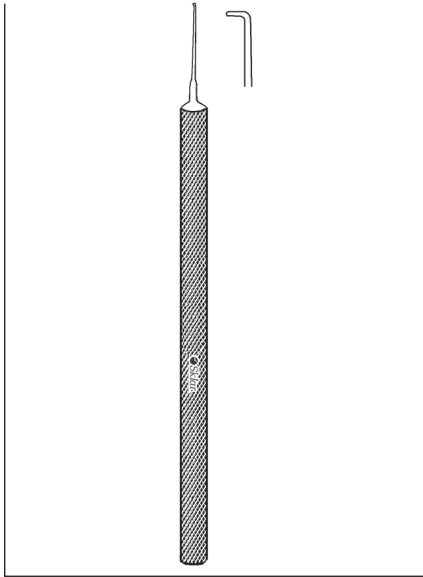


AZAR LENS MANIPULATING HOOK

66-7700 .2mm Hook Malleable Shaft

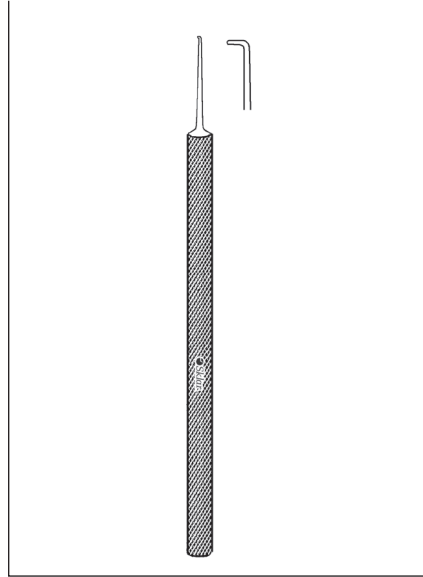


SINSKEY MICRO IRIS HOOKS

BLUNT
66-7710 .25mm Straight
66-7712 .25mm Angled 45°

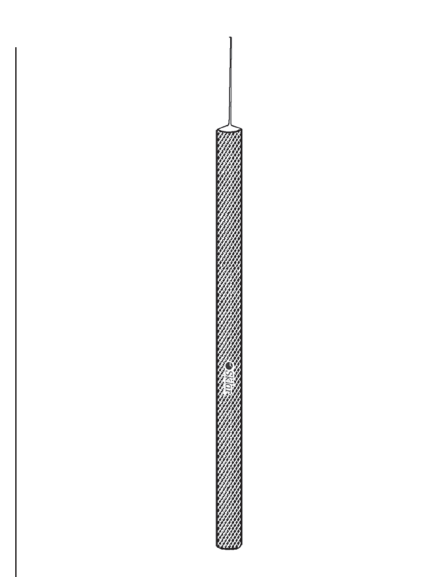


SINSKEY II & III LENS MANIPULATING HOOK

66-7720 .2mm Blunt Tip Straight

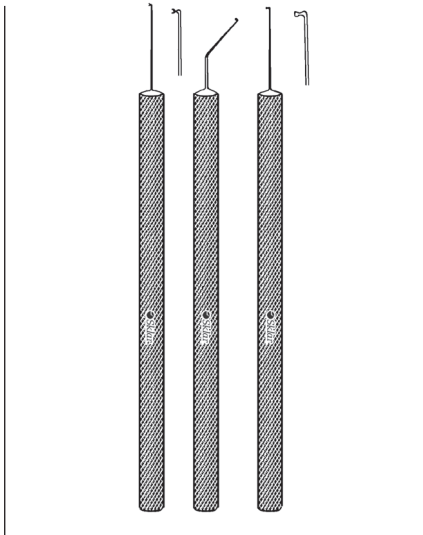


KUGLEN IRIS HOOKS & LENS MANIPULATOR PUSH PULLS

66-7725 Straight
66-7726 Angled
66-7727 Cloverleaf

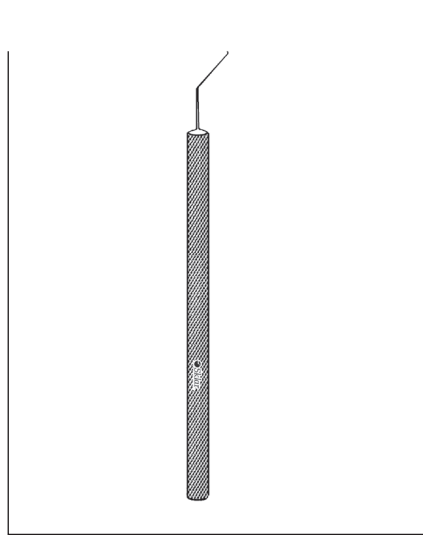


HARRIS-SINSKEY LENS MANIPULATING HOOK

BLUNT, ANGLED
66-7735

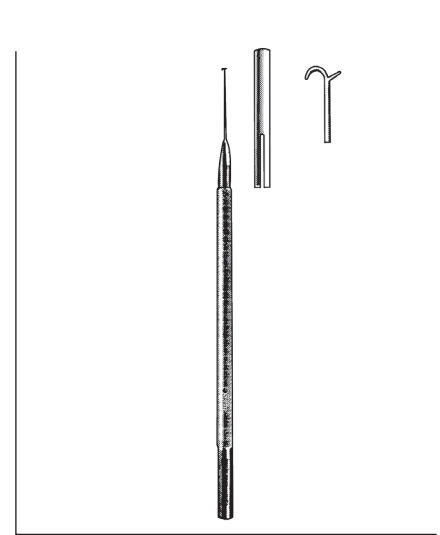


RENTSCH LENS MANIPULATOR HOOK


BOAT HOOK
66-7706 Straight with Guard

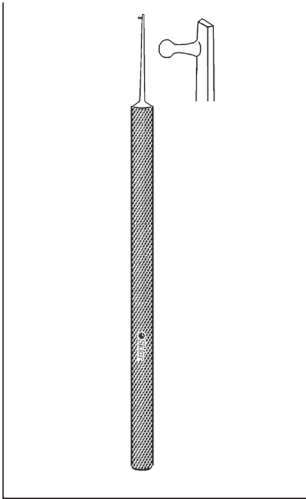


LESTER II LENS MANIPULATOR
66-7752 4½" Angled 45°

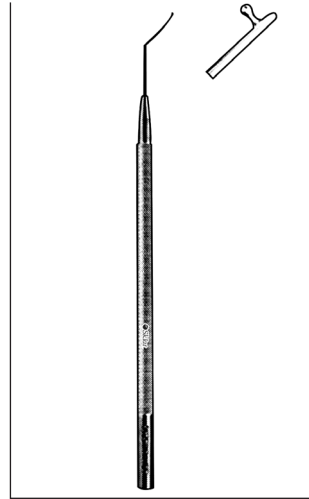


LESTER IOL MANIPULATOR
66-7744 Vaulted Shank

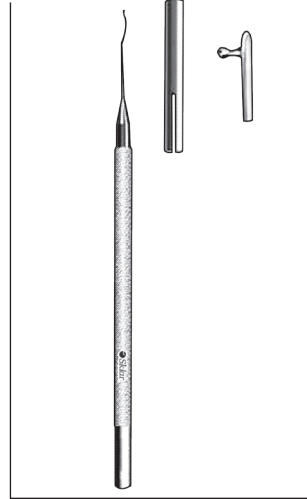


MALONEY IOL MANIPULATOR
66-7745 Vaulted Shank with Guard

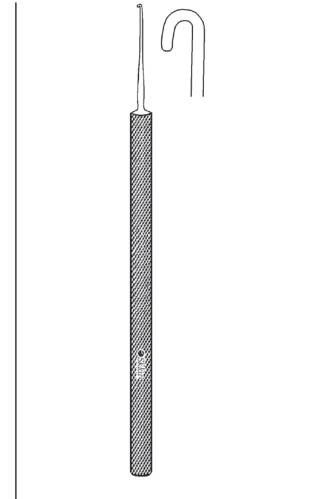


IRIS HOOK
66-7775 5" Blunt

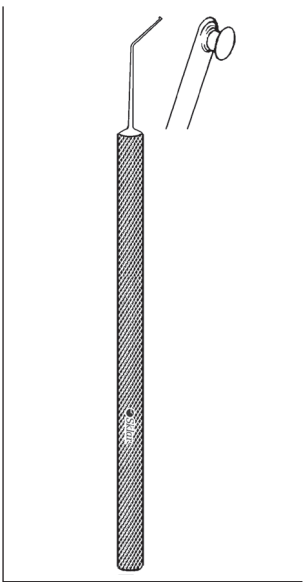

GRAETHER COLLAR BUTTON
66-7780 Angled 45°, 10mm from bend to tip

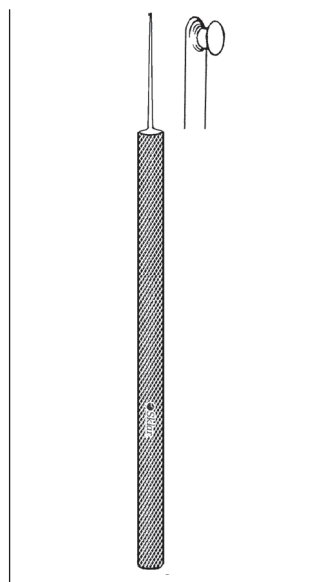


GRAETHER COLLAR BUTTON
66-7782 Straight

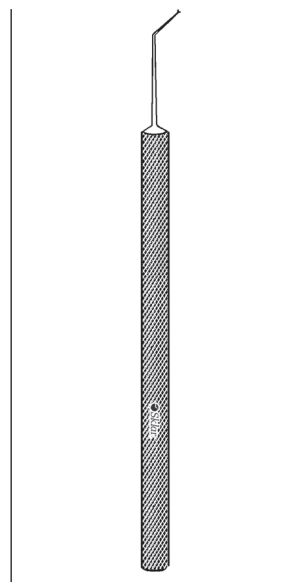


BECHERT NUCLEUS ROTATOR
66-7795 4¾" Rotator on front of shaft




NAGAHARA-KAGAWA PHACO CHOPPER
66-6510 4"

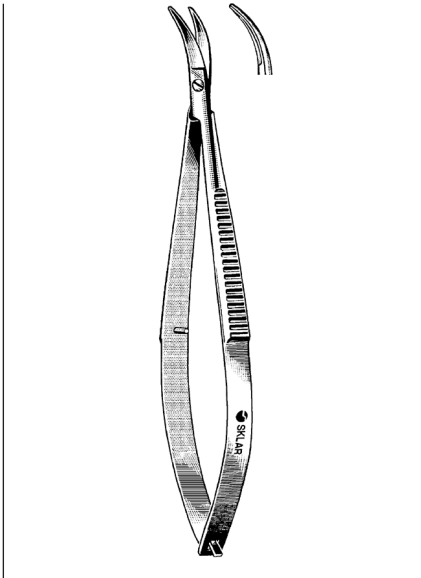


CASTROVIEJO CORNEAL SCISSORS
64-2442 4" Curved Blunt Blades
Spring Handles

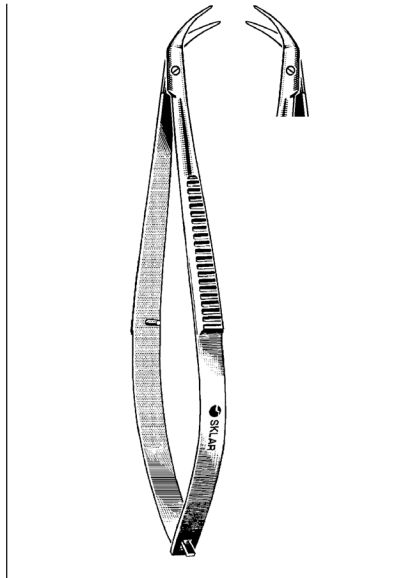


CASTROVIEJO CORNEAL SCISSORS
MINIATURE WITH INNER BLADE 0.5MM LONGER THAN OUTER BLADE
64-2242 4¼" Curved Right
64-2342 4¼" Curved Left

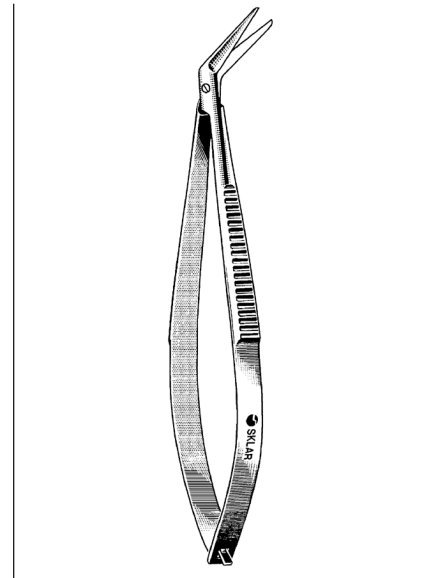

CASTROVIEJO CORNEAL SCISSORS
64-2535 4" 13mm Blunt Angled Blades

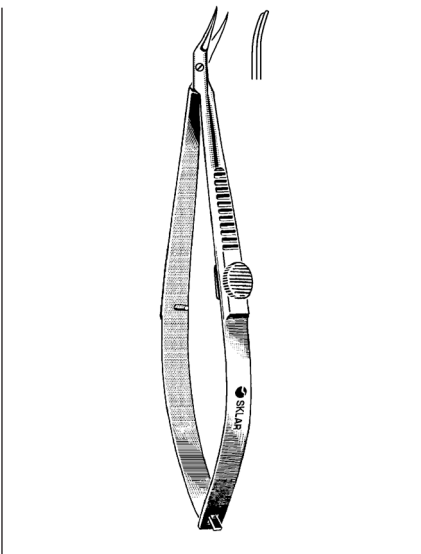


CASTROVIEJO CORNEAL SCISSORS
INNER BLADE LONGER THAN OUTER BLADE
64-3294 4" Right

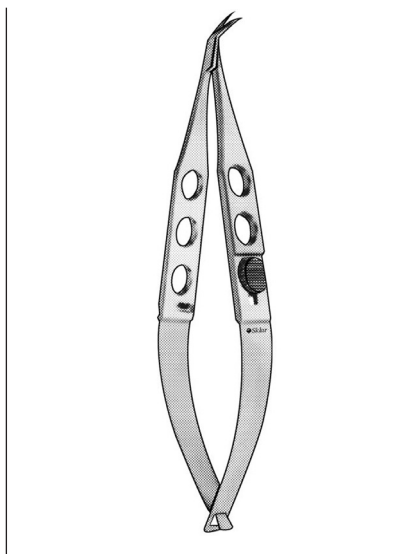

TROUTMAN-KATZIN CORNEAL TRANSPLANT SCISSORS
WITH STOP, SMALL BLADES COUNTER-CLOCKWISE
64-2230
WITH STOP, SMALL BLADES CLOCKWISE
64-2231




KATZIN CORNEAL TRANSPLANT SCISSORS
SMALL BLADES COUNTER-CLOCKWISE
64-2232
SMALL BLADES CLOCKWISE
64-2233

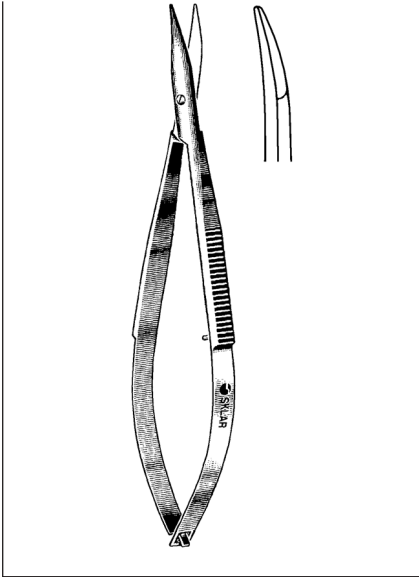


WESTCOTT TENOTOMY SCISSORS

64-3146 4¼" Blunt Tips with Spring Handle

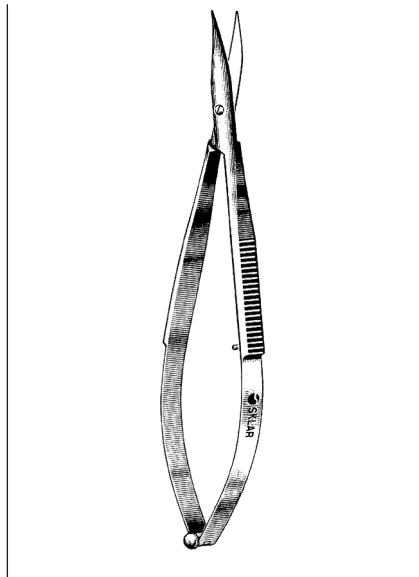


WESTCOTT TENOTOMY SCISSORS

CURVED, SPRING HANDLE
 64-3142 5" Right
 64-3144 5" Left

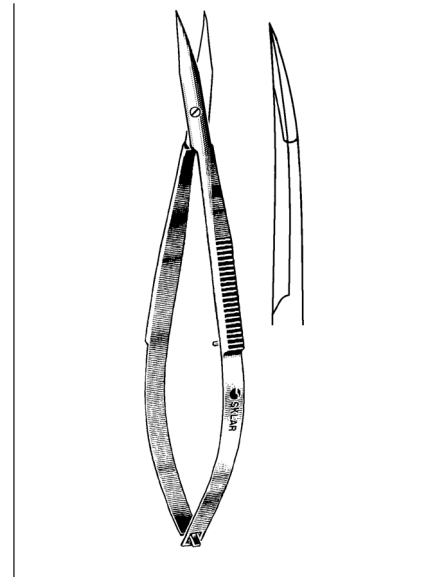

WESTCOTT SCISSORS

64-3145 4¼" Sharp Tips with Spring Handle

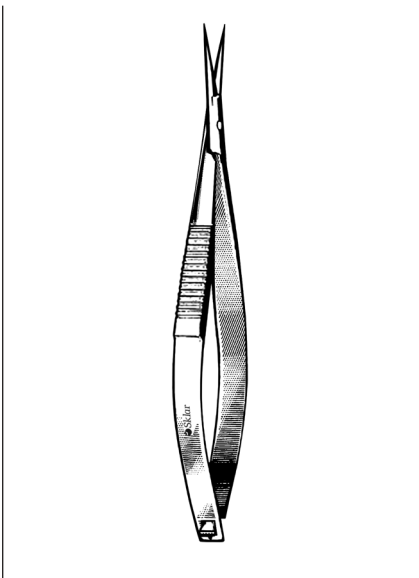


NOYES IRIS SCISSORS STRAIGHT

64-3210 4½" Blunt/Blunt
 64-3211 4½" Sharp/Blunt
 64-3212 4½" Sharp/Sharp


CURVED

64-3215 4½" Sharp/Sharp

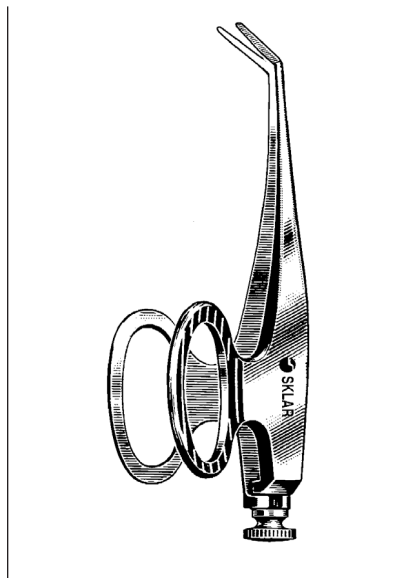


BARRAQUER-DEWECKER IRIS SCISSORS

64-3225 2¼" Fine 7mm Blades
 64-3228 2¼" Delicate 10mm Blades

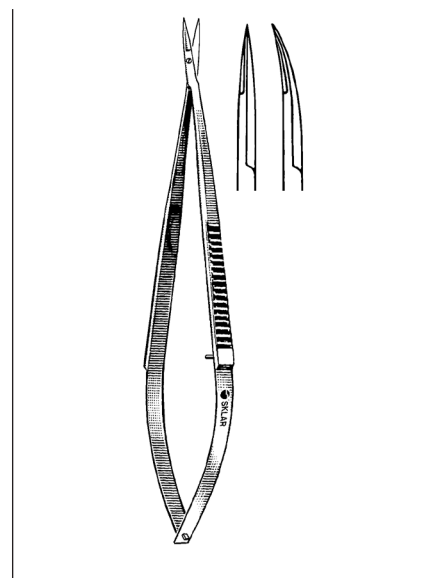

IRIS MICRO-MINIATURE SCISSORS


EXTRA DELICATE, SHARP POINTS
 64-3050 4" Straight
 64-3052 4" Curved

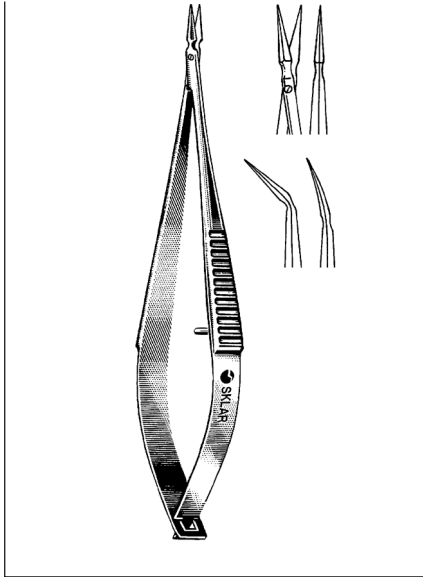

VANNAS SPRING HANDLE SCISSORS

VERY DELICATE


- 64-1950 3" Straight
- 64-1951 3" Curved
- 64-1952 3" Angular

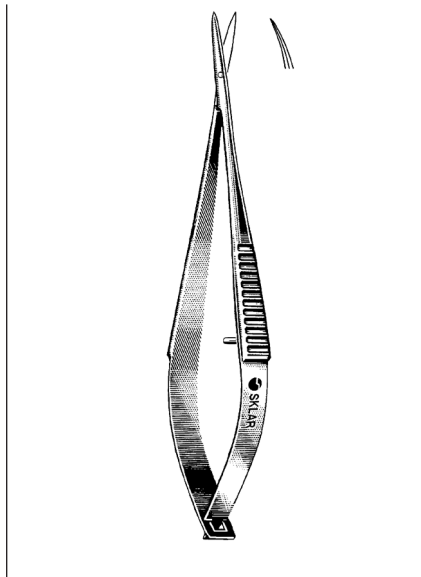

MCPHERSON-VANNAS SCISSORS


VERY DELICATE

- 64-1960 3" Straight
- 64-1961 3" Curved

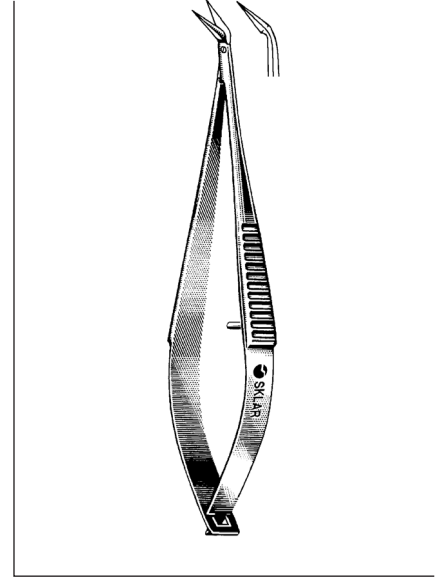


VANNAS SCISSORS
EXTREMELY DELICATE BLADES WITH SPRING HANDLE

- 64-1955 3" Angled on flat
- 64-1958 3" Angled to side

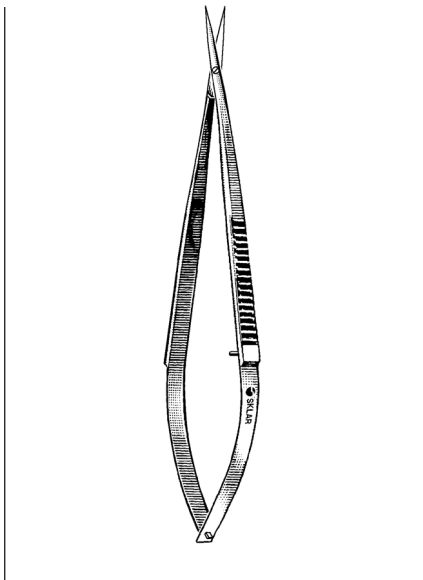

FINE STITCH SCISSORS

- 64-3260 4 1/4" Delicate Sharp Blades

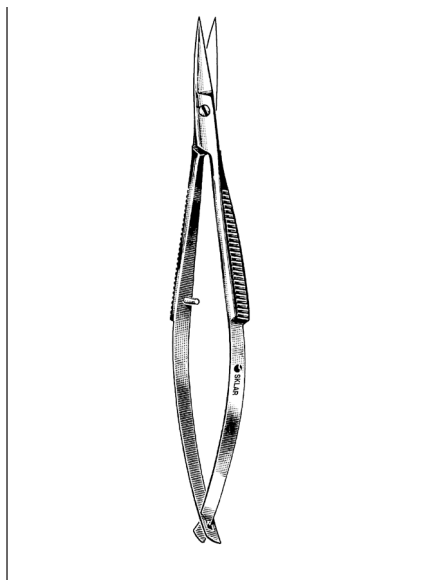

MCCLURE SPRING HANDLE SCISSORS

- 64-1742 4 1/4" Angled
- 64-1842 4 1/4" Straight
- 64-1942 4 1/4" Curved



VITRECTOMY SCISSORS

- STRAIGHT**
- 64-2235 20 GA Serrated
- ANGLED**
- 64-2236 20 GA 30 Degrees Serrated
- 64-2237 20 GA 60 Degrees Serrated
- 64-2238 20 GA 80 Degrees Serrated



STEVENS TENOTOMY SCISSORS

STRAIGHT			CURVED		
64-1242	4½"	B/B	64-1342	4½"	B/B
64-1243	4½"	S/S	64-1343	4½"	S/S



SKLARLITE™ STEVENS TENOTOMY SCISSORS

STRAIGHT			CURVED		
23-1156	4½"	B/B	23-1157	4½"	B/B




STEVENS TENOTOMY SCISSORS DELICATE

STRAIGHT			CURVED		
22-1242	4½"	B/B	22-1342	4½"	B/B
22-1240	4½"	S/S	22-1241	4½"	S/S




SKLARLITE XD™ STEVENS TENOTOMY SCISSORS

STRAIGHT			CURVED		
23-1160	4½"	B/B	23-1161	4½"	B/B

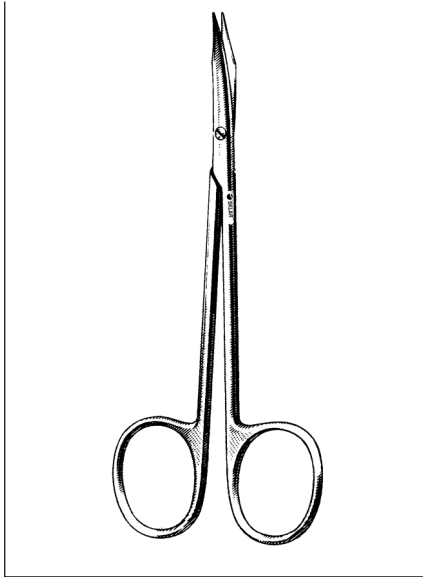


KLEINERT-KUTZ TENOTOMY SCISSORS

64-1250 5" Curved

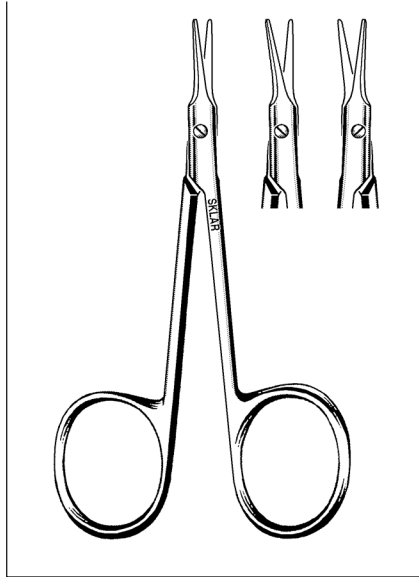

AEBLI CORNEAL SCISSORS

64-3270 4" Straight

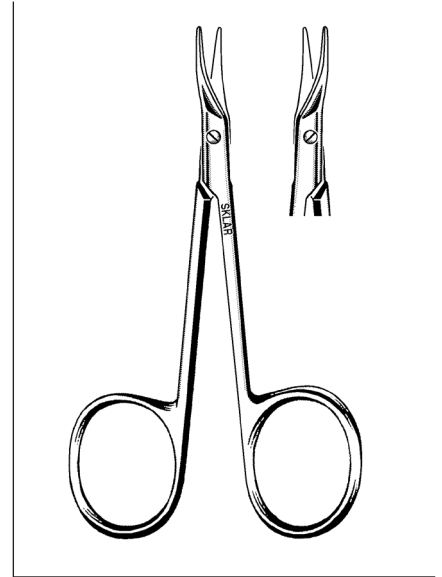

MCGUIRE CORNEAL SCISSORS

64-3297 4¼" 16mm delicate
Sharp Blades Right
64-3298 4¼" 6mm Delicate
Sharp Blades Left




IRIS SCISSORS PROBE POINT

64-2042 4¼" Angled



KNAPP IRIS SCISSORS

CURVED
64-3230 4" Sharp/Blunt
64-3235 4" Blunt/Blunt
STRAIGHT
64-3430 4" Sharp/Blunt
64-3435 4" Blunt/Blunt

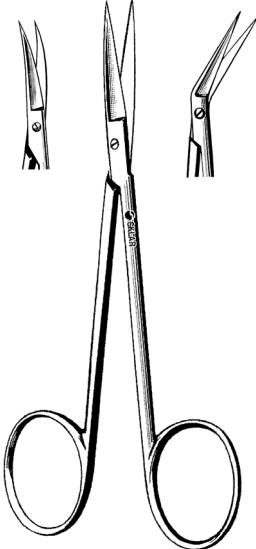

IRIS SCISSORS

STRAIGHT


47-1135	S/S	3½"
47-1140	B/B	3½"
47-1143	S/S	4"
47-1441	S/S	4", Heavy
47-1145	S/S	4½"
47-1144	B/B	4½"

CURVED

47-1235	S/S	3½"
47-1240	B/B	3½"
47-1243	S/S	4"
47-1442	S/S	4", Heavy
47-1245	S/S	4½"
47-1244	B/B	4½"

ANGULAR

64-2142	S/S	4½"
---------	-----	-----

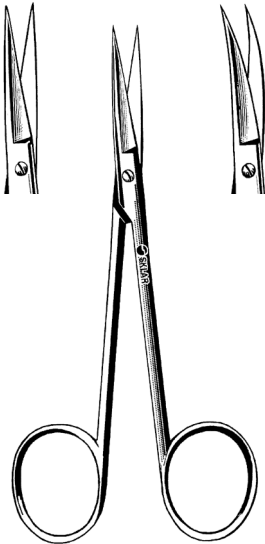

SKLARLITE™ IRIS SCISSORS

STRAIGHT


23-1148	S/S	3½"
23-1149	S/S	4½"

CURVED

23-1150	S/S	3½"
23-1151	S/S	4½"

SKLARLITE™
THE LIGHT TOUCH

SKLARLITE™ AND SKLARLITE XD™ SCISSORS DELIVER THE ULTIMATE IN PRECISION

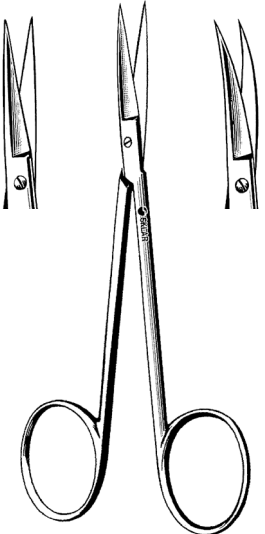


IRIS SCISSORS DELICATE

STRAIGHT

22-1135	S/S	3½"
47-1440	S/S	4"
47-1246	S/S	4½"

CURVED

22-1235	S/S	3½"
47-1248	S/S	4½"

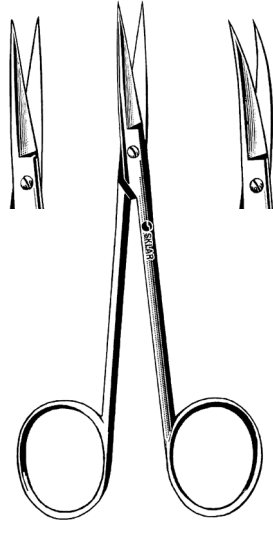

SKLARLITE XD™ IRIS SCISSORS


STRAIGHT

23-1152	S/S	3½"
23-1153	S/S	4½"

CURVED


23-1154	S/S	3½"
23-1155	S/S	4½"

SKLARLITE XD
EXTRA DELICATE

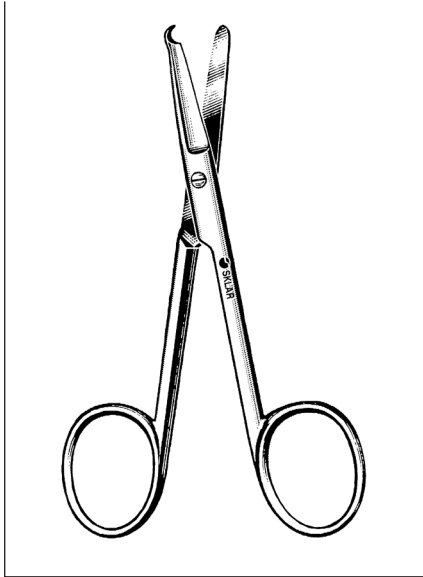


SPENCER STITCH SCISSORS

22-2835 3½"
 22-2845 4½"
 15-1355 5½" Littauer

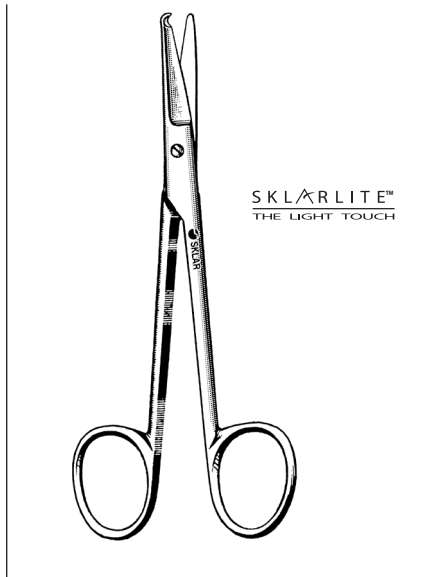


SKLARLITE™ SPENCER STITCH SCISSORS

23-1188 3½"
 23-1189 4½"
 23-1190 5½" Littauer

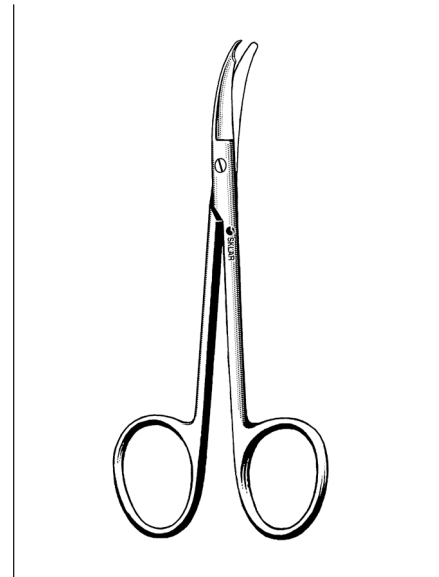


SUTURE SCISSORS

22-2935 3½" Shortbent
 22-2947 4¾" Northbent

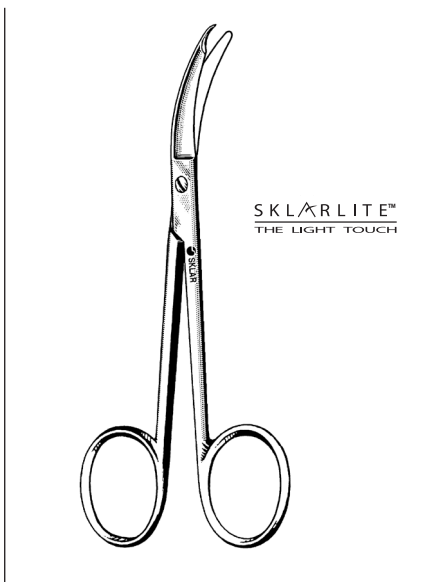


SKLARLITE™ SUTURE SCISSORS

23-1191 3½" Shortbent

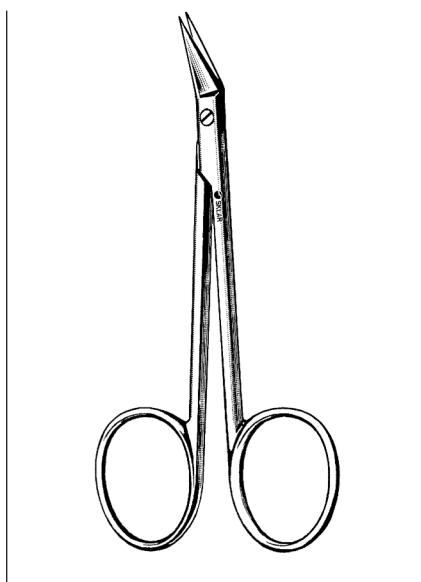

**WILMER SCISSORS
 ANGLED ON FLAT**

64-1735 4"

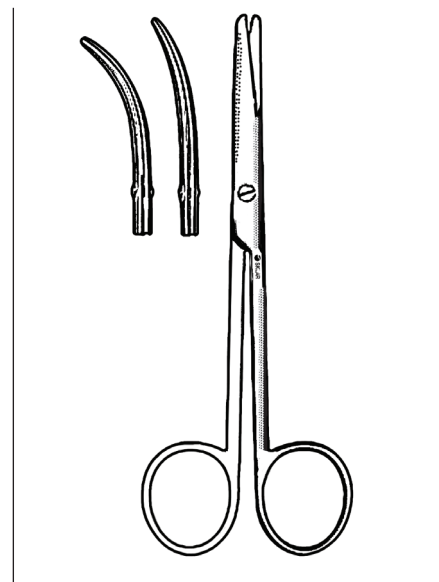

LANDOLT ENUCLEATION SCISSORS

SLIGHT CURVE
 64-1150 5"
MEDIUM CURVE
 64-1151 5"
FULL CURVE
 64-1152 5"




RIBBON STEVENS TENOTOMY SCISSORS

STRAIGHT	CURVED
64-1241 4"	64-1341 4"




RIBBON STITCH SCISSORS

STRAIGHT	CURVED
64-1345 4"	64-1445 4"

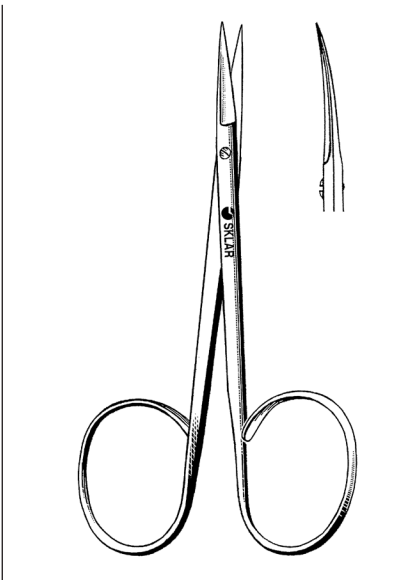


RIBBON IRIS SCISSORS

STRAIGHT	CURVED
47-1250 4"	47-1255 4"

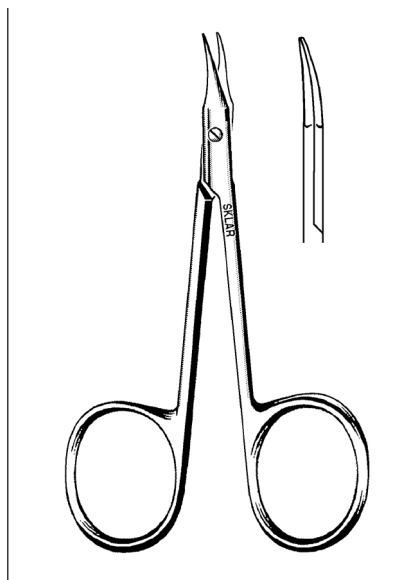

GRADLE SCISSORS

SLIGHTLY CURVED
SHARP
 64-3280 3¾"
 64-3284 4¾"

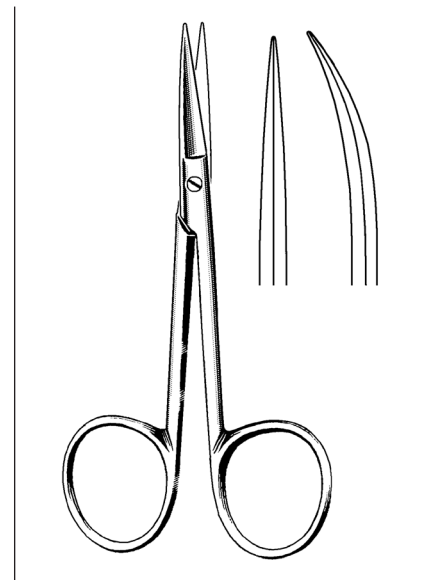

SEALY DISSECTING SCISSORS

TWO BLUNT TIPS, MEDIUM WEIGHT
DELICATE BLADES
 64-3285 4¼" Straight
 64-3288 4¼" Curved



STRABISMUS SCISSORS
STRAIGHT 64-1442 4½"
CURVED 64-1542 4½"



SKLARLITE™ STRABISMUS SCISSORS
STRAIGHT 23-1158 4½"
CURVED 23-1159 4½"



STRABISMUS SCISSORS
DELICATE
STRAIGHT 22-1442 4½"
CURVED 22-1542 4½"



SKLARLITE XD™ STRABISMUS SCISSORS
STRAIGHT 23-1162 4½"
CURVED 23-1163 4½"



OPERATING SCISSORS

STRAIGHT

SHARP/SHARP

13-1045	4½"
13-1050	5"
13-1055	5½"
13-1060	6"
13-1065	6½"
13-1072	7¼"

SHARP/BLUNT

14-1045	4½"
14-1050	5"
14-1055	5½"
14-1060	6"
14-1065	6½"
14-1072	7¼"

BLUNT/BLUNT

15-1045	4½"
15-1050	5"
15-1055	5½"
15-1060	6"
15-1065	6½"



SKLARLITE™
THE LIGHT TOUCH

SKLARLITE™ OPERATING SCISSORS

STRAIGHT

SHARP/SHARP

23-1100	4½"
23-1101	5"
23-1102	5½"

SHARP/BLUNT

23-1104	4½"
23-1105	5"
23-1106	5½"



OPERATING SCISSORS

DELICATE, STRAIGHT

SHARP/SHARP

22-1145	4½"
22-1150	5"
22-1155	5½"

SHARP/BLUNT

22-1245	4½"
22-1250	5"
22-1255	5½"

BLUNT/BLUNT

22-1345	4½"
22-1350	5"
22-1355	5½"



SKLARLITE XD™
EXTRA DELICATE

SKLARLITE XD™ OPERATING SCISSORS

STRAIGHT

SHARP/SHARP

23-1112	4½"
23-1113	5"
23-1114	5½"

SHARP/BLUNT

23-1116	4½"
23-1117	5"
23-1118	5½"



OPERATING SCISSORS

CURVED

SHARP/SHARP

13-2045	4½"
13-2050	5"
13-2055	5½"
13-2060	6"
13-2065	6½"
13-2072	7¼"

SHARP/BLUNT

14-2045	4½"
14-2050	5"
14-2055	5½"
14-2060	6"
14-2065	6½"
14-2072	7¼"

BLUNT/BLUNT

15-2045	4½"
15-2050	5"
15-2055	5½"
15-2060	6"
15-2065	6½"



SKLARLITE™
THE LIGHT TOUCH

SKLARLITE™ OPERATING SCISSORS

CURVED

SHARP/SHARP

23-1124	4½"
23-1126	5½"

SHARP/BLUNT

23-1128	4½"
23-1130	5½"



OPERATING SCISSORS

DELICATE, CURVED

SHARP/SHARP

22-1445	4½"
22-1450	5"
22-1455	5½"

SHARP/BLUNT

22-1545	4½"
22-1550	5"
22-1555	5½"

BLUNT/BLUNT

22-1645	4½"
22-1650	5"
22-1655	5½"



SKLARLITE XD™
EXTRA DELICATE

SKLARLITE XD™ OPERATING SCISSORS


CURVED

SHARP/SHARP

23-1136	4½"
23-1138	5½"


SHARP/BLUNT

23-1140	4½"
23-1142	5½"

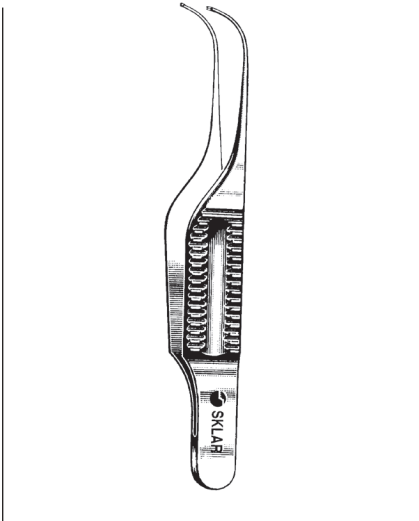


GILL IRIS FORCEPS

66-7130 Curved 3" Delicate
66-7230 Curved 3" Extra Delicate

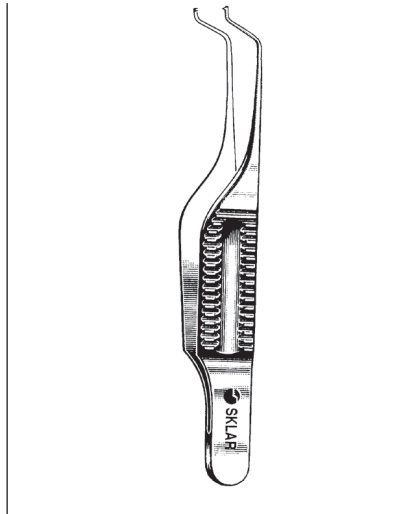


GILL-HESS IRIS FORCEPS

66-7238 2¾" with 1x2 Teeth Forward .5mm
66-7240 2¾" with 1x2 Teeth Down .5mm

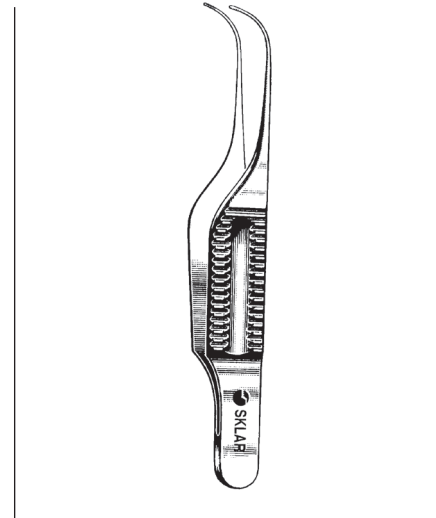


GILL (CHANDLER) IRIS FORCEPS

66-7244 3" Fine Cross Serrated Tips, .6mm wide
66-7246 3" 1x2 Teeth, 0.6mm Wide

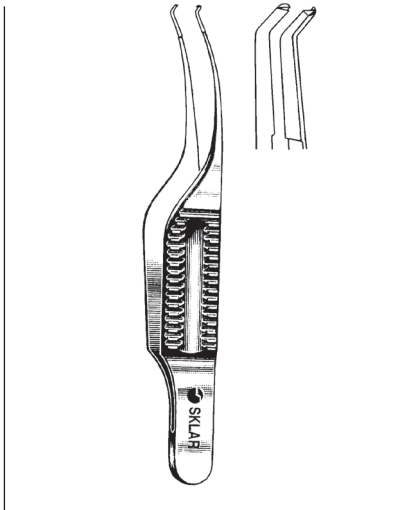


TROUTMAN-BARRAQUER CORNEAL UTILITY FORCEPS COLIBRI TYPE

66-7250 1x2 .5mm Teeth, set at 45° with Tying Platform

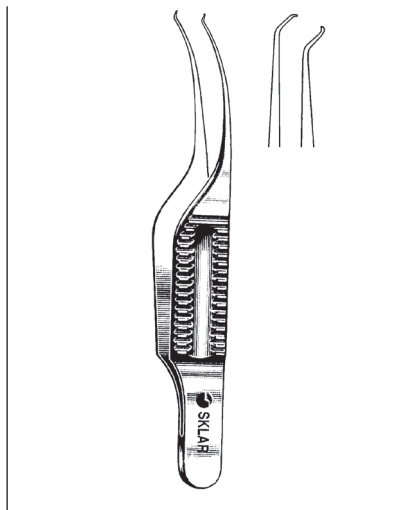


COLIBRI CORNEAL UTILITY FORCEPS

3"
66-7259 1x2 Teeth .12mm
66-7261 1x2 Teeth .4mm

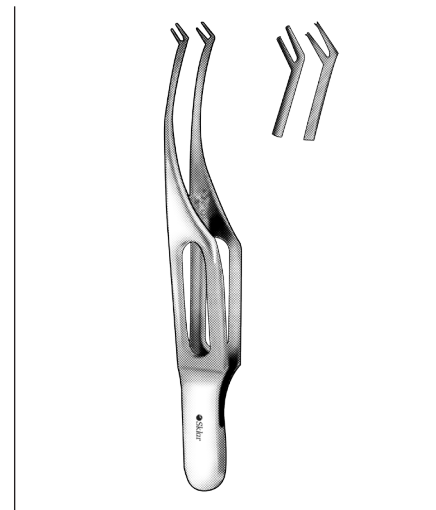


POLACK CORNEAL FIXATION FORCEPS


66-4140 .4mm

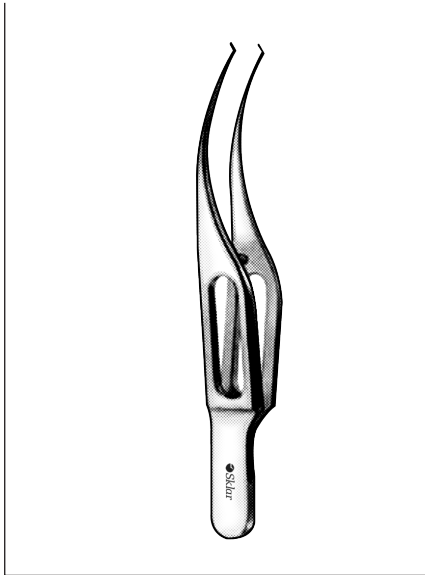


GILLS-COLIBRI FORCEPS
WITH TYING PLATFORM
 66-6128 2¾" Sharp, Curved

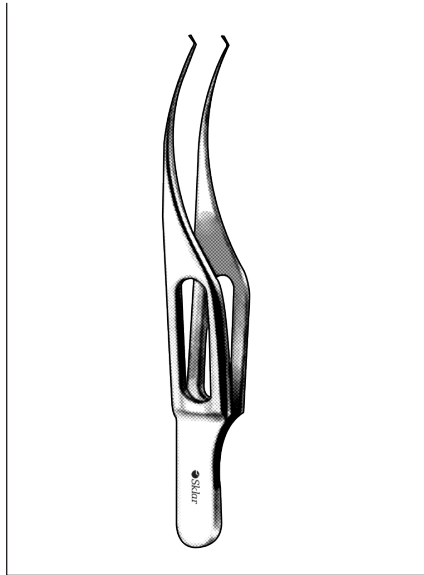


HARMS-COLIBRI FORCEPS
1X2 TEETH WITH TYING PLATFORM
 66-6144 .12mm

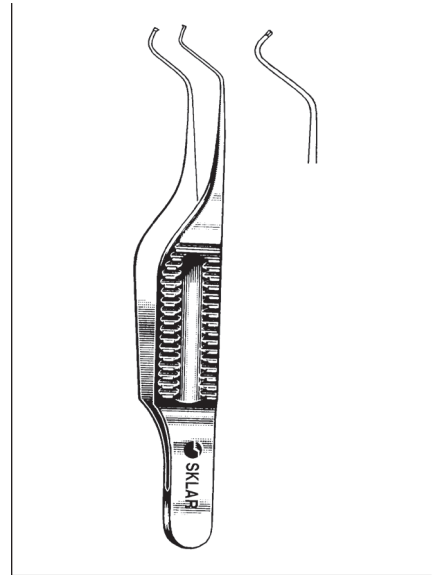


BOTVIN IRIS FORCEPS
 66-7258 3" Fine Cross Serrated Tips,
 .5mm Wide

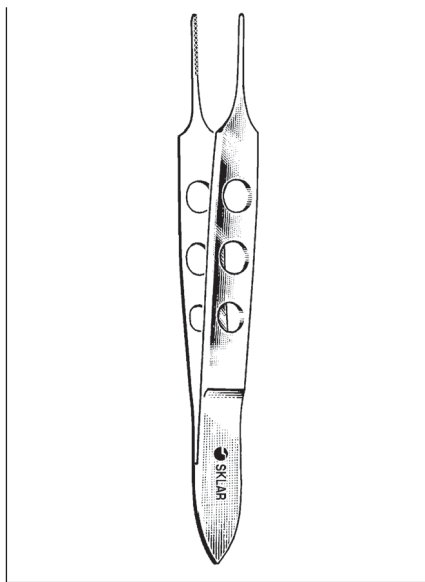


BISHOP-HARMON IRIS FORCEPS
SERRATED, STRAIGHT TIPS
 66-4032 3½"

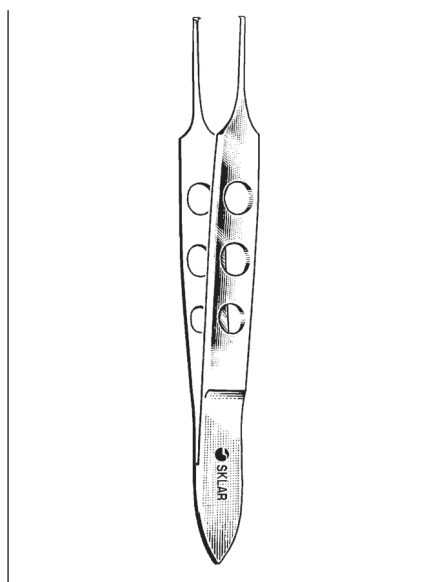

BISHOP-HARMON IRIS FORCEPS
1X2 STRAIGHT TEETH, DELICATE TIP
 66-4232 3½"

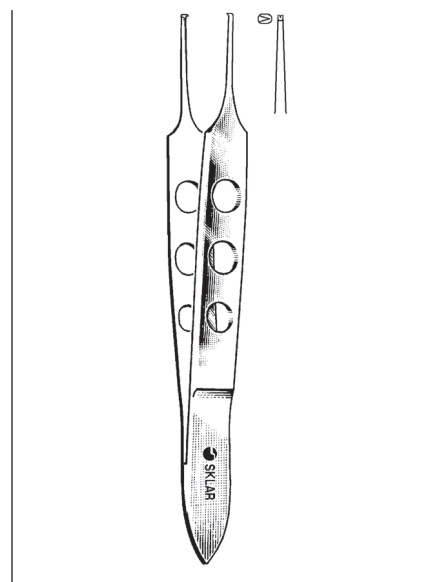


BISHOP-HARMON IRIS FORCEPS
3½" STRAIGHT
 66-4132 1x2 Teeth Standard Pattern
 66-4133 Extra Delicate for Microsurgery

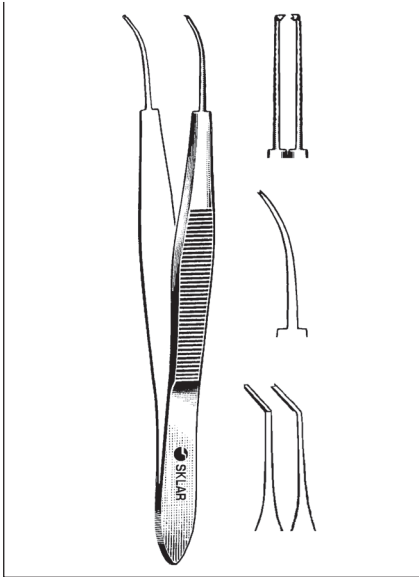

STEVENS IRIS FORCEPS

WITHOUT CATCH, 4"


- | | | |
|---------|-----------|----------|
| 66-4947 | Serrated | Straight |
| 66-4948 | Serrated | Curved |
| 66-4949 | Serrated | Angled |
| 66-4950 | 1x2 Teeth | Straight |
| 66-4951 | 1x2 Teeth | Curved |
| 66-4955 | 1x2 Teeth | Angled |

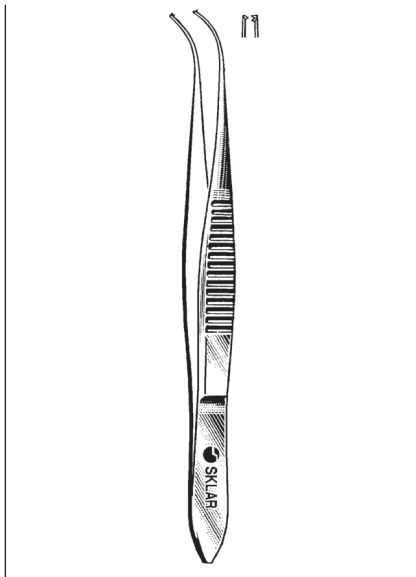

FORESTER IRIS FORCEPS


3 3/4" CURVED TIPS, 1X2 TEETH ON OUTSIDE OF CURVE

- | | |
|---------|-----------------|
| 66-4530 | Standard Handle |
|---------|-----------------|

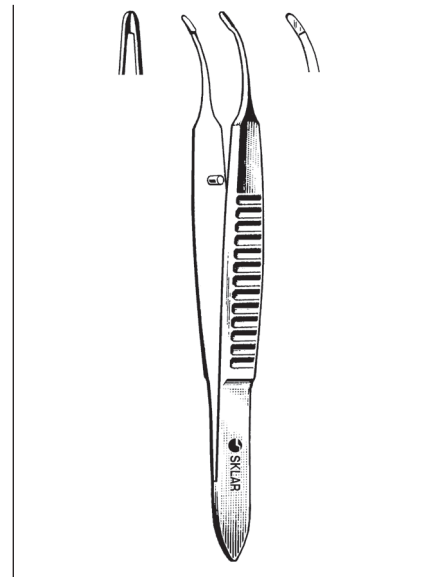

CHANDLER IRIS FORCEPS


66-3927 2 3/4" Curved

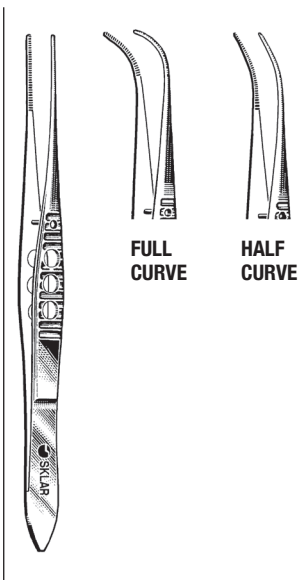

SKLAR LITEGRIP™ IRIS FORCEPS

SERRATED


- | | | |
|---------|----|------------|
| 66-2742 | 4" | Straight |
| 66-2942 | 4" | 1/2 Curved |
| 66-3142 | 4" | Full Curve |

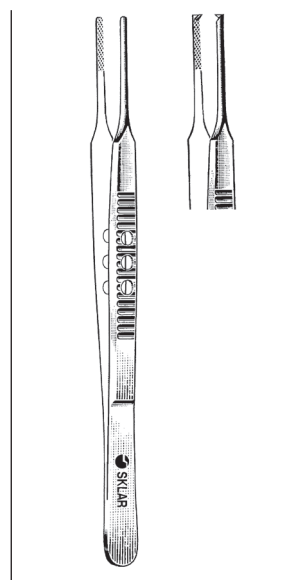

SKLAR LITEGRIP™ IRIS FORCEPS


1X2 TEETH

- | | | |
|---------|----|------------|
| 66-3342 | 4" | Straight |
| 66-3542 | 4" | 1/2 Curved |
| 66-3742 | 4" | Full Curve |

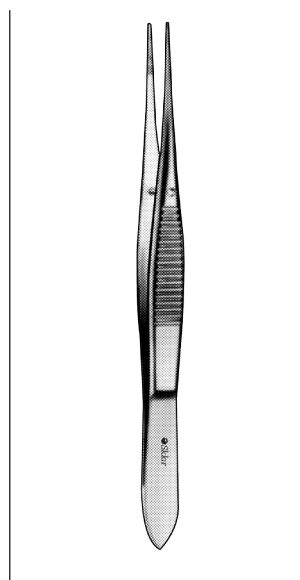


OSHER IRIS FORCEPS

66-7430 Angled

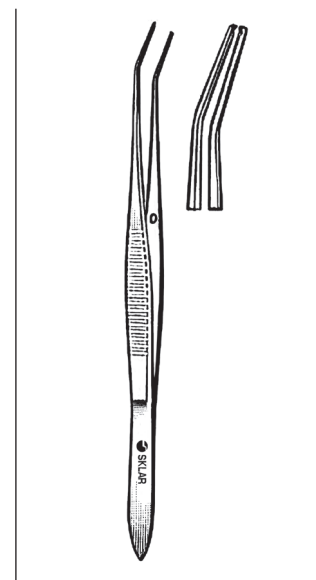


O'CONNOR IRIS FORCEPS

66-4956 3 3/4" Angular Jaws .5x9mm with 1x2 Teeth

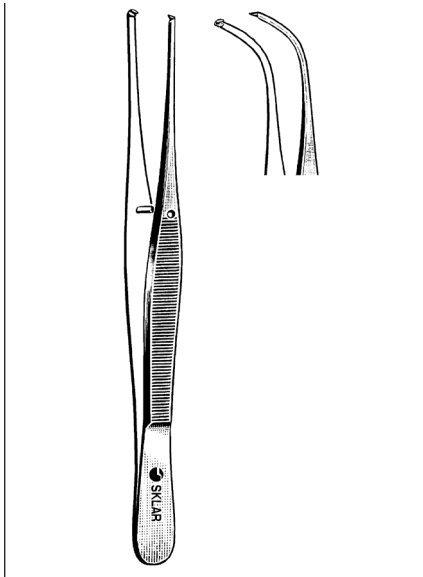


STRABISMUS FORCEPS

66-4840 4" 1x2 Teeth

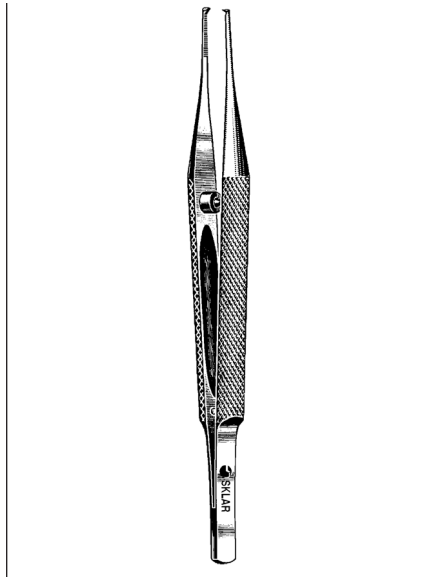


KIRBY IRIS FORCEPS

66-2750 4" Straight 1x2 Teeth
 66-2755 4" Curved 1x2 Teeth
 66-2760 4" Angled 1x2 Teeth

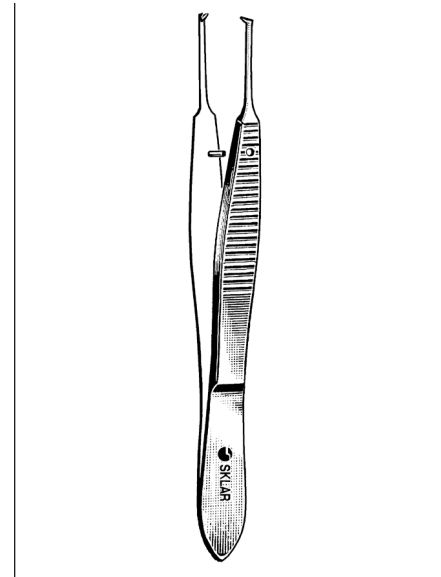


GRAEFE FORCEPS

66-4525 2¾" Straight 1x2 Teeth
 66-4625 2¾" Curved 1x2 Teeth
 66-4526 2¾" Straight Serrated
 66-4626 2¾" Curved Serrated

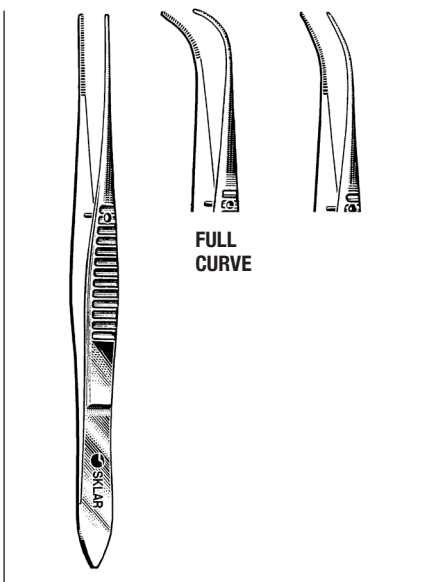


EYE DRESSING FORCEPS

SERRATED
 66-2740 4" Straight
 66-2940 4" ½ Curve
 66-3140 4" Full Curve
NON-MAGNETIC
 66-2840 4" Straight

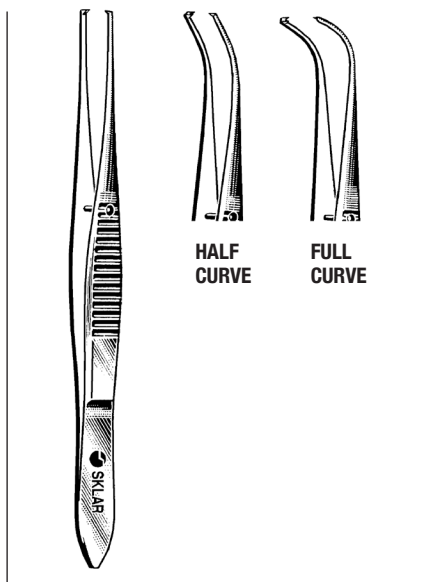


EYE TISSUE FORCEPS

1X2 TEETH
 66-3340 4" Straight
 66-3540 4" ½ Curve
 66-3740 4" Full Curve
NON-MAGNETIC
 66-3240 4" Straight

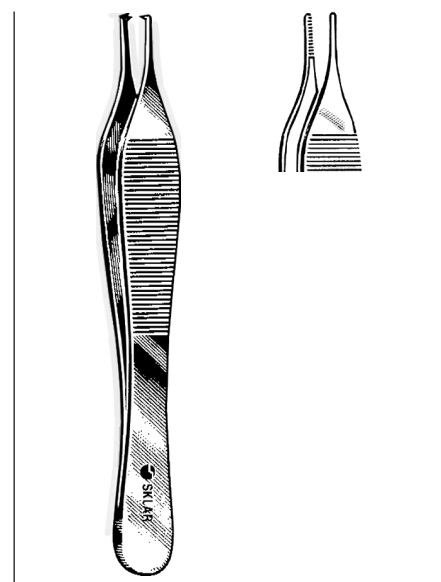


ADSON TISSUE FORCEPS


1X2 TEETH **SERRATED**
 50-3047 4¾" 50-3147 4¾"

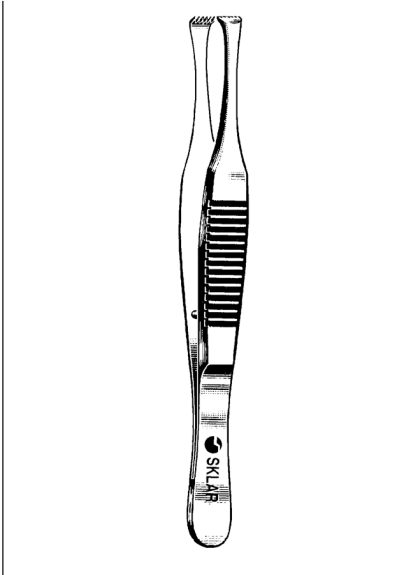


HARMON FORCEPS
66-1930 3"

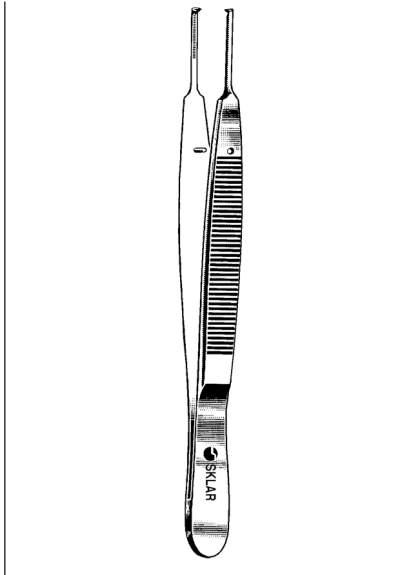


LESTER FIXATION FORCEPS
DELICATE
66-4237 3½" 1x2 Teeth
66-4238 3½" 2x3 Teeth

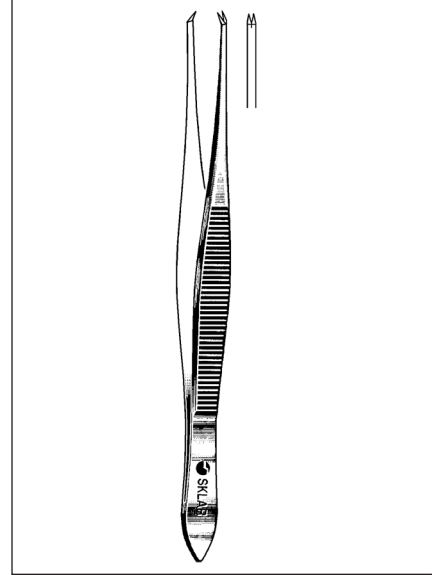


ELSCHMIG FIXATION FORCEPS
4¼" ANGLED UPWARDS
66-2350 Strong 1x2 Teeth
66-2351 Strong 2x3 Teeth
66-2352 Strong 3x4 Teeth

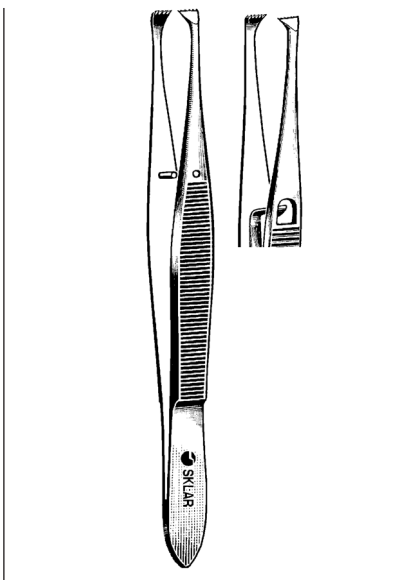


GRAEFE FIXATION FORCEPS
66-1940 4" Without Lock
66-2140 4" With Lock

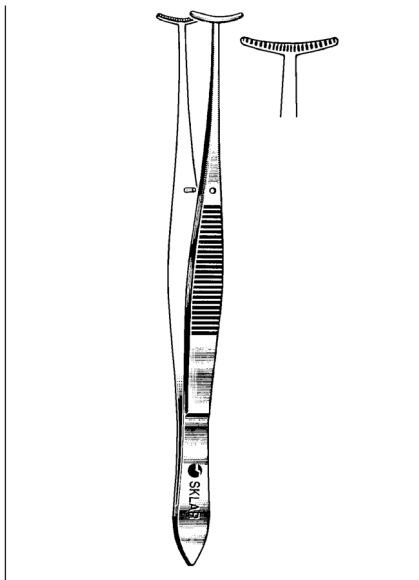


GREEN FIXATION FORCEPS
66-2005 4" 5mm Jaws
66-2009 4" 9mm Jaws

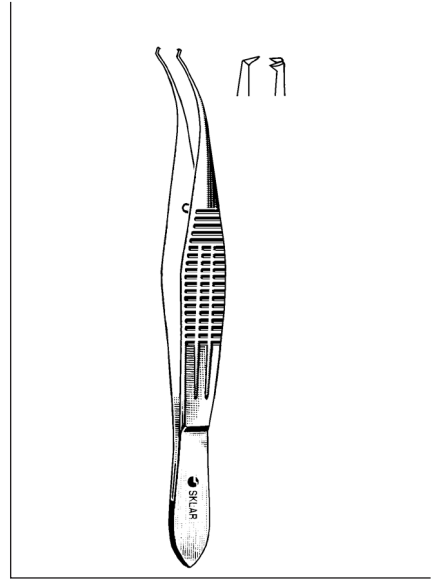


CASTROVIEJO COLIBRI FORCEPS
WIDE HANDLE 4" EXTRA DELICATE
66-3450 1x2 .12mm Teeth

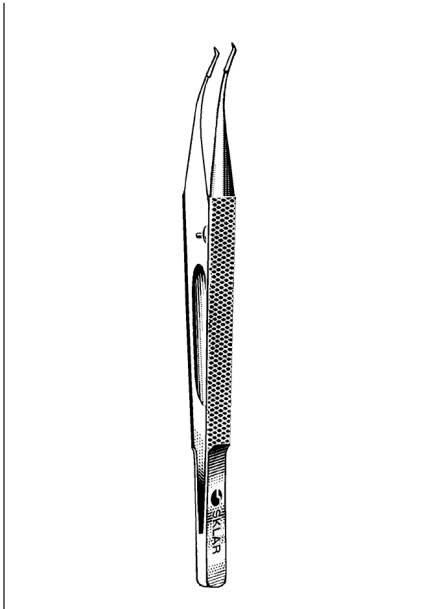


GIRARD CORNEAL UTILITY FORCEPS
 4.0" WITH TYING PLATFORM, SMOOTH TIP,
 0.5MM WIDE
 66-6120

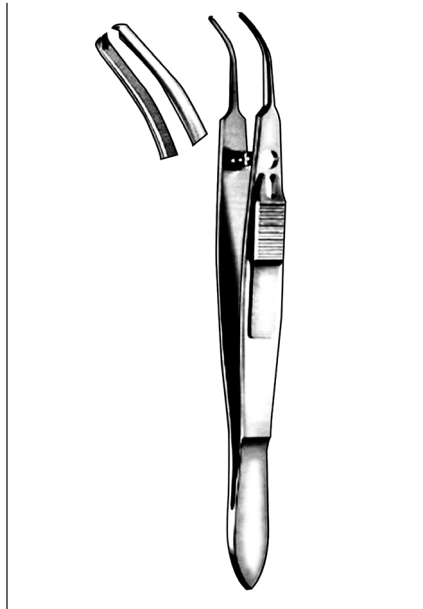


MOODY FIXATION FORCEPS
 1X2 TEETH
 66-4135 .5mm Left
 66-4136 .5mm Right

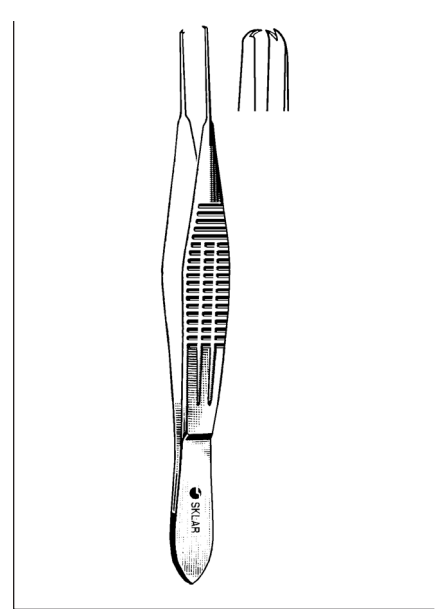


CASTROVIEJO FIXATION FORCEPS
 1X2 TEETH, 4 1/4"
 66-2640 .3mm Teeth with Tying Platform
 66-2635 .5mm Teeth without Tying Platform

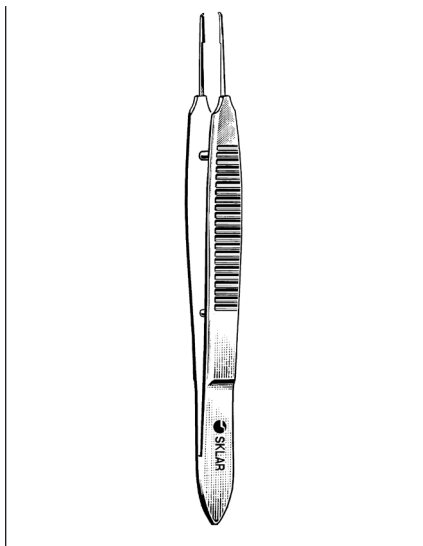

CASTROVIEJO FORCEPS
 66-6119 4" .12mm Teeth
 66-6103 4" .3mm Teeth
 66-6105 4" .5mm Teeth
 66-6109 4" .9mm Teeth

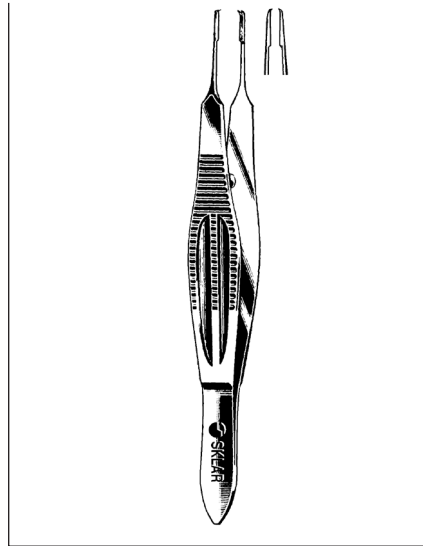

CASTROVIEJO SUTURE FORCEPS
 4" 1X2 PLATFORM BEHIND
 TEETH LIMITS DEPTH OF PENETRATION WIDE HANDLE
 66-6116 .12mm Delicate Teeth
 66-6117 .3mm Delicate Teeth
 66-6115 .5mm Delicate Teeth
 66-6118 1.5mm Delicate Teeth

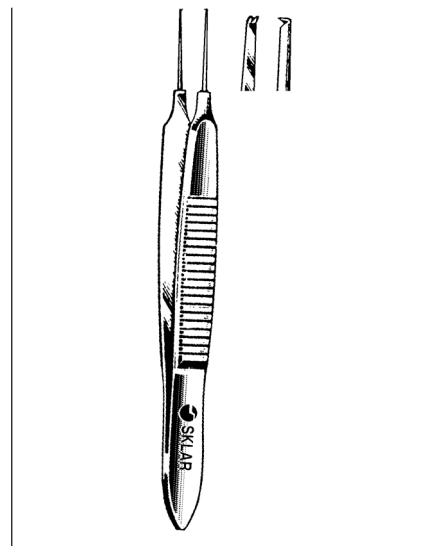


BONN MICRO SUTURING FORCEPS
 1X2 TEETH
 66-6125 2 3/4" .12mm, with Tying Platform
 66-6126 3 3/4" .12mm with Tying Platform




MCPHERSON TYING FORCEPS
TITANIUM
 66-6151 5mm Smooth Jaw




HELVESTON TYING FORCEPS
 66-6145 Smooth

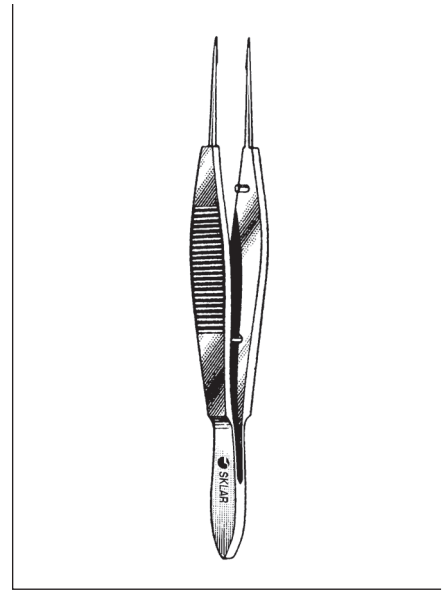


BONN TYING FORCEPS
WITH TYING PLATFORM
.3MM TIP WIDTH
 66-6160 Straight 4"

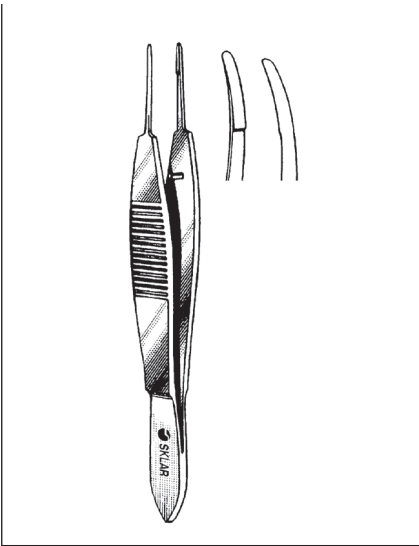


HARMS TYING FORCEPS
WITH TYING PLATFORM .5MM TIP WIDTH 4"
 66-6140 Straight
 66-6142 Curved

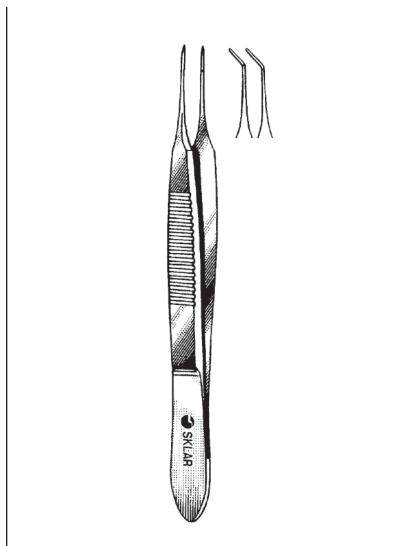


MCPHERSON TYING FORCEPS
WITH TYING PLATFORM 4MM LONG
.3MM TIP WIDTH 3.38"
 66-6150 3/4" Straight
 66-6152 4" Angled

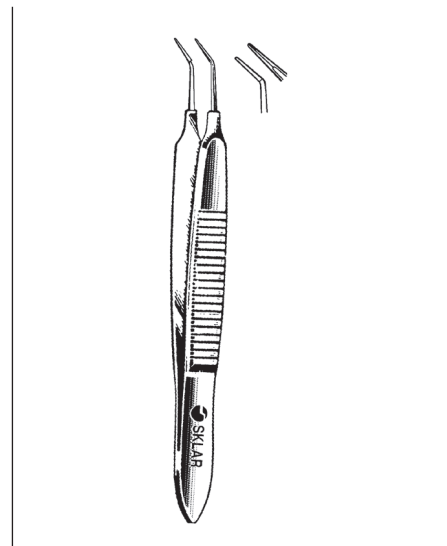


MCPHERSON MICRO IRIS
SUTURING FORCEPS
SMOOTH PLATFORM .3X5MM
 66-6164 3/2" Straight
 66-6166 3/2" Angled

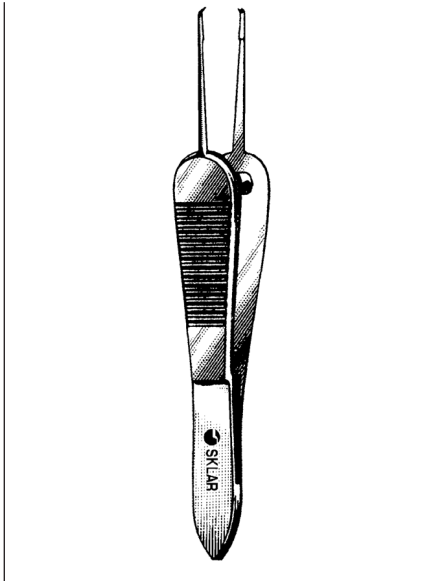


**MANHATTAN EYE & EAR
SUTURING FORCEPS**
3½" WITH SMOOTH PLATFORM
1X2 CURVED TEETH OVERLAPPING EACH OTHER
.7MM WIDE
66-6168

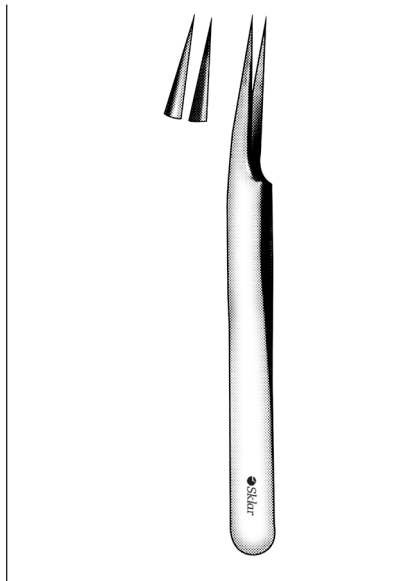


SUTURE REMOVAL FORCEPS
66-7629 #5A

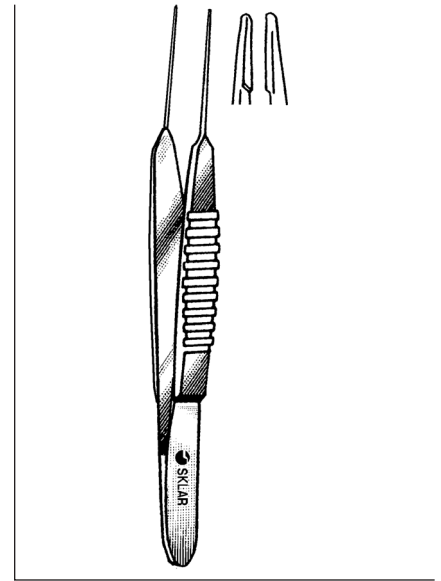


**KELMAN-MCPHERSON TYING
FORCEPS**
STRAIGHT 86MM
66-6175 Smooth

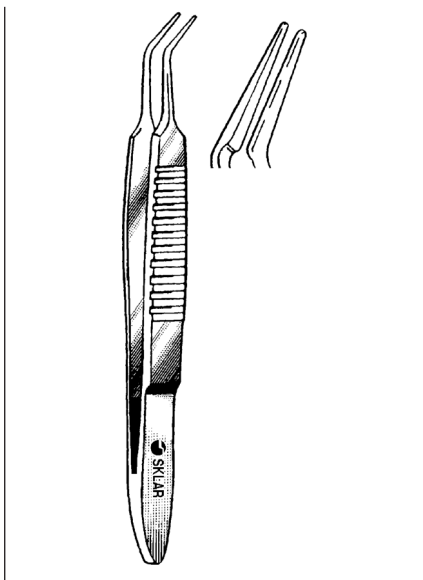


**BECHERT-MCPHERSON TYING
FORCEPS**
10MM TYING PLATFORM, 83MM
66-6178 Smooth Jaws, Angled

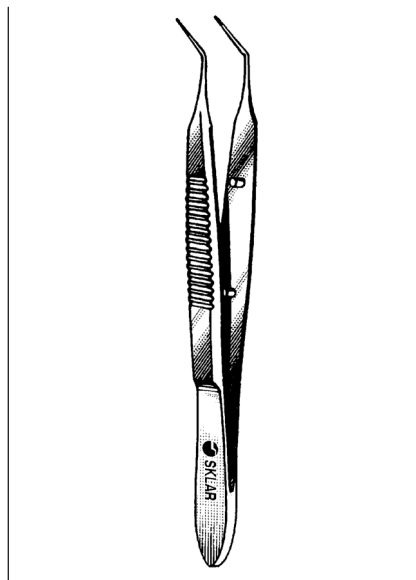


**KELMAN-MCPHERSON TYING
FORCEPS**
WITH TYING PLATFORM 7.5MM LONG
0.3MM TIP WIDTH ANGLED 3.3"
66-6180

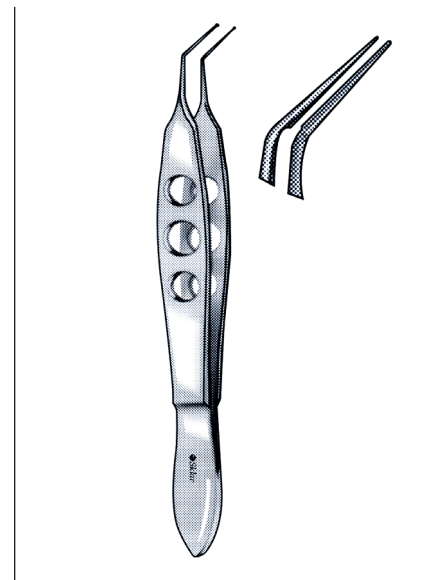


**TITANIUM
KELMAN-MCPHERSON TYING
FORCEPS**
WITH TYING PLATFORM 7.5MM SMOOTH JAW
66-7681

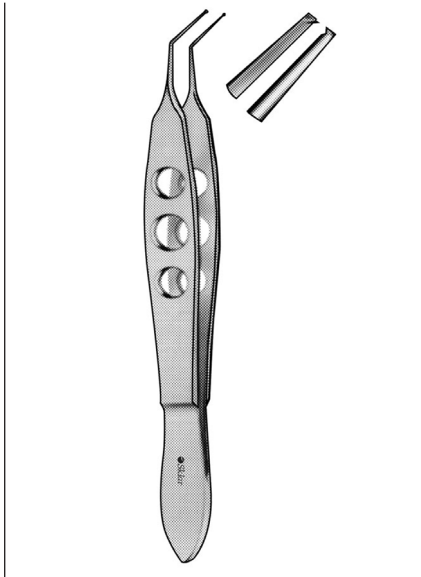

**KELMAN-MCPHERSON
FORCEPS**
WITH TYING PLATFORM
66-7680 1x2 Teeth 8mm Jaw

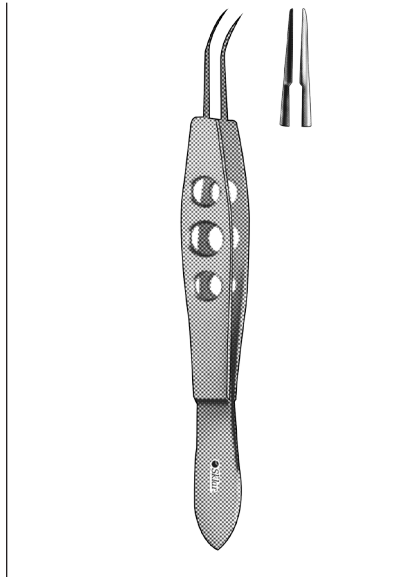


JAFFE TYING FORCEPS
66-6165 Curved
66-6167 Straight

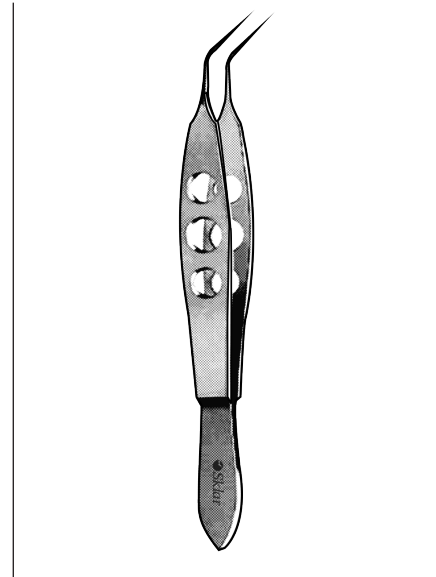


GASKIN FRAGMENT FORCEPS
66-4018 11mm Angled



**FECHTNER CONJUNCTIVA
FORCEPS**
WITH TYING PLATFORM
66-4020 Extra Delicate

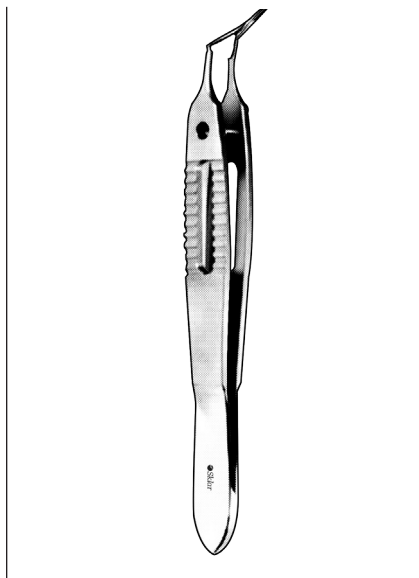

**MCDONALD LENS LOADING
FORCEPS**
66-6350 Curved

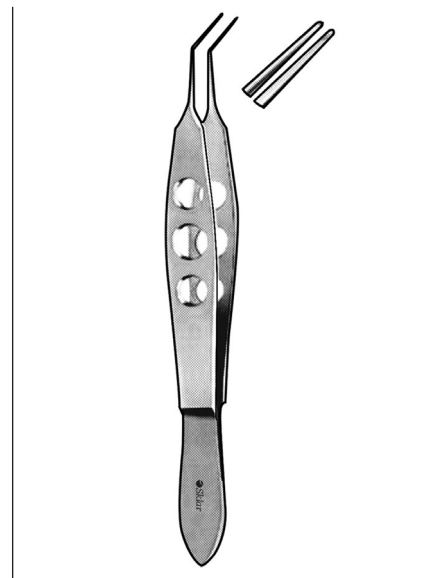


**HABERLE-MCPHERSON LENS
HOLDING FORCEPS**
WITH DIAGONAL GROOVE
66-7699 7.5mm Smooth Long Jaws



FAULKNER SOFT INTRAOCULAR LENS HOLDING FORCEPS

66-4025 Round Smooth Jaws

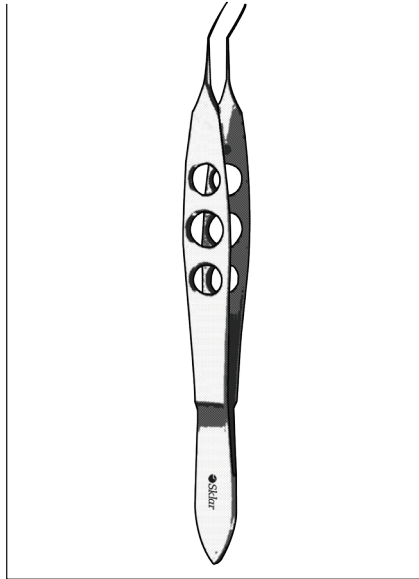


BURATTO SOFT INTRAOCULAR LENS FORCEPS

66-4028 Convex Jaws



STEINERT SOFT INTRAOCULAR LENS FORCEPS

66-4239 Conically Shaped Jaws

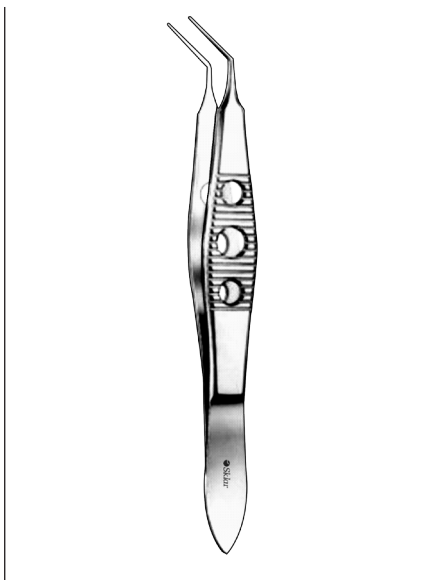


LIVERNIOS SOFT INTRAOCULAR LENS FOLDING FORCEPS

66-4240 Angled
66-4241 Straight
66-4242 Concave




SPALECK LENS FOLDER FORCEPS

66-4245 4"

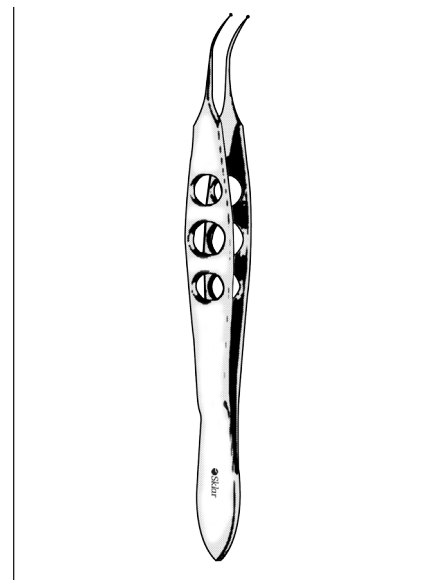


BURATTO LASIK FLAP FORCEPS


66-4030

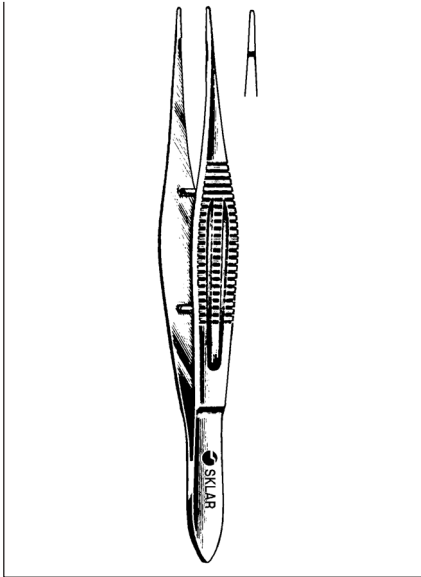


WALTON SUTURE FORCEPS
WITH TYING PLATFORM
66-6182 4¼" with Handle

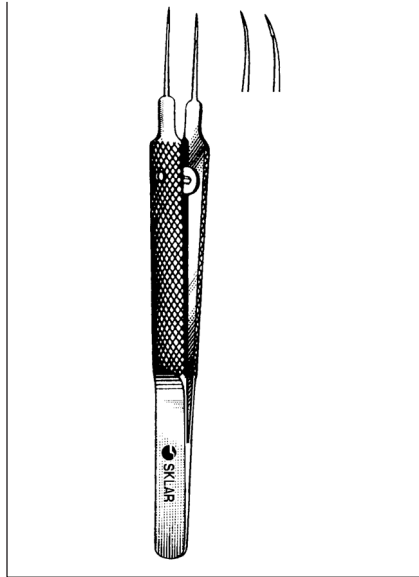


TENNANT TYING FORCEPS
EXTRA DELICATE ROUND HANDLE TYING
PLATFORM 6MM .3MM TIP WIDTH
66-6194 4" Straight
66-6196 4" Curved

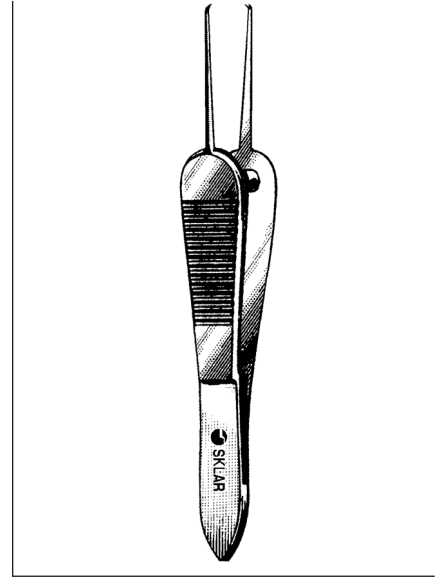


**SAUER SUTURING
FORCEPS**
1X2 CURVED TEETH OVERLAPPING EACH OTHER
.6MM WIDE
66-6330 3½"

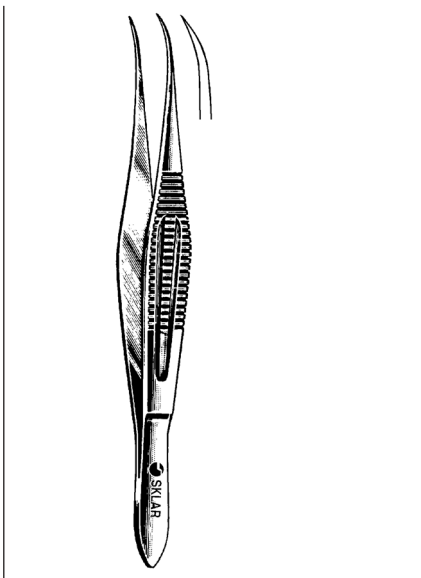


**MANHATTAN EYE AND EAR
SUTURE FORCEPS**
66-6250 4¼" with Fine Tips

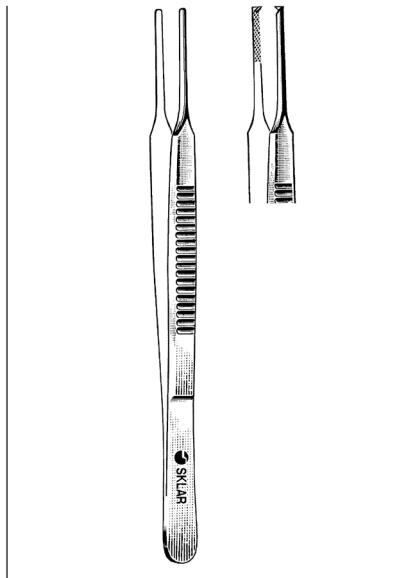

**MCCULLOUGH UTILITY
FORCEPS**
66-6110 4" Serrated
66-6112 4" Smooth
66-6114 4" 1x2 Teeth with Serrations

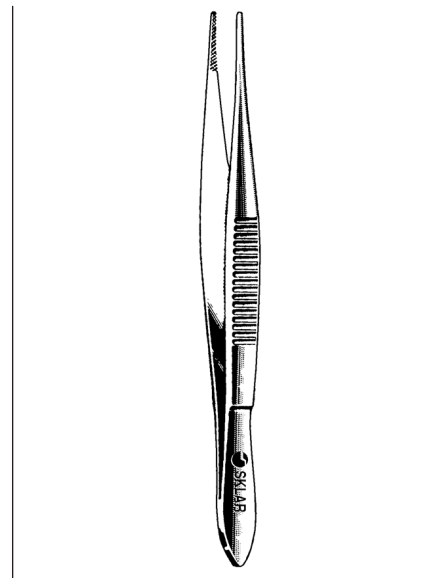


**WILLS HOSPITAL
UTILITY FORCEPS**
66-6268 4" Cross Serrations on Tips

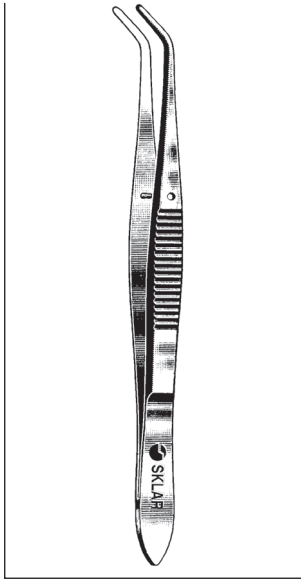


NUGENT UTILITY FORCEPS

- 66-6240 4¼" Angular Smooth Tips
- 66-6242 4¼" Angular Serrated Tips

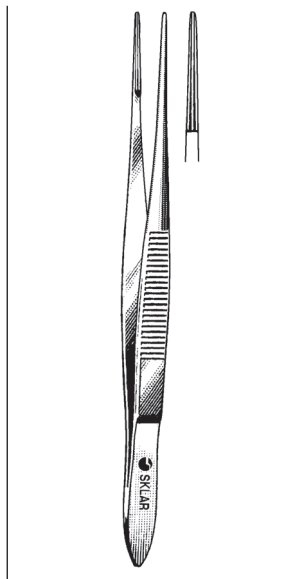


BONACCOLTO UTILITY FORCEPS

- CROSS SERRATED AT TIPS FOR BETTER PICK UP DELICATE PATTERN**
- 66-6340 1.2mm Jaws
- 66-6338 1.7mm Jaws

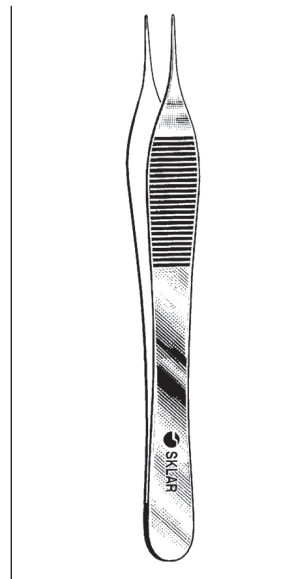


THORPE CORNEOSCLERAL FORCEPS

- 66-7347 4¾"

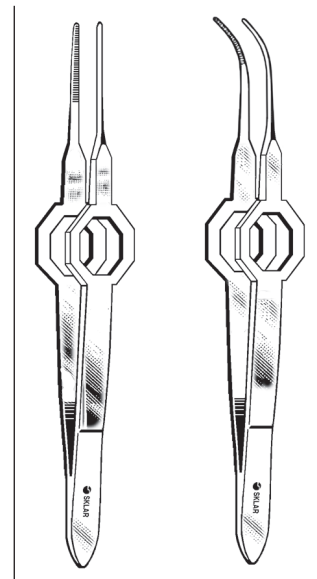

FOERSTER EYE DRESSING FORCEPS


- OCTAGONAL GRIP, .5MM**
- 66-6036 3¾" Straight, Serrated
- 66-6038 3¾" Straight, 1x2 Teeth
- 66-6037 3¾" Full Curve, Serrated
- 66-6039 3¾" Full Curve, 1x2 Teeth

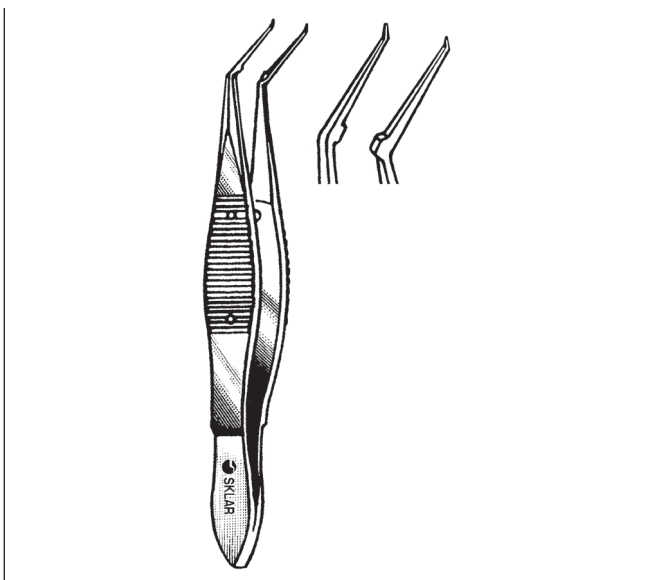

KRAFF-UTRATA CAPSULORHEXIS FORCEPS


- WITH IRIS STOPS**
- 12MM LONG SMOOTH JAWS**
- .3MM TIP 4"**

- 66-5351 Curved Tips, Flat Handle
- 66-5352 Angled Jaws, Round Handle

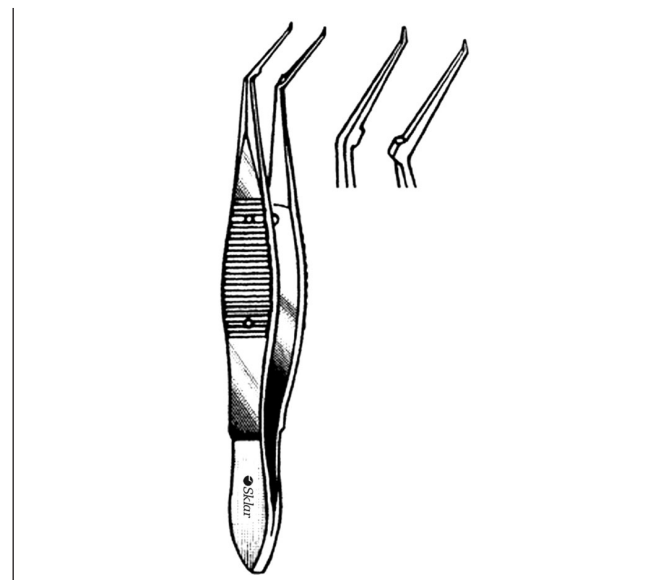

KRAFF CAPSULORHEXIS FORCEPS

- 66-5354 4" 11mm Long Jaws

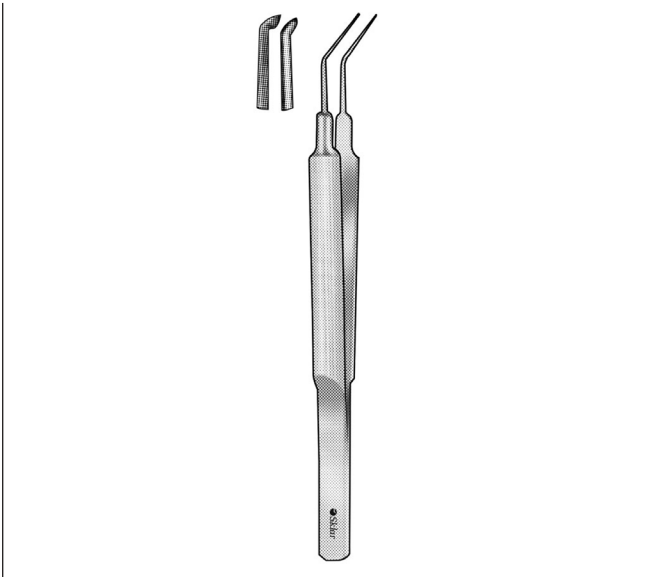


ULTRATA CAPSULORHEXIS FORCEPS

66-7630 11mm Think Shanks Round



ULTRATA CAPSULORHEXIS FORCEPS

66-7631 Short Handle

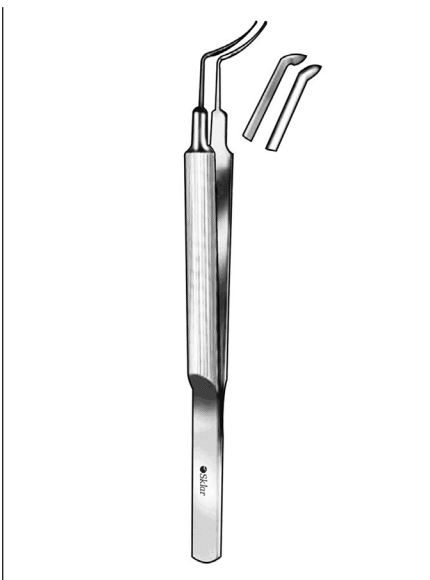

MASKET CAPSULORHEXIS FORCEPS

EXTRA DELICATE


66-7670 11mm Vaulted Shank with Guide Pin



MASKET CAPSULORHEXIS FORCEPS


EXTRA DELICATE

66-7671 11mm Vaulted Shank

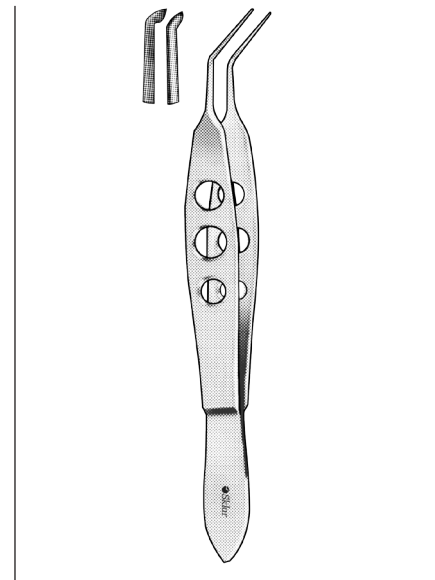


CAPSULORHEXIS FORCEPS


66-4145 11mm Long Shanks Delicate

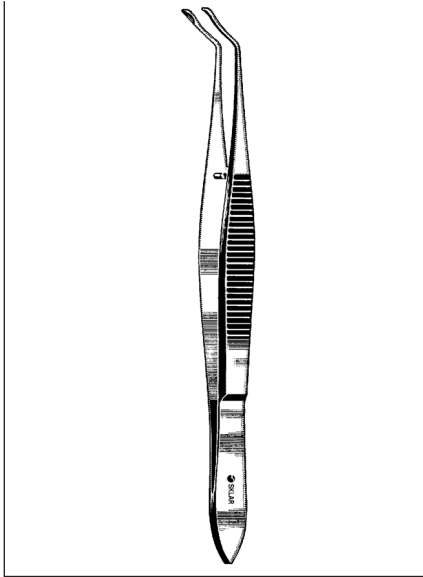


KALT CAPSULE FORCEPS
66-5240 4"

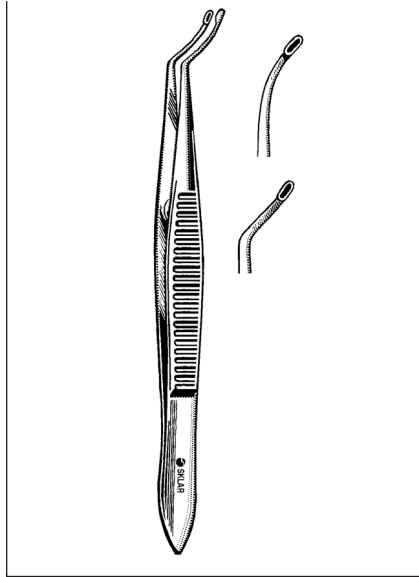


ARRUGA CAPSULE FORCEPS
66-5340 4" Standard Curve
66-5342 4" New Curve

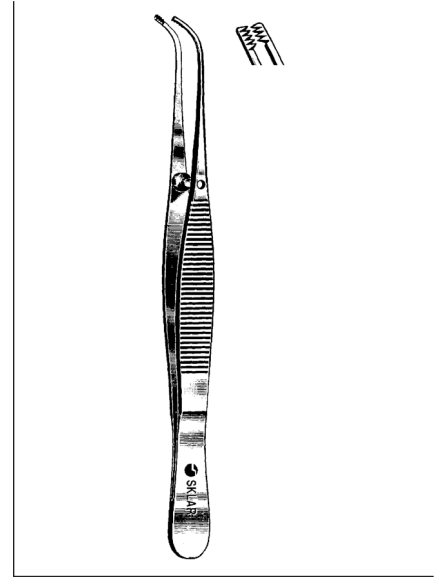


SCHWEIGER EXTRA CAPSULAR FORCEPS
66-5355 3 3/4" 4x5 Teeth

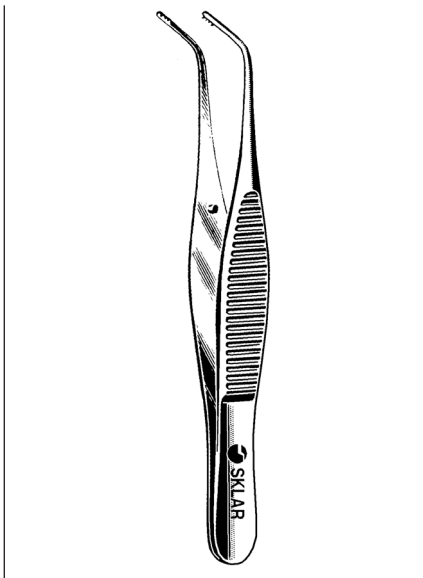


FUCHS EXTRA CAPSULAR FORCEPS
66-5530 2 3/4" 5x5 Teeth

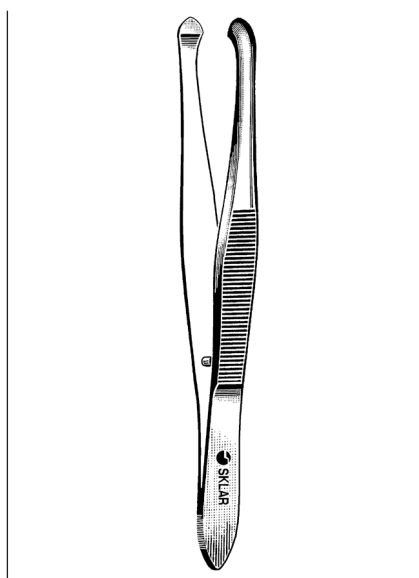


ZIEGLER CILIA FORCEPS
66-1535 3 1/2"

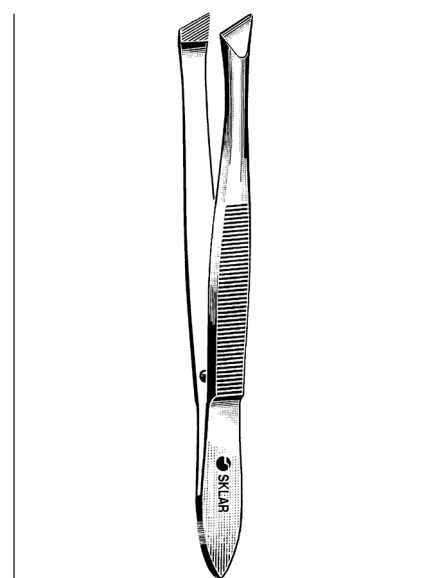


BERGH CILIA FORCEPS
66-1635 3 1/2"

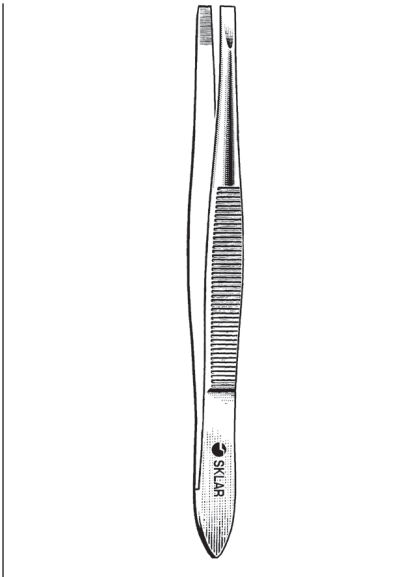


DOUGLASS CILIA FORCEPS
66-1830 3½"

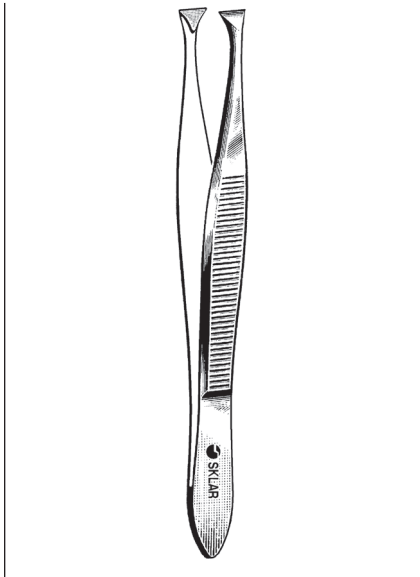


LITTAUER CILIA FORCEPS
66-1735 3½"

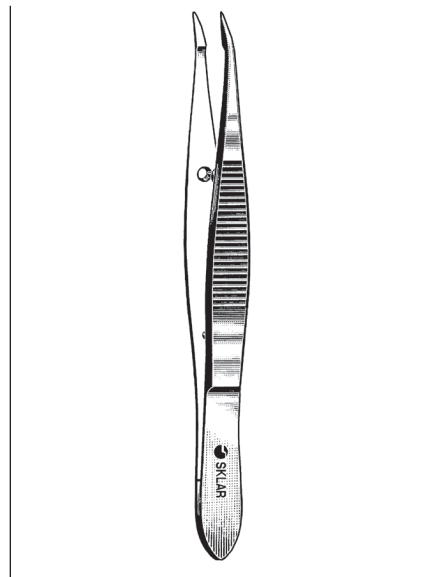


BARRAQUER CILIA FORCEPS
66-1742 4¼"

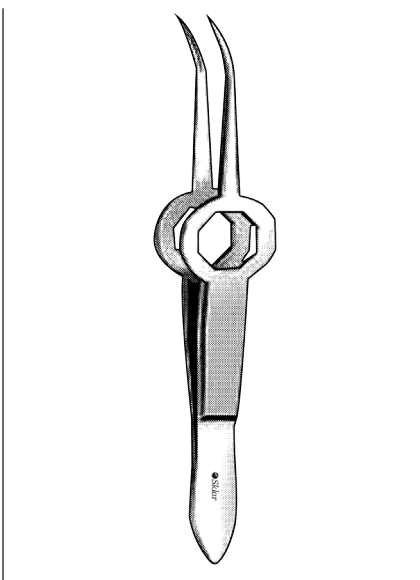


SHAFF CILIA FORCEPS
66-6034 4¼"

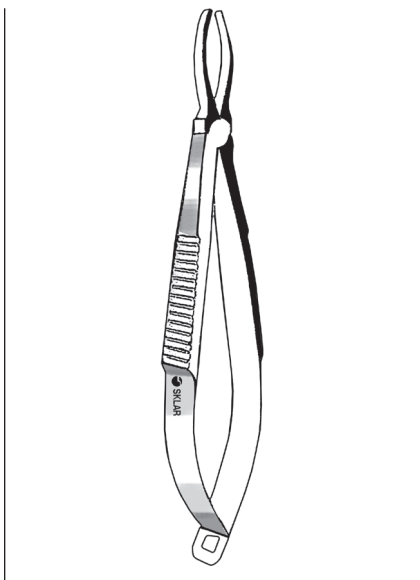

GRADLE CILIA FORCEPS
66-1745 4" Precision Ground Jaws
1.4mm Wide

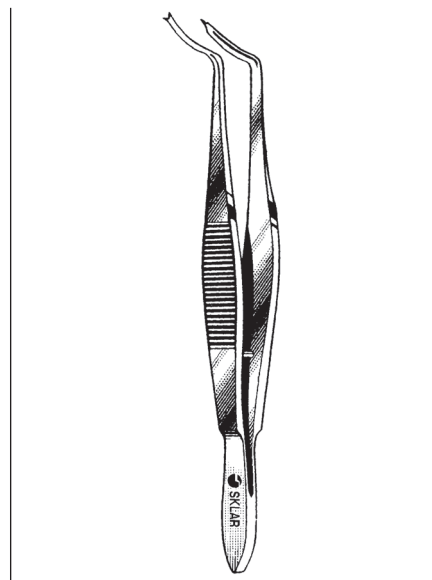


TENNANT-TROUTMAN SUPERIOR RECTUS FORCEPS
66-5950 4" Curved Shanks,
1x2 Teeth

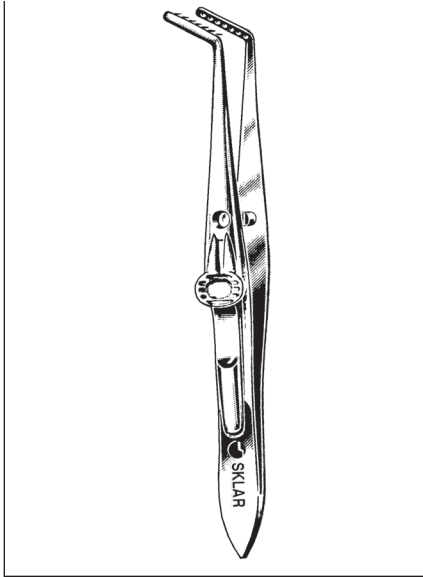


JAMESON RECESSON FORCEPS

66-5960 3¾" Left
66-5962 3¾" Right

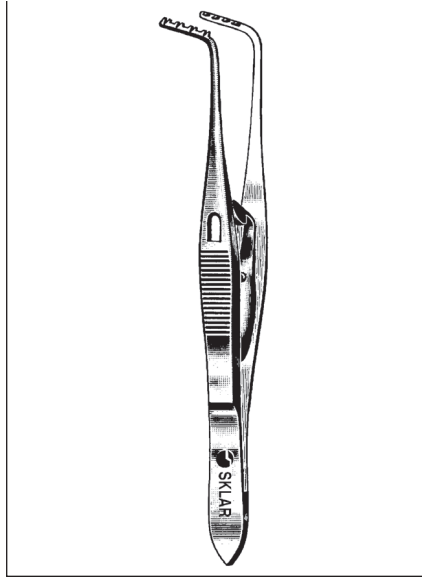


PRINCE MUSCLE FORCEPS

66-5942 4" Left

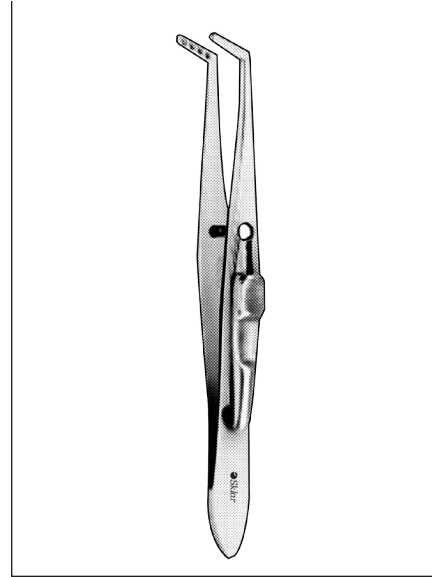


JAMESON MUSCLE FORCEPS SMALL

66-5956 Left, with 4 Teeth
66-5958 Right, with 4 Teeth

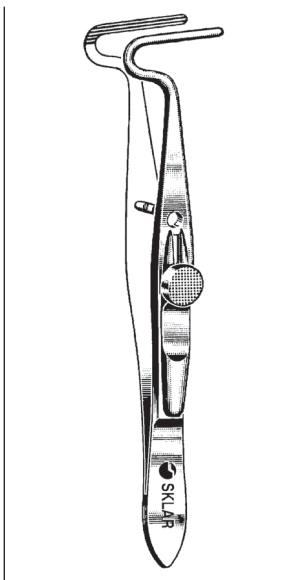


BERKE PTOSIS FORCEPS

66-6342 4¼"

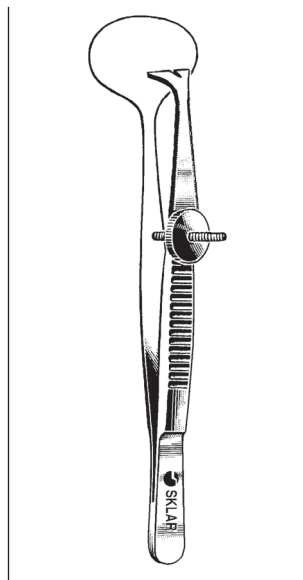


ERHARDT LID FORCEPS SERRATED AND FENESTRATED

66-1030 3½" 15x24mm Plate

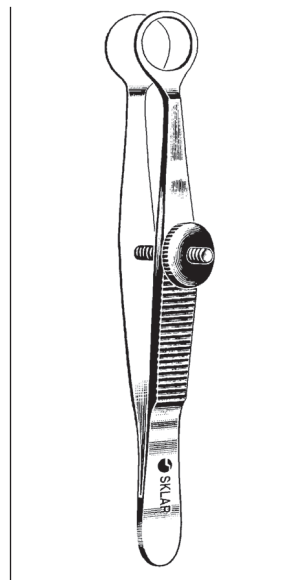


LAMBERT CHALAZION FORCEPS FENESTRATED SMOOTH JAWS

66-1235 3½" 8mm
66-1236 3½" 11mm

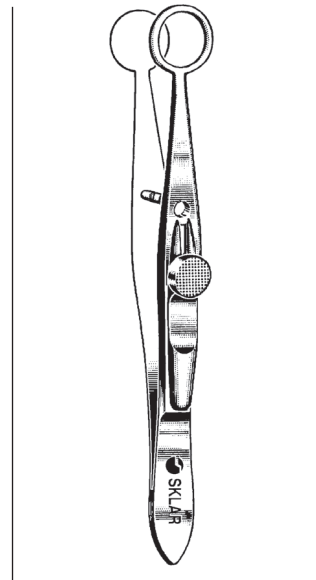


SPENCER CHALAZION FORCEPS WITH LOCK FENESTRATED SMOOTH JAWS

66-1240 3¾" 9.5mm Jaw

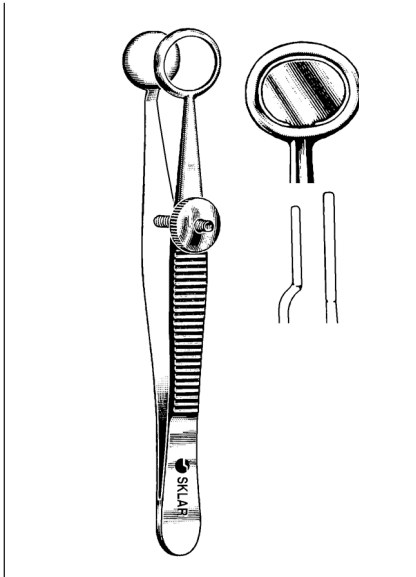


BAIRD CHALAZION FORCEPS

3¾" LONG
66-1250 8x12mm Upper Plate

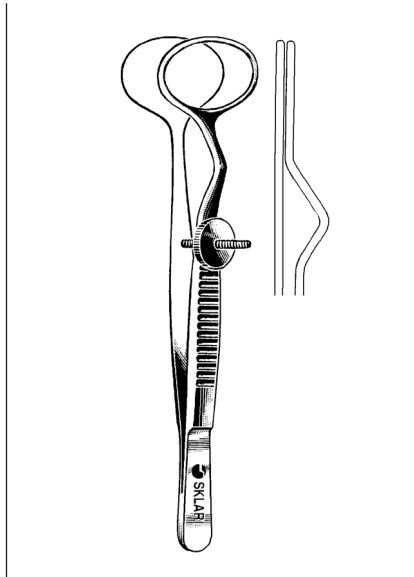


FRANCIS CHALAZION FORCEPS

3¾" LONG
66-1255 12x14mm Upper Plate

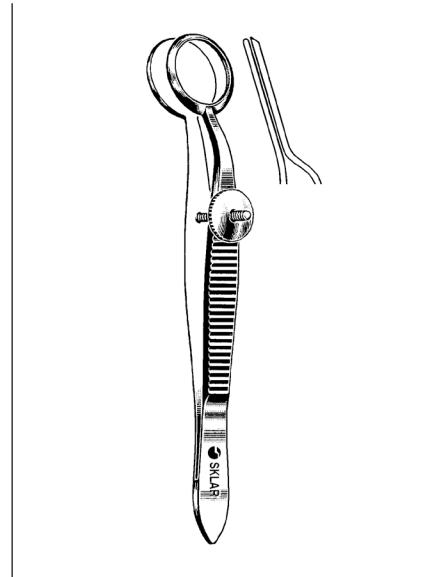


HEATH CHALAZION FORCEPS

LOWER SHAFT CURVED
UPPER SHAFT OFFSET TO PREVENT
CRUSHING OF LID
66-1037 3½" 11mm diameter
66-1137 3½" 16mm diameter

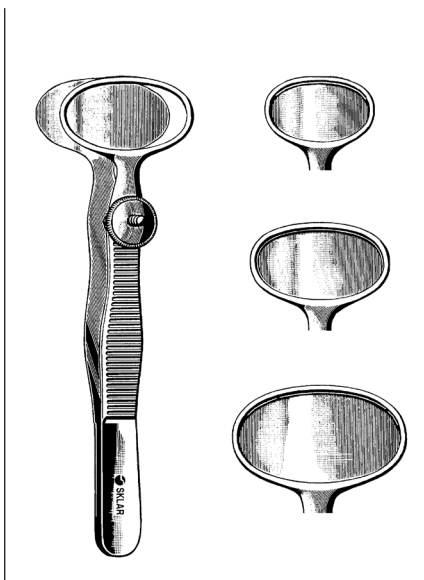


DESMARRES CHALAZION FORCEPS

66-1038 3½" Small
66-1036 3½" Medium
66-1035 3½" Large

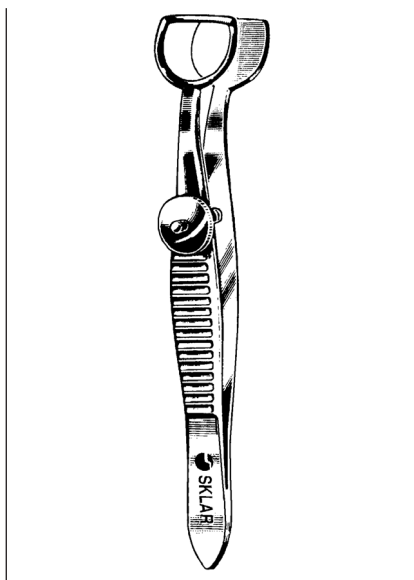


WEIS CHALAZION FORCEPS

SERRATED AND FENESTRATED
66-1265 4" 9x12mm jaw

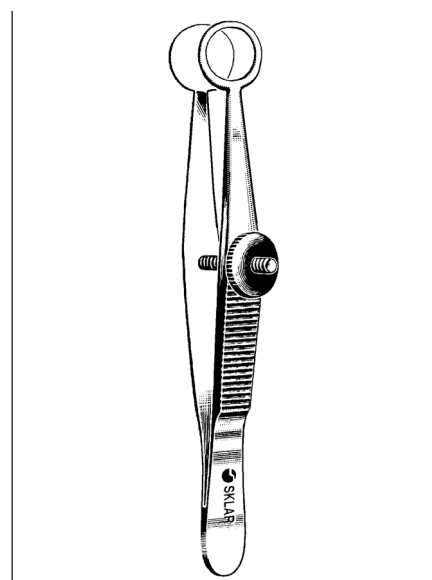


AYER CHALAZION FORCEPS

66-1435 3½" 9mm Inside Diameter

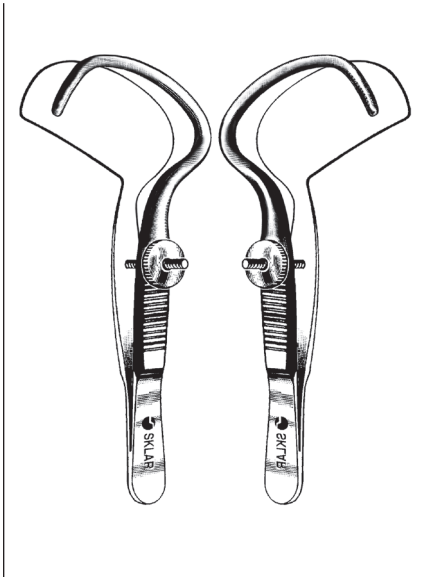


**SNELLEN ENTROPIUM
FORCEPS**

3½" 35MM
66-1280 Left
66-1282 Right

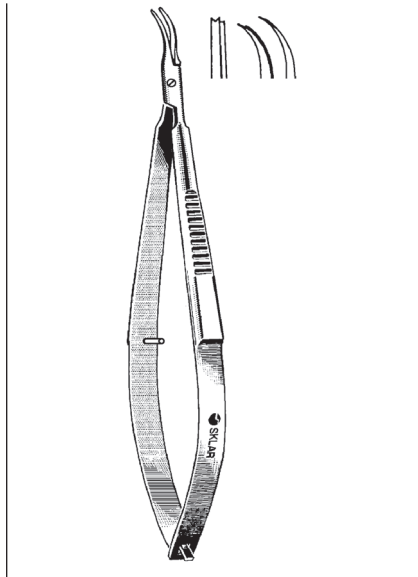


**SHEPARD II - IOL INSERTING
FORCEPS**

NOTCHED TIP (.5MM X .4MM) 4.4"
66-7656 With Lock-with Irrigation

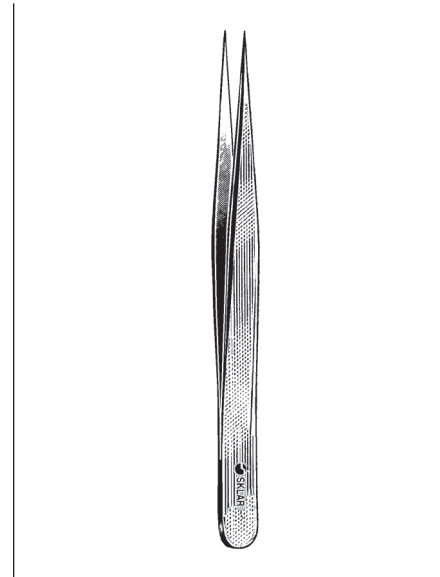

JEWELERS STYLE FORCEPS

66-7438 4½" #1 Fine Tip

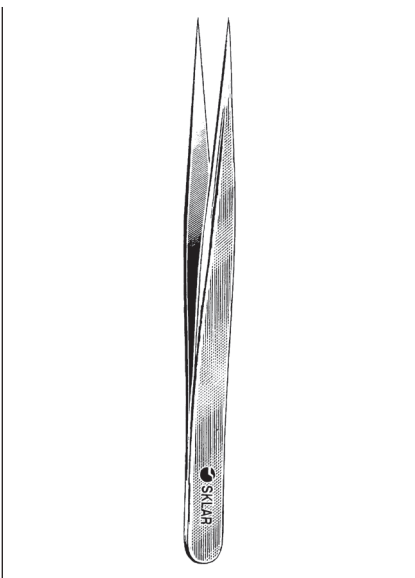

JEWELERS STYLE FORCEPS

66-7447 #3 Straight



JEWELERS STYLE FORCEPS

66-7442 #3C Straight Delicate



JEWELERS STYLE FORCEPS


66-7542 #4 Straight Delicate




JEWELERS STYLE FORCEPS
66-7642 #5 Straight




JEWELERS STYLE FORCEPS
66-7645 #5 Angled

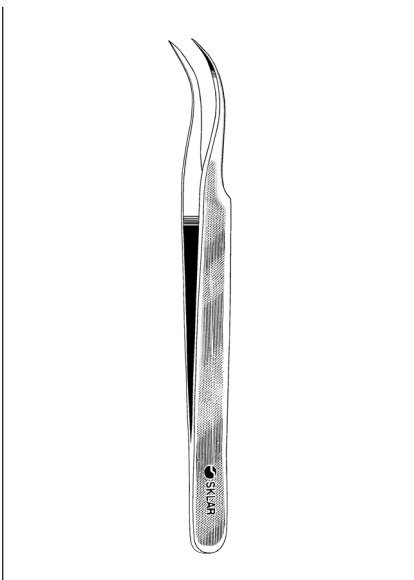


JEWELERS STYLE FORCEPS
66-7644 #7 Curved

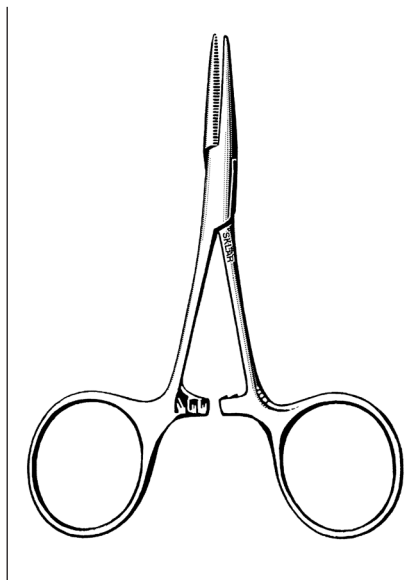

HARTMANN MOQUITO FORCEPS
17-1235 3¾" Straight
17-1335 3¾" Curved

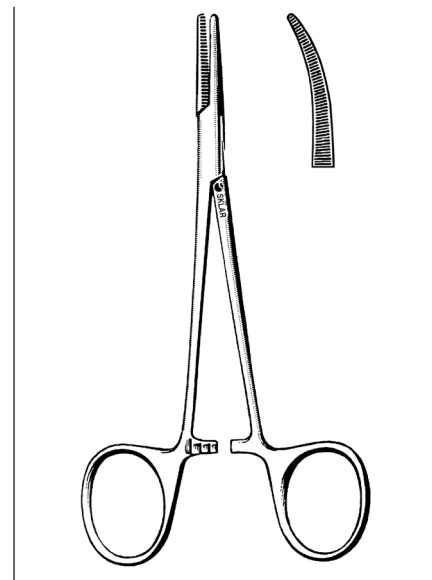


HALSTED MOQUITO FORCEPS
17-1450 5" Straight
17-1550 5" Curved

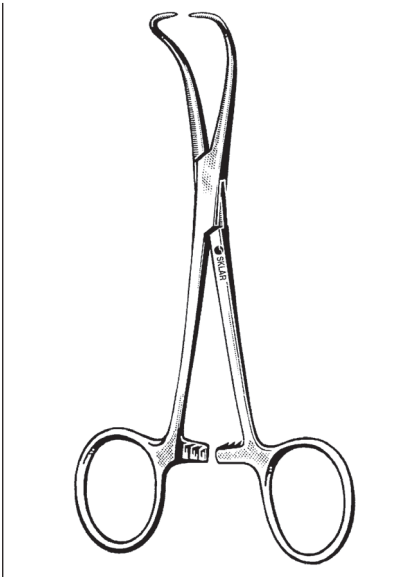


BACKHAUS TOWEL CLAMP

47-2935 3½"
47-2955 5"

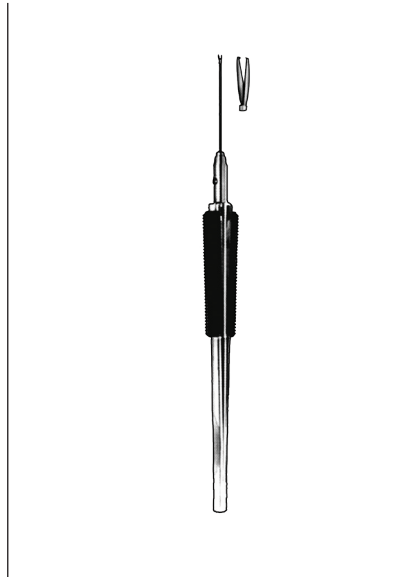

VITREORETINAL TISSUE FORCEPS

66-4023 1x2 Teeth

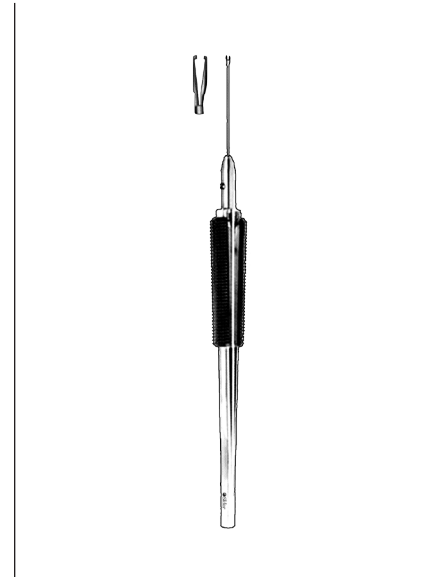

VITREORETINAL FORCEPS

- 66-6155 Straight End Grip, Smooth, 20 Gauge Tips with Plastic Handle
- 66-6156 Angled 90°, Smooth, 20 Gauge Tips with Plastic Handle
- 66-6157 Straight, Serrated, 20 Gauge Tips with Plastic Handle
- 66-6158 Straight, 20 Gauge Tips with Plastic Handle




DODICK NUCLEUS CRACKER

FENESTRATED HANDLE
66-6257 Curved




ERNEST NUCLEUS CRACKER

EXTRA DELICATE
66-6255 Cross-Action Serrated Paddle Shaped Jaws

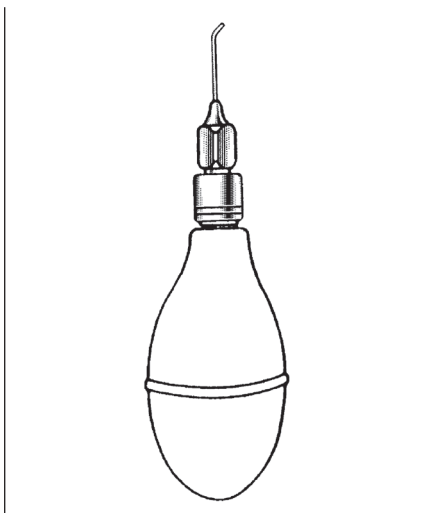


BISHOP-HARMON ANTERIOR CHAMBER CANNULAS

- 66-6810 Flattened 19 Gauge Angled 7mm tip Complete with Bulb and Adapter
- 66-6811 Bishop-Harmon Cannula Only
- 66-6812 Adaptor for Irrigator
- 66-6813 Silicone Bulb Only
- 66-6814 Silicone and Adaptor

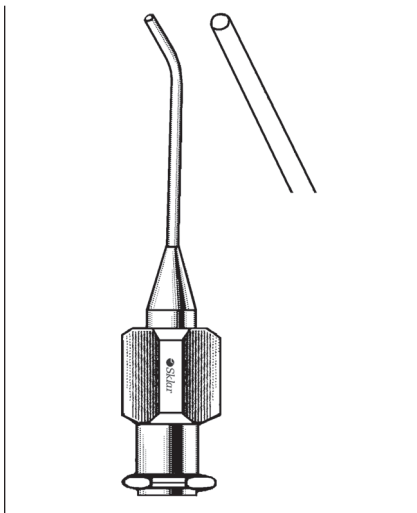


AIR INJECTION CANNULA

66-6900 27 Gauge

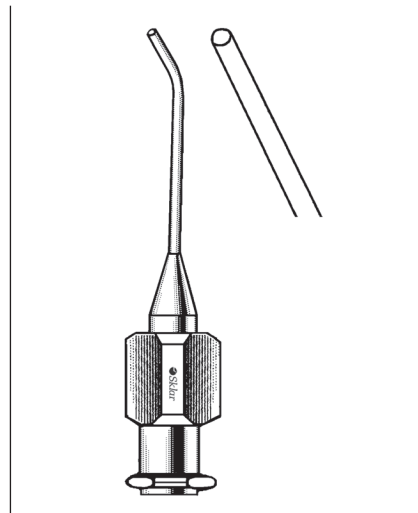


AIR INJECTION CANNULA

66-6902 30 Gauge

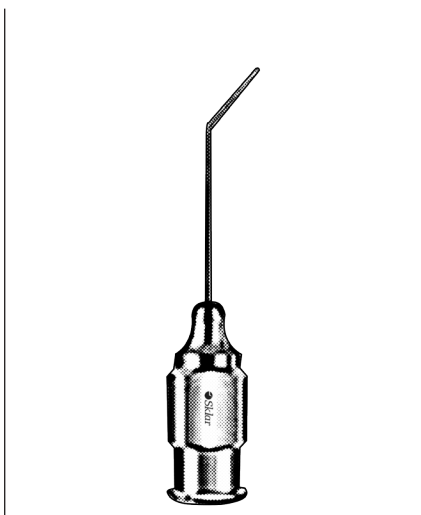


RAININ AIR INJECTION CANNULA

- 66-6901 30 Gauge
- 66-6903 27 Gauge

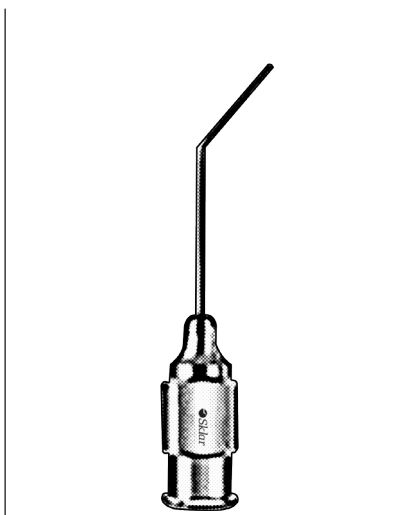


VISCOELASTIC ASPIRATING CANNULA

66-6917 22 Gauge

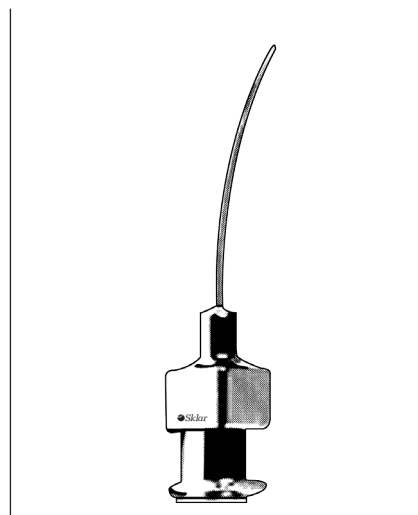

SAUTER AIR HYDRODISSECTION CANNULA

- 66-6965 27 Gauge Vertically Flattened Tip

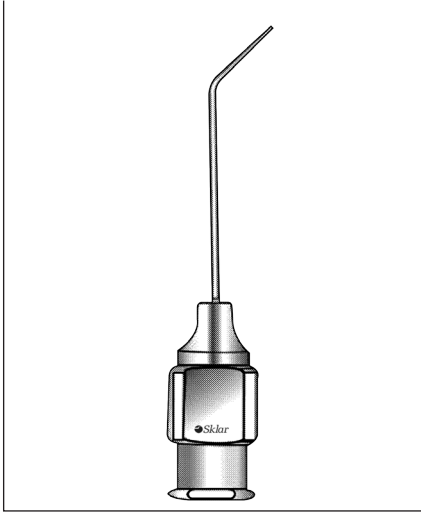


PEARCE HYDRODISSECTION CANNULA

66-6960 27 Gauge




STEVENS SUB-TENON'S ANESTHESIA CANNULA

66-6942 19 Gauge

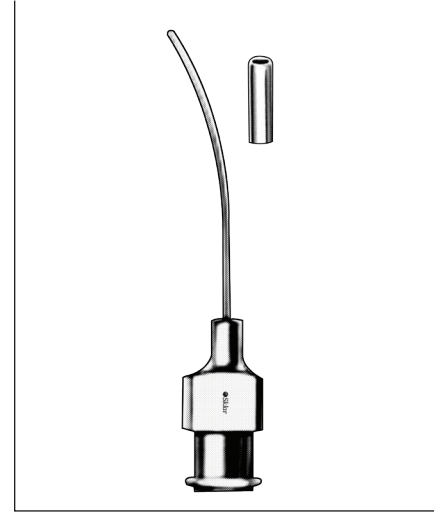


FUKASAKU ANESTHESIA CANNULA

66-6940 24 Gauge Smooth Tip

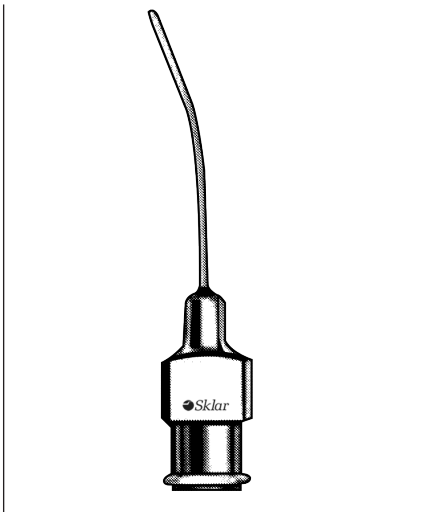


CORBIN ANESTHESIA CANNULA

66-6944 23 Gauge

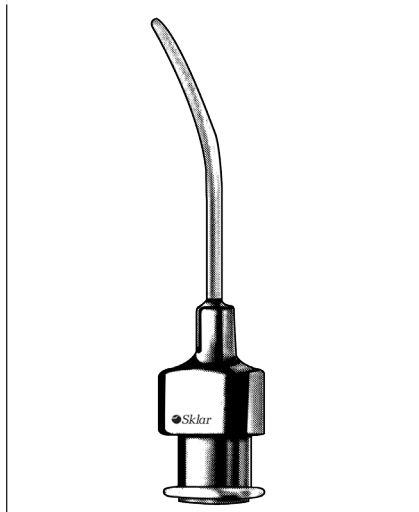


GROSS ANESTHESIA CANNULA

66-6946 21 Gauge

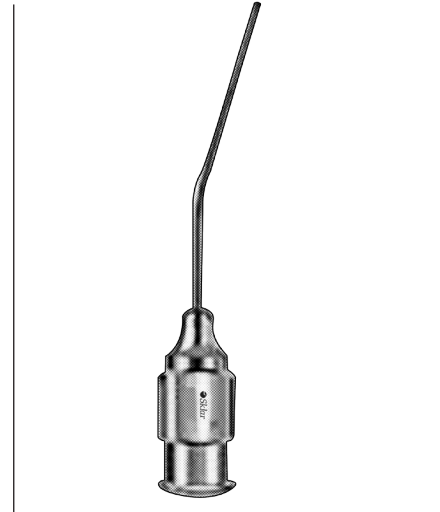


TROUTMAN CANNULA


66-6948 25 Gauge Delicate Olive Tip

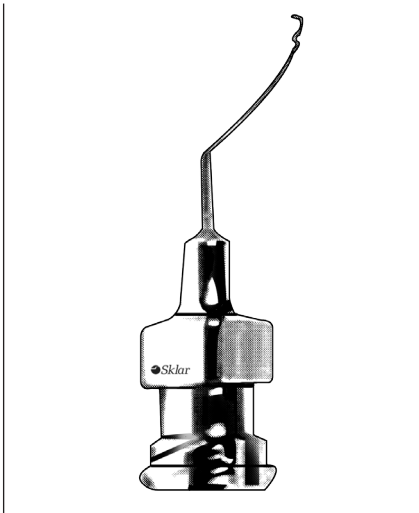


KRATZ CYSTOTOME CANNULA
66-6950 22 Gauge Sharp Cutting Edge

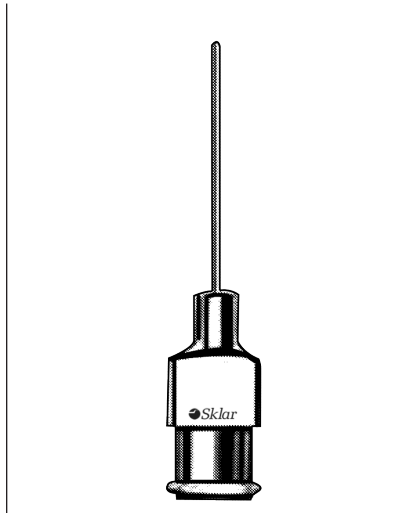


MCINTYRE CANNULAS
SMOOTH
66-6980 23 Gauge Straight, Blunt Tip
66-6981 23 Gauge Angled, Blunt Tip
66-6982 26 Gauge Straight
66-6983 26 Gauge Angled
66-6984 30 Gauge Straight
66-6985 30 Gauge Angled

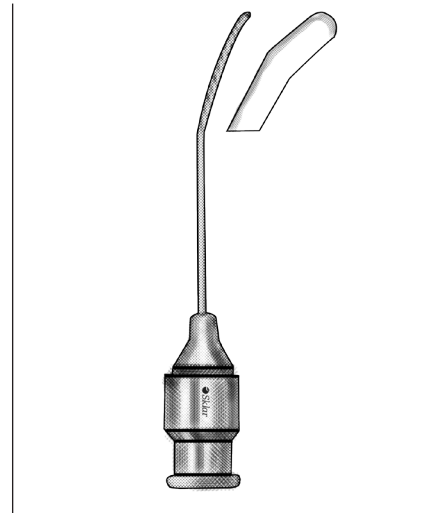


KELMAN CANNULA
66-6911 19 Gauge Right
66-6913 19 Gauge Left

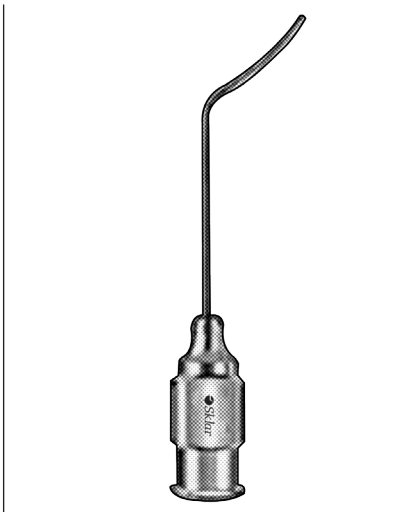


RANDOLPH CANNULA
66-6919 19 Gauge Flat Tip

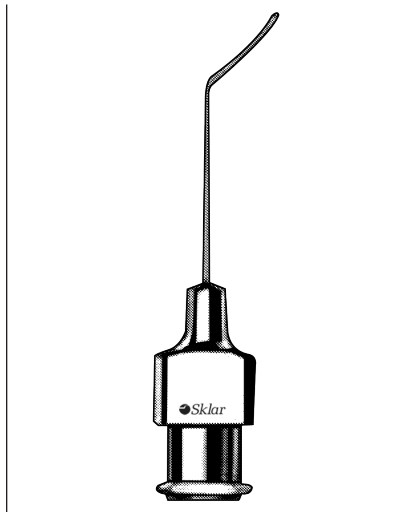

FUKASAKU LASIK CANNULA
66-6970 23 Gauge

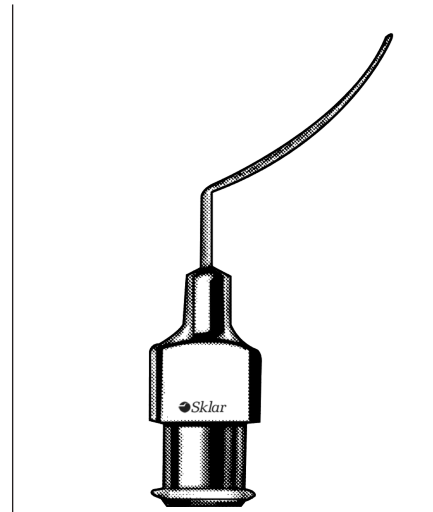


GIMBEL LASIK CANNULAS
66-6971 25 Gauge Right
66-6972 25 Gauge Left

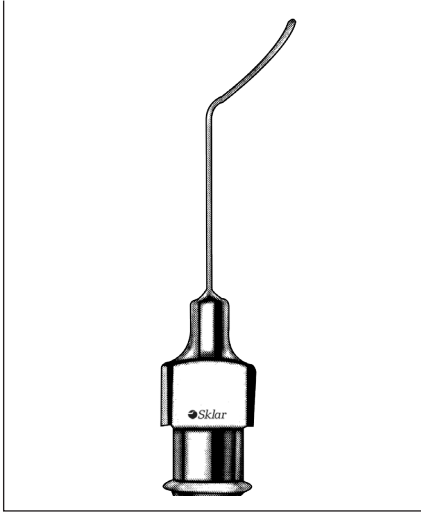


BURRATO LASIK CANNULAS

66-6975 23 Gauge
66-6976 25 Gauge

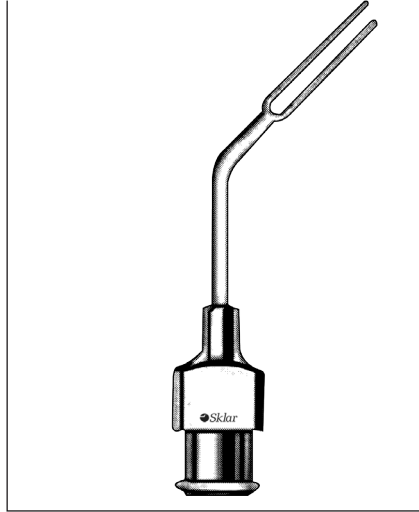


VIDAURRI LASIK CANNULAS

66-6978 Vertical
66-6979 Horizontal

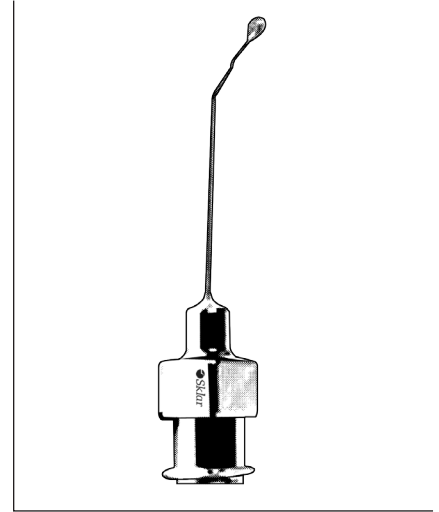


WELSH CANNULAS

66-6935 25 Gauge Angled, Flat Olive Tip
66-6936 25 Gauge Right
66-6937 25 Gauge Left

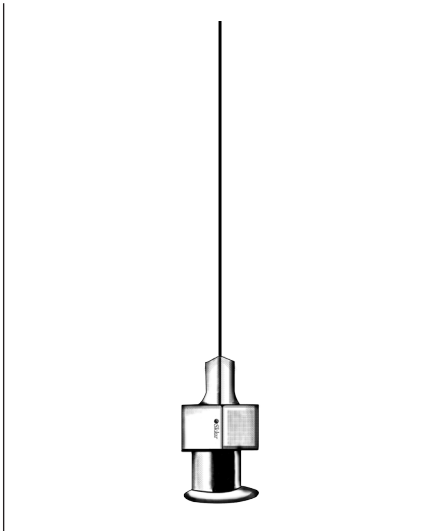


MCINTYRE CANNULAS

66-6904 23 Gauge Straight
66-6905 23 Gauge Curved

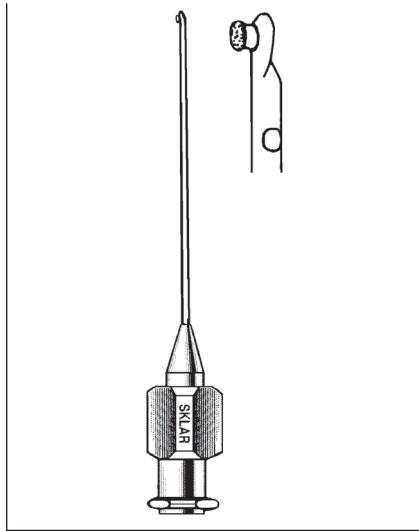


**GRAETHER COLLAR
BUTTON MICRO-IRIS
RETRACTORS**

ROUGHED POSTERIOR SURFACE, 25MM
66-6920 23 Gauge Straight
66-6922 23 Gauge Angled 45°

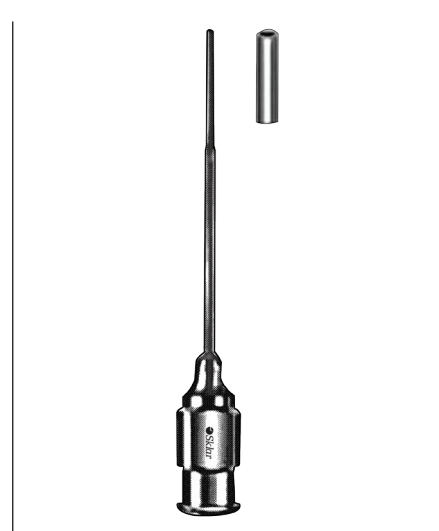


**LACRIMAL CANNULAS
MALLEABLE**

66-6907 23 Gauge Straight
66-6908 23 Gauge Curved

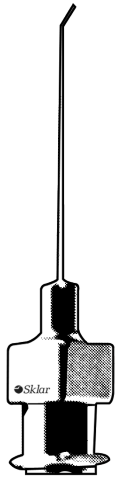


KNOLLE IRRIGATING CANNULAS

66-6967 23 Gauge
66-6969 30 Gauge

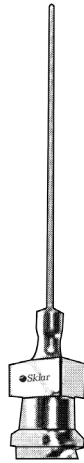


KELMAN IRRIGATING CANNULA

66-6906 22 Gauge

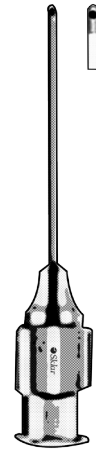


ANIS ASPIRATING CANNULAS

66-6990 23 Gauge 4mm, Straight
66-6991 23 Gauge 4mm, Angled

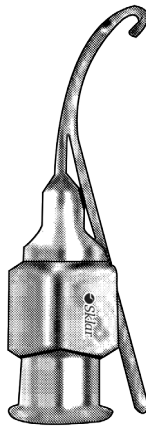


SIMCOE IRRIGATING/ASPIRATING CANNULA

66-6932 23 Gauge U-Shaped with 0.3mm Anterior ASP
Port with Tubing and Adapter, Curved Right

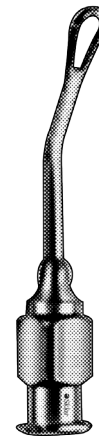

SHEETS IRRIGATING VECTIS


66-1000 20 Gauge Loop

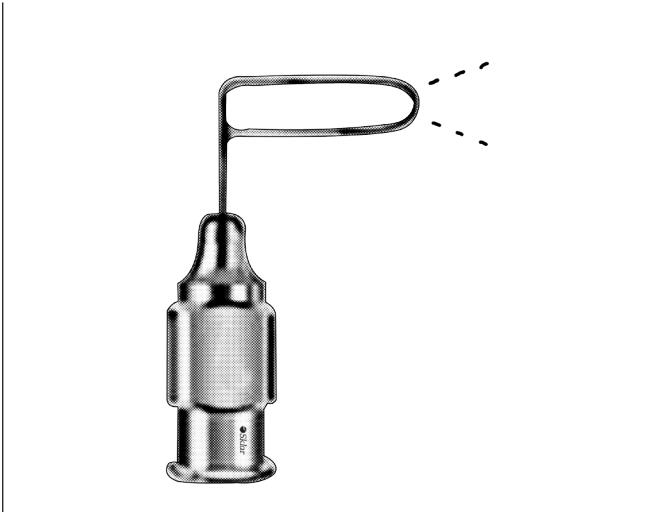

SIMCOE IRRIGATION NUCLEUS LOOPS

65-3296 Right

65-3297 Left

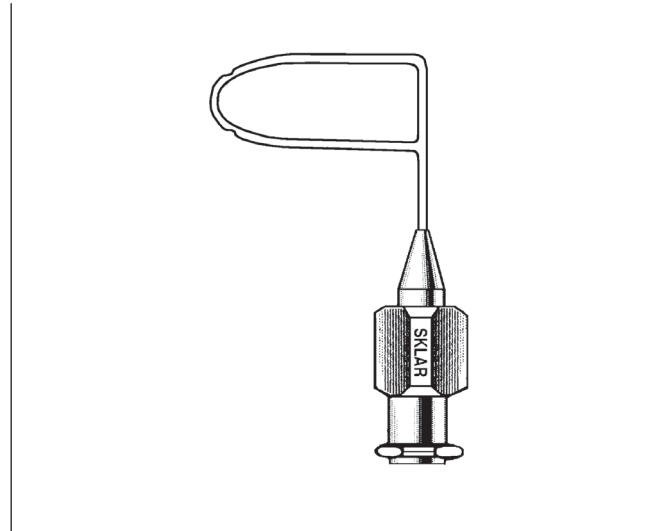


SIMCOE NUCLEUS DELIVERY CANNULA

THIN WALL

66-6925 25 Gauge

Two ports loop 13mm long
4mm Wide Serrations on Lens Loop,
Right

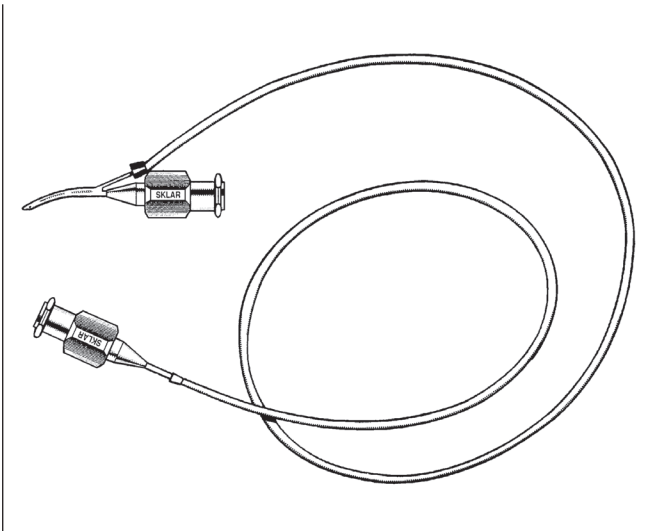

SIMCOE IRRIGATING/ASPIRATING CANNULA


REVERSE 15MM LONG

66-6930 23 Gauge T.W. x 23 gauge T.W. Aspirating Port Facing into Curve, Aspiration on Needle Hub, Irrigation in Tubing Hub

SIMCOE CANNULA


23 GAUGE THIN WALL

66-6932 Double Barreled Irrigating-Aspirating

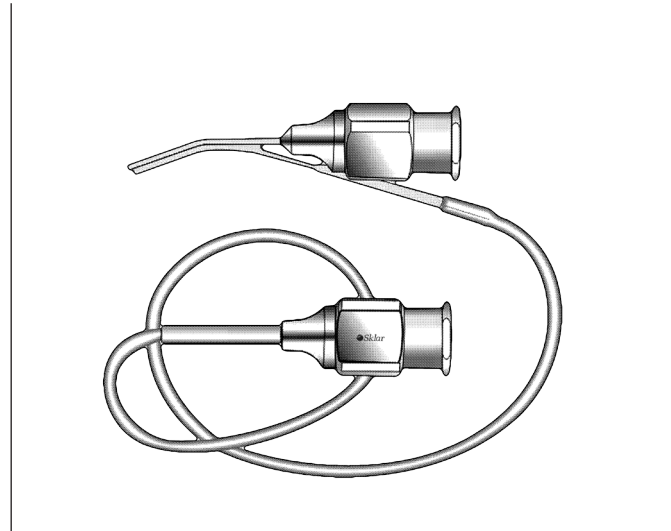


GILLS IRRIGATING/ASPIRATING CANNULA

66-6933 23 Gauge Angled

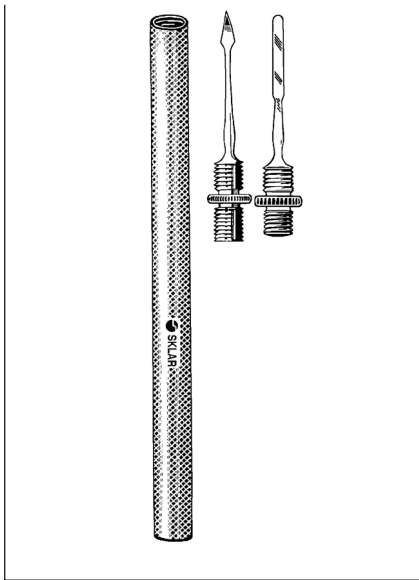


DIX NEEDLE AND SPUD
REVERSIBLE HANDLE

- 65-4030 Complete
- 65-4025 Needle, Blade Only
- 65-4026 Spud Only
- 65-4031 Single End Handle Only Chrome
- 65-4032 Double End Handle Only Chrome

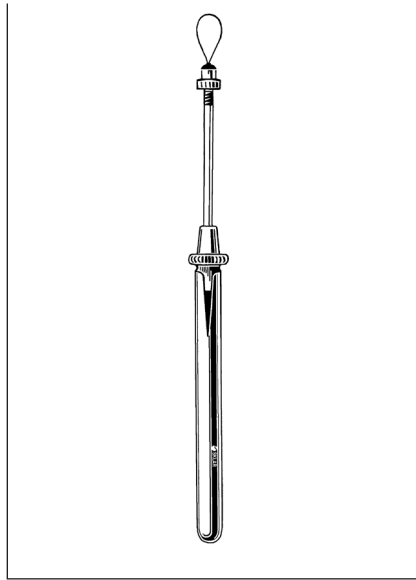


BAILEY FOREIGN BODY REMOVER
REVERSIBLE HANDLE

- 65-4080 Chrome Plated

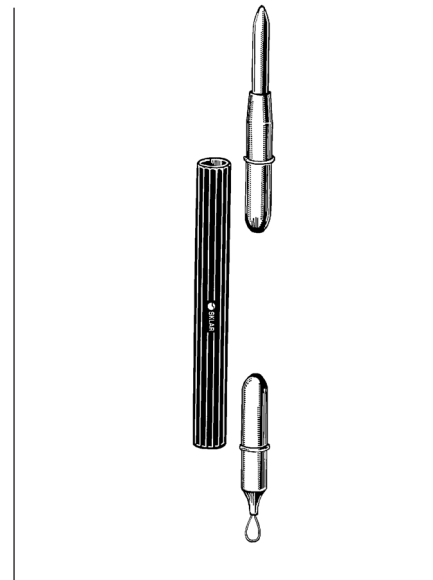


JUNIOR MAGNETIC PROBE & FOREIGN BODY LOOP
REVERSIBLE HANDLE

- 65-7422

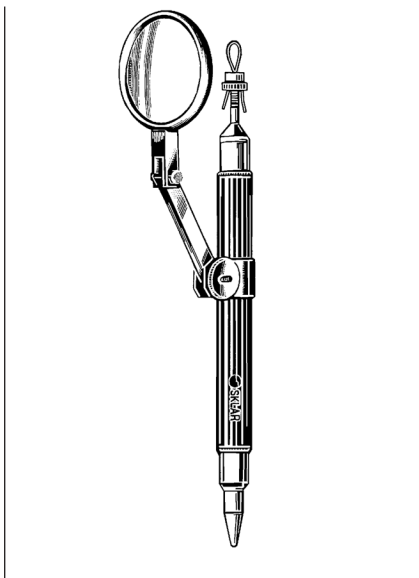


JUNIOR MAGNETIC PROBE & FOREIGN BODY LOOP
REVERSIBLE HANDLE

- 65-7425 With Magnifier Chrome
- Non-Autoclavable

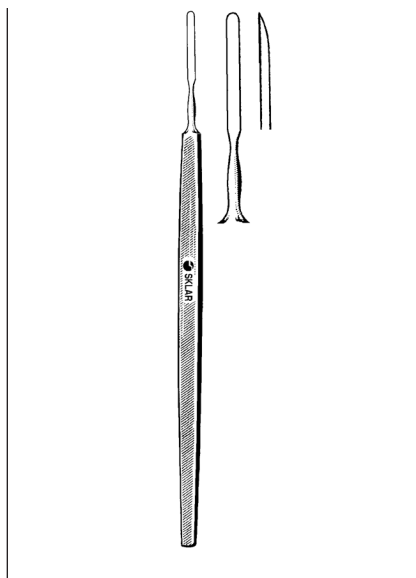

DIX FOREIGN BODY SPUD

- 65-7254 Blade Beveled at Tip

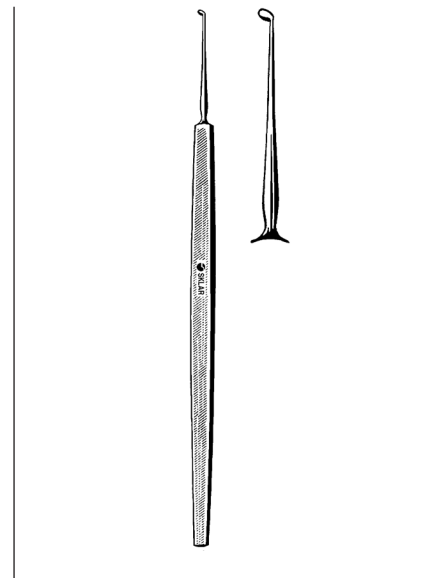


LAFORCE FOREIGN BODY SPUD


- 65-7255 4 3/4" 2mm

FOREIGN BODY INSTRUMENTS


Foreign Body Spuds

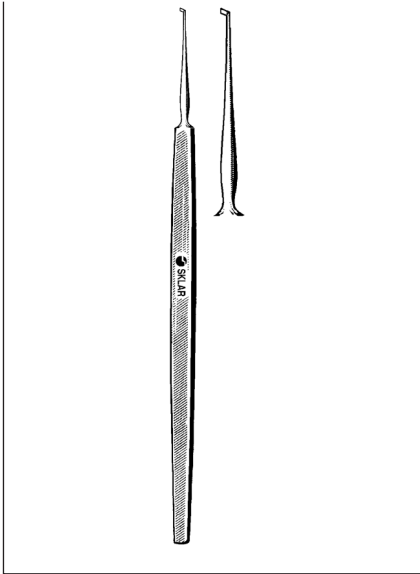

FRANCIS FOREIGN BODY SPUD
65-7257 Flag-Shaped Blade

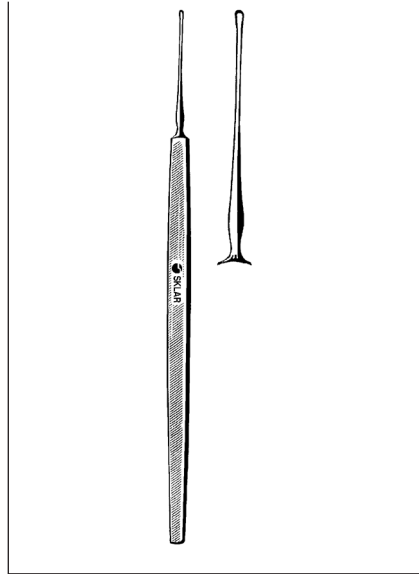

DAVIS FOREIGN BODY SPUD
65-7260 Chisel Shape

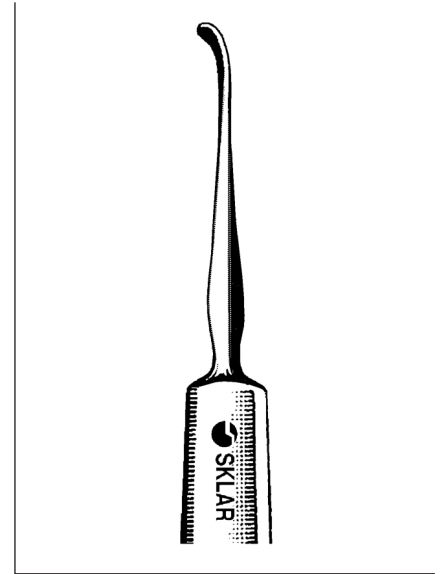


ELLIS FOREIGN BODY SPUD
CURVED
65-7265 4½" 1.2mm Blade



FOREIGN BODY NEEDLES
65-7270 Straight
65-7272 Curved




ALVIS FOREIGN BODY SPUD
WITH FINE POINT
65-7275 4¾" 1x3mm Oval Curette

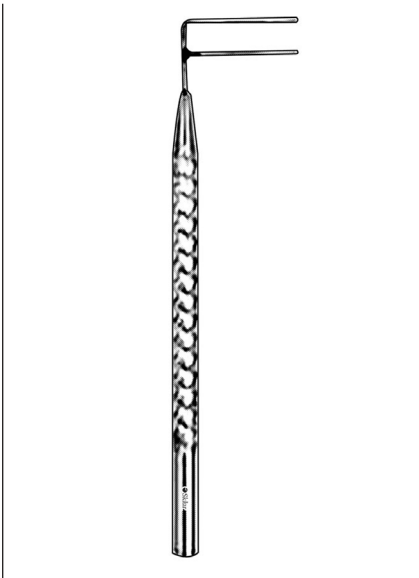


HARMS TRABECULOTOMY PROBE

65-4140 Right
65-4141 Left

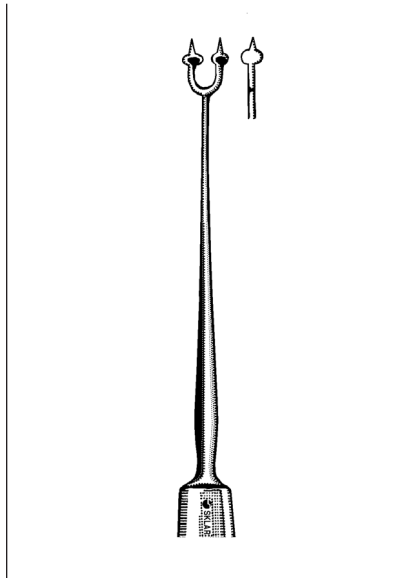


BURCH FIXATION PICK

65-3255 5/4" 2 Prong

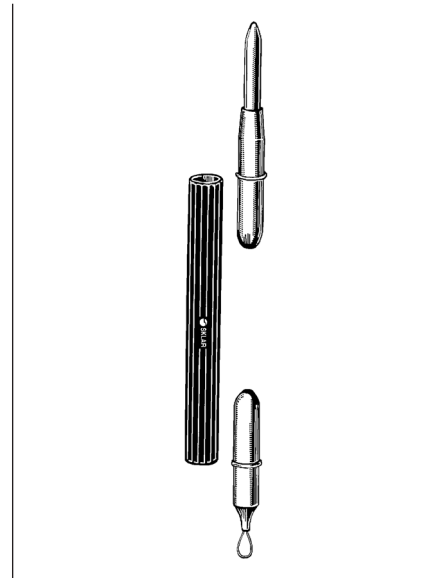

MAGNETIC PROBE AND FOREIGN BODY REMOVER


65-7422 Autoclavable to 143° Celsius

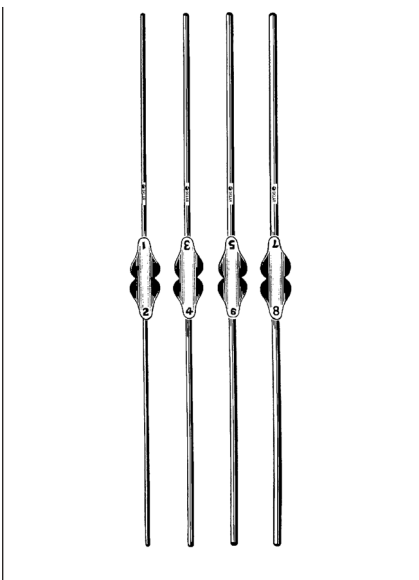

BOWMAN LACHRYMAL DILATORS

DOUBLE END


65-4098 #0000/000
65-4099 #00/0
65-4101 #1-2
65-4103 #3-4
65-4105 #5-6
65-4107 #7-8
65-4100 Set of 4 #1 thru #8

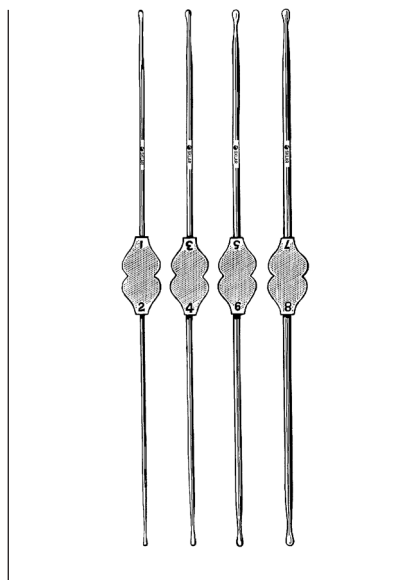

WILLIAMS LACHRYMAL DILATORS

DOUBLE END STERLING SILVER

65-4129 #0000-000
65-4128 #00-0
65-4121 #1-2
65-4123 #3-4
65-4125 #5-6
65-4127 #7-8
65-4120 Set of 4 #1 thru #8

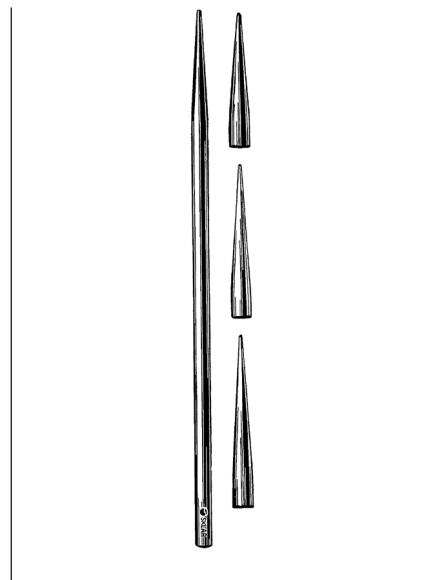

WILDER LACHRYMAL DILATORS


65-7001 Short Taper
65-7002 Medium Taper
65-7003 Long Taper



INFANT LACHRYMAL DILATOR

EXTRA DELICATE 4¼"

65-7000 .5mm at tip

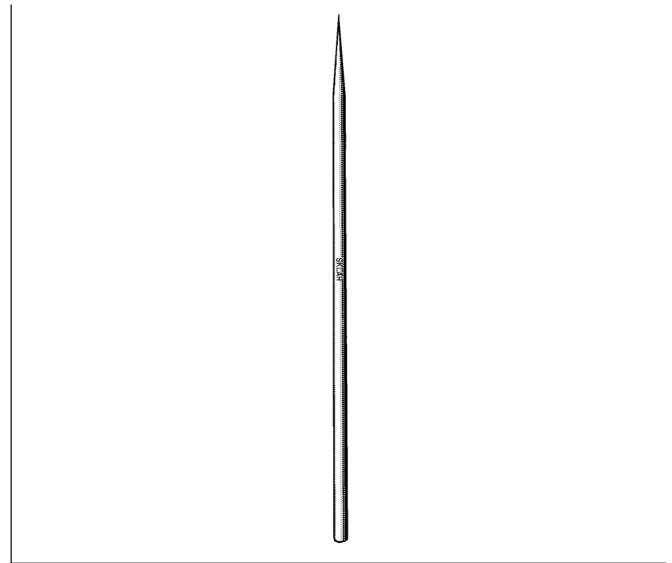

RUEDEMANN LACHRYMAL DILATOR

2¾"

65-6950 Short Delicate
Needle Tips



MULDOON LACHRYMAL DILATOR

WITH STRAIGHT DISTAL END TO GUIDE DILATOR 4"


65-6952



CASTROVEIJO LACHRYMAL DILATOR

DOUBLE END TAPERS, 22x18MM


65-6960 5½"

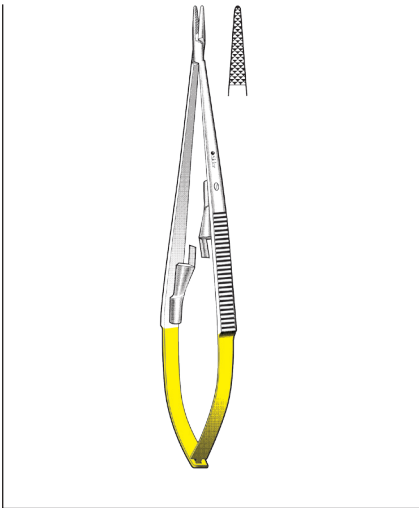

**TC CASTROVIEJO
NEEDLE HOLDER**

WITH LOCK


21-8046 5½" Straight

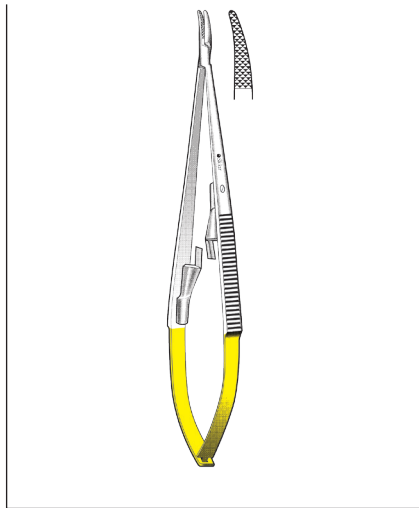

**TC CASTROVIEJO
NEEDLE HOLDER**


WITH LOCK

21-8047 5½" Curved

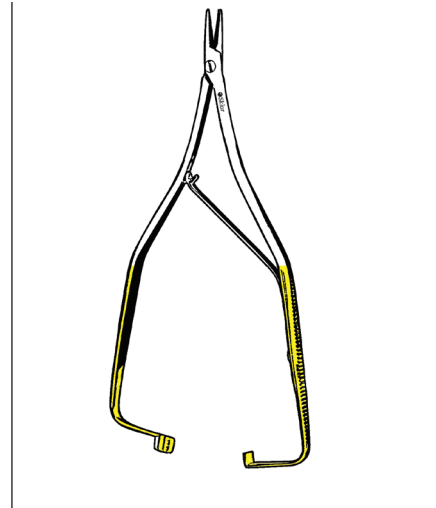


**TC BOYNTON
NEEDLE HOLDER**

21-8041 5"

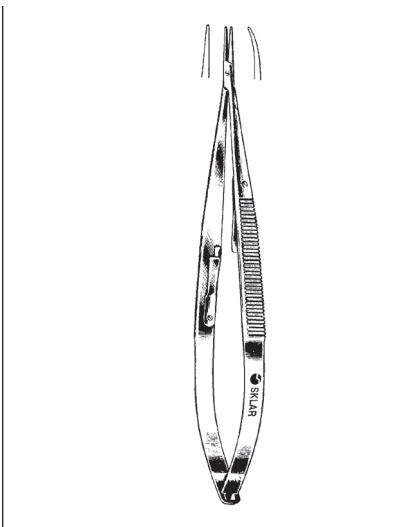

CASTROVIEJO NEEDLE HOLDERS
STANDARD HANDLE, EXTRA DELICATE, SMOOTH
JAWS TO HOLD 7-0 SUTURES (ESPECIALLY MILLED
HANDLE TO IMPROVE TRACTION)


- 65-6350 5½" Straight Jaw without Catch
- 65-6352 5½" Straight Jaw with Catch
- 65-6354 5½" Curved Jaw with Catch
- 65-6351 5½" Curved Jaw without Catch

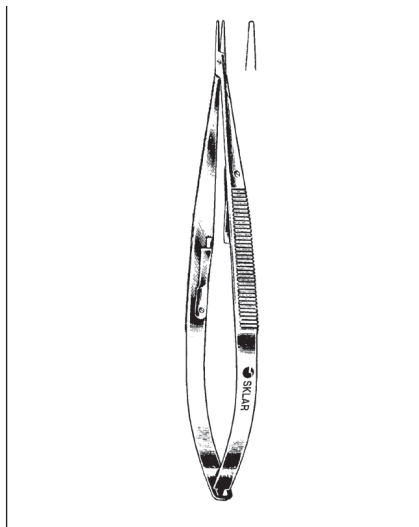

**CASTROVIEJO
NEEDLE HOLDERS**


WITH CATCH

- 65-6357 5½" Straight
- 65-6358 5½" Curved

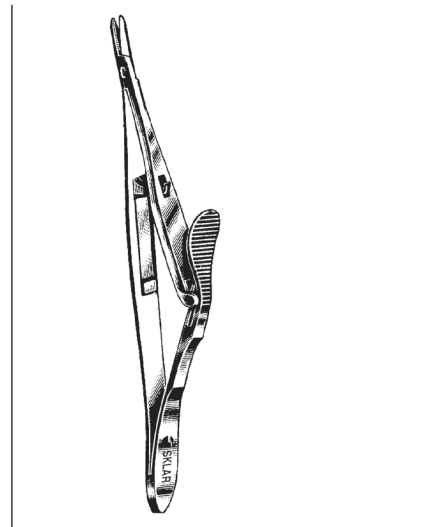

**CASTROVIEJO-KALT
NEEDLE HOLDER**

65-6055 6½" Straight

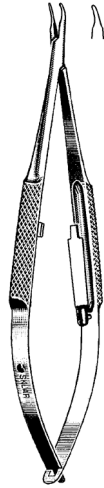


**BARRAQUER
NEEDLE HOLDERS**
ROUND GRIP SPRING HANDLE,
WITH INSIDE CATCH
CURVED JAW

65-6446 5½" Delicate
65-6450 7½" Delicate


STRAIGHT JAW

65-6445 5½" Delicate
65-6449 7½" Delicate

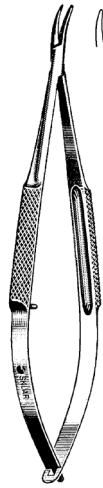

**BARRAQUER
NEEDLE HOLDERS**
DELICATE WITHOUT LOCK


65-6452	5¼"	Curved
65-6454	5½"	Straight
65-6455	5½"	Curved
65-6456	7½"	Straight
65-6457	7½"	Curved

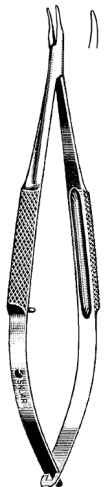

**TROUTMAN
NEEDLE HOLDERS**

EXTRA DELICATE, ROUND GRIP SPRING HANDLE

65-6461 4" Straight, without Inside Catch
65-6460 4" Curved, with Inside Catch
65-6462 4¾" Curved, without Inside Catch

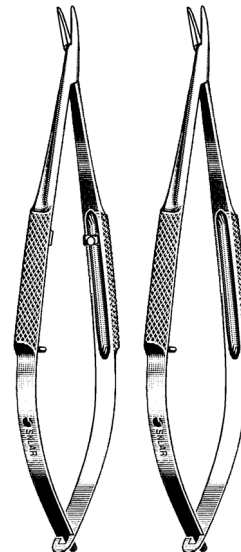

COHAN NEEDLE HOLDERS

MINI-MICRO 7MM JAWS, 4" (100MM)

65-6470 Curved without Lock
65-6475 Curved with Lock



BOYNTON NEEDLE HOLDER

20-3080 4¾"



NEIVERT NEEDLE HOLDER

20-1760 5" One Offset Ring



OLSEN-HEGAR NEEDLE HOLDERS

SCISSOR COMBINATION

20-2748	4¾"	Smooth	Delicate
20-2747	4¾"	Serrated	Delicate
20-2755	5½"	Serrated	
20-2765	6½"	Serrated	
20-2775	7"	Serrated	



STEVENS NEEDLE HOLDER

65-6265 6"



98-1034 BASIC EYE SURGERY SET

PRODUCT NUMBER	DESCRIPTION	FEATURES	QUANTITIES
06-2903	Scalpel Handle	#3	1
06-2907	Scalpel Handle	#7	1
18-1050	Halsted Mosquito Forceps	Straight, 5"	2
18-1150	Halsted Mosquito Forceps	Curved, 5"	4
20-2070	Mayo-Hegar Needle Holder	7"	1
21-1260	TC Crile-Murray Needle Holder	Serrated, 6"	1
21-8013	TC Ryder Micro Needle Holder	Serrated, 6"	1
22-2652	Ragnell Dissecting Scissor	Curved, Flat Tip, 5"	1
22-2760	Reynolds Scissor	Curved, Narrow Tip, 6"	1
22-4150	Semken Dressing Forceps	Serrated, Narrow, 5"	2
22-4350	Semken Tissue Forceps	1x2 Teeth, Narrow, 5"	2
22-8360	Miller-Senn Retractor	Double End, Sharp, 6"	1
22-8460	Miller-Senn Retractor	Double End, Blunt, 6"	1
22-9027	Alm Retractor	4x4 Teeth, Blunt, 2¾"	1
23-1106	Sklarlite™ Operating Scissors	Straight, Sharp/Blunt, 5½"	2
23-1158	Sklarlite™ Strabismus Scissor	Straight, 4½"	1
23-1164	Sklarlite™ Mayo Dissecting Scissor	Straight, 5½"	1
23-1167	Sklarlite™ Mayo Dissecting Scissor	Curved, 6¾"	1
23-1280	Sklarlite™ Sklarcut Scissor	Curved, Blunt, 5¾"	1
23-2510	Sklarlite XD™ Crile Forceps	Straight, 5½"	12
23-2513	Sklarlite XD™ Crile Forceps	Curved, 5½"	12
47-1047	Wagner Scissors	Straight, Sharp/Sharp, 4¾"	2
47-1068	Wagner Scissor	Curved, Sharp/Blunt, 4¾"	1
47-1145	Iris Scissor	Straight, Sharp/Sharp, 4½"	1
47-1235	Iris Scissor	Curved, Sharp/Sharp, 3½"	1
47-1245	Iris Scissor	Curved, Sharp/Sharp, 4½"	1
47-1447	Adson Brown Forceps	9x9 Teeth, Side Grasping, 4¾"	2
47-2834	Jansen Retractor	3x3 Teeth, Sharp, 4"	1
47-2955	Backhaus Towel Clamp	5¼"	1
50-1372	Adson Forceps	Straight, 7¼"	3
50-1472	Adson Forceps	Curved, 7¼"	3
50-2012	Frazier Suction Tubes	12 French	2
50-3047	Adson Tissue Forceps	1x2 Teeth, 4¾"	1
50-3147	Adson Dressing Forceps	Serrated, 4¾"	1
55-2855	Gemini Clamps	Angled, Delicate, 5½"	2
57-1445	Crile Retractors	Double End, Small, 4½"	2
60-3598	Zalkind Ribbon Retractors	Malleable, ¾" x 7¾"	2
64-1242	Stevens Tenotomy Scissor	Straight, Blunt/Blunt, 4½"	1
64-1342	Stevens Tenotomy Scissor	Curved, Blunt/Blunt, 4½"	1
64-2242	Castroviejo Corneal Scissor	Curved Right, 4¼"	1
64-2342	Castroviejo Corneal Scissor	Curved Left, 4¼"	1
65-1532	Lancaster Eye Speculum	3¼"	1
65-1535	Castroviejo Speculum	3½"	1
65-1842	Desmarres Lid Retractor	14mm	1
65-6357	Castroviejo Needle Holder	Straight, Smooth, with Lock, 5½"	1
66-2740	Iris Forceps	Straight, Serrated, 4"	1
66-2940	Eye Dressing Forceps	Half Curve, Serrated, 4"	1
66-3340	Eye Tissue Forceps	Straight, 1x2 Teeth, 4"	1

98-1034 BASIC EYE SURGERY SET (CONTINUED)

PRODUCT NUMBER	DESCRIPTION	FEATURES	QUANTITIES
66-3540	Eye Tissue Forceps	Half-Curve, 1x2 Teeth, 4"	1
66-5340	Arruga Capsule Forceps	Standard Curve, 4"	1

98-1094 CATARACT SURGERY SET

PRODUCT NUMBER	DESCRIPTION	FEATURES	QUANTITIES
06-2907	Scalpel Handle	#7	1
06-2910	Scalpel Handle	#3 With Metric Scale	1
21-8040	TC Castroviejo Needle Holder	Straight, Smooth, with Lock, 5½"	1
21-8043	TC Castroviejo Needle Holder	Curved with Lock, Delicate, 5½"	1
22-1241	Stevens Tenotomy Scissor	Curved, Delicate, Sharp/Sharp, 4½"	1
23-2505	Sklarlite XD™ Halsted Mosquito Forceps	Straight, 5"	4
23-2506	Sklarlite XD™ Halsted Mosquito Forceps	Curved, 5"	6
47-1135	Iris Scissor	Straight, Sharp/Sharp, 3½"	1
47-1235	Iris Scissor	Curved, Sharp/Sharp, 3½"	1
47-2935	Backhaus Towel Clamps	3½"	4
64-1735	Wilmer Scissor	Angled on Flat, 4"	1
64-1950	Vannas Scissor	Straight, Delicate, 3"	1
64-2242	Castroviejo Corneal Scissor	Curved Right, 4¼"	1
64-2342	Castroviejo Corneal Scissor	Curved Left, 4¼"	1
64-3050	Iris Micro-Miniature Scissor	Straight, Extra-Delicate, 4"	1
64-3142	Westcott Tenotomy Scissor	Right, Blunt, 5"	1
64-3280	Gradle Dissecting Scissor	Curved, Sharp, 3¾"	1
65-1528	Park-Maumenee Eye Speculum		1
65-1532	Lancaster Eye Speculum	3¼"	1
65-1535	Castroviejo Speculum	3½"	1
65-1842	Desmarres Lid Retractor	14mm	1
65-2960	Troutman Blade Breaker		1
65-3216	Graefe Strabismus Hooks	Medium	2
65-4062	Castroviejo Cyclodialysis Spatula	Double End, .5mm	1
65-6732	Castroviejo Caliper	3¼"	1
65-7635	Fox Aluminum Eye Shields		2
65-8216	Wilder Lens Scoop	Smooth	1
65-9085	Kirby Hook & Expressor	Double End, 10mm x 2mm Hooks, 5¾"	1
66-1036	Desmarres Chalazion Forceps	Medium, 3½"	1
66-1940	Graefe Fixation Forceps	Without Catch, 4mm, 4½"	1
66-2350	Elsching Fixation Forceps	Angled Up, 4¼"	1
66-2640	Castroviejo Fixation Forceps	1x2 Teeth, .3mm, 4"	1
66-2740	Iris Forceps	Straight, Serrated, 4"	1
66-2940	Eye Dressing Forceps	Half Curve, Serrated, 4"	1
66-3240	Eye Tissue Forceps	Straight, 1x2 Teeth, 4", Non-Magnetic	1
66-3340	Eye Tissue Forceps	Straight, 1x2 Teeth, 4"	1
66-5340	Arruga Capsule Forceps	Standard Curve, 4"	1
66-6105	Castroviejo Suture Forceps	Straight, .5mm, 4"	1
66-7261	Colibri Corneal Utility Forceps	1x2 Teeth, .4mm, 3"	1
97-1450	Scalpel Handle	#3K For Miniature Edge Blades	1



98-1031 CHALAZION SET


PRODUCT NUMBER	DESCRIPTION	FEATURES	QUANTITIES
06-2911	Scalpel Handle	#9	1
17-1335	Hartmann Mosquito Forceps	Curved, Serrated, 3½"	2
21-8040	TC Castroviejo Needle Holder	Straight, Smooth, with Lock, 5½"	1
47-1145	Iris Scissor	Straight, Sharp, 4½"	1
47-2935	Backhaus Towel Clamps	3½"	2
64-1342	Stevens Tenotomy Scissor	Curved, Blunt/Blunt, 4½"	1
64-1960	McPherson Vannas Scissor	Straight, 3"	1
64-3142	Westcott Tenotomy Scissor	Right, Blunt, 5"	1
64-3144	Westcott Tenotomy Scissor	Left, Blunt, 5"	1
65-1742	Barraq Wire Speculum	Large, Fenestrated Blades, 15mm	1
65-1840	Desmarres Lid Retractors	10mm	2
65-1843	Desmarres Lid Retractors	16mm	2
65-3170	Meyhoff Chalazion Curette	#0, 1.5mm	1
65-3171	Meyhoff Chalazion Curette	#1, 1.8mm	1
65-3172	Meyhoff Chalazion Curette	#2, 2.2mm	1
65-6358	Castroviejo Needle Holder	Curved, Smooth, with Lock, 5½"	1
66-1036	Desmarres Chalazion Forceps	Medium, 3 ½"	1
66-1137	Heath Chalazion Forceps	16mm, 3½"	1
66-1236	Lambert Chalazion Forceps	11mm, 3½"	1
66-2640	Castroviejo Fixation Forceps	1x2 Teeth, .3mm, 4"	1
66-4032	Bishop-Harmon Eye Forceps	Serrated, 3½"	1
66-4132	Bishop-Harmon Eye Forceps	1x2 Teeth, 3½"	1
66-6115	Castroviejo Suture Forceps	1x2 Teeth, .5mm, 4¼"	1
66-6900	Air Injection Cannula	27 Gauge	1
75-5460	Metzenbaum Scissor	Straight, 5¾"	1
97-1450	Scalpel Handle	#3K For Miniature Edge Blades	1

98-1033 ENUCLEATION SET

PRODUCT NUMBER	DESCRIPTION	FEATURES	QUANTITIES
17-1550	Halsted Mosquito Forceps	Curved, 5"	2
64-1150	Enucleation Scissor	Curved, 5"	1
64-1242	Stevens Tenotomy Scissor	Straight, Blunt/Blunt, 4½"	1
64-1342	Stevens Tenotomy Scissor	Curved, Blunt/Blunt, 4½"	1
65-1532	Lancaster Eye Speculum	3¼"	1
65-3195	Jamison Strabismus Hook	10mm, 5"	1
65-8151	Bunge Evisceration Spoon	Small	1
65-8155	Bunge Evisceration Spoon	Large	1
66-2742	Sklar LiteGrip™ Iris Forceps	Straight, Serrated, Fenestrated Handle, 4"	1

98-1035 EYE PREP SET


PRODUCT NUMBER	DESCRIPTION	FEATURES	QUANTITIES
10-1497	Emesis Basin	10oz.	1
10-1670	Iodine Cups	6oz., 37/16" x 2"	2
10-1675	Medicine Cups	2oz., Graduated Measure	2
10-1691	Mayo Tray	13 11/16" x 9 13/16" x 3/4"	1
15-1567	Mayo Dissecting Scissor	Straight, 6¾"	1
74-1470	Ballenger Sponge Forceps	Straight, Serrated, 7"	2

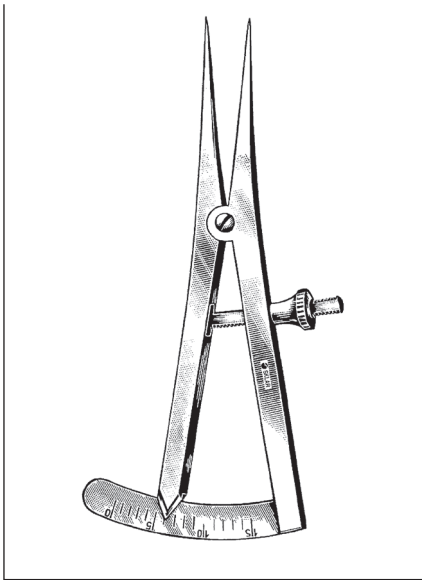

CASTROVIEJO CALIPERS

3/4" GRADUATED MEASURE


- 65-6730 0-15mm Straight
- 65-6732 0-20mm Straight
- 65-6734 0-20mm Curved

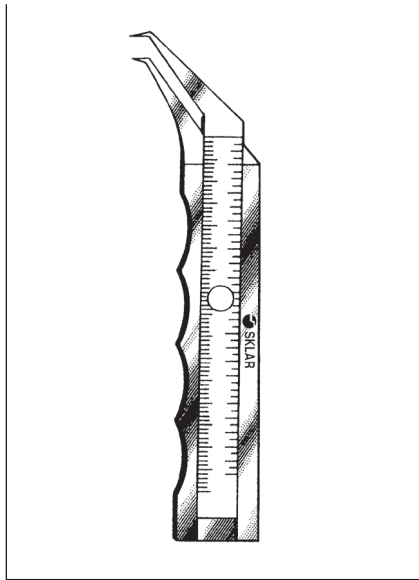

THORPE CALIPER


GRADUATED IN INCHES AND MM

- 65-6735 4 1/2"

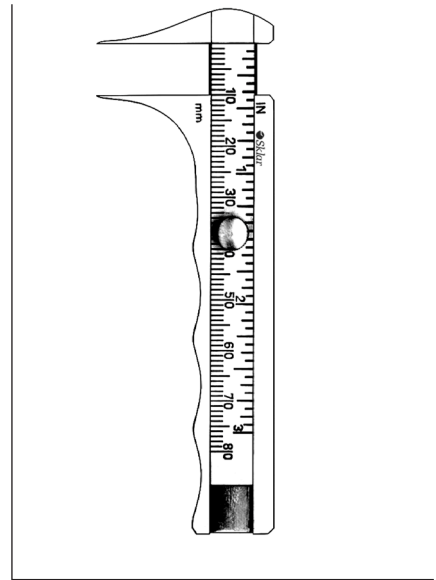

JAMESON CALIPER

65-6743 Up to 80mm

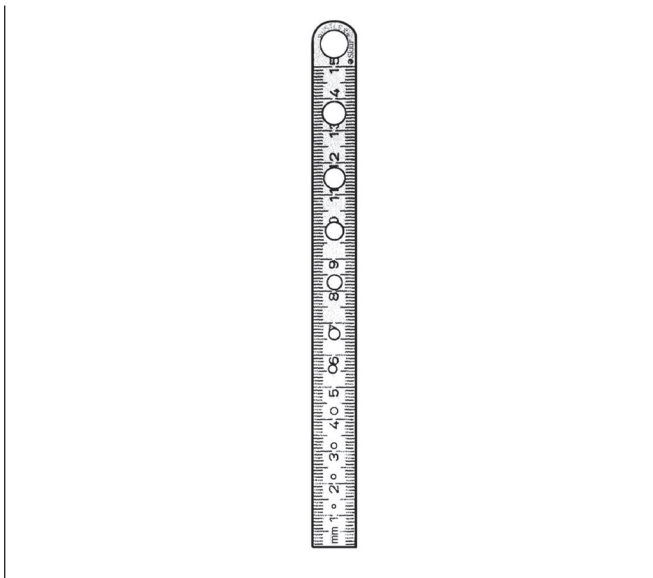

PERFORATED RULERS


- 06-2859 6"



STEEL RULERS


GRADUATED IN INCHES AND MM

- 06-2860 6"
- 06-2862 8"
- 06-2861 12"
- 06-2863 20"

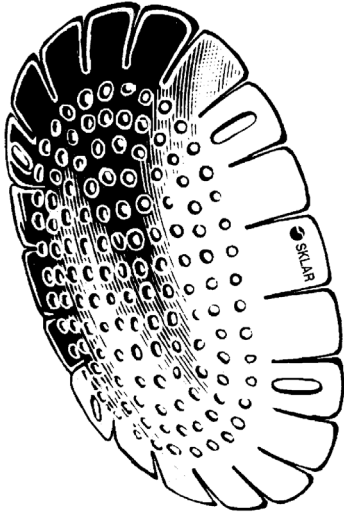

FOX ALUMINUM EYE SHIELDS


65-7634 Package of 12
65-7635 Package of 1

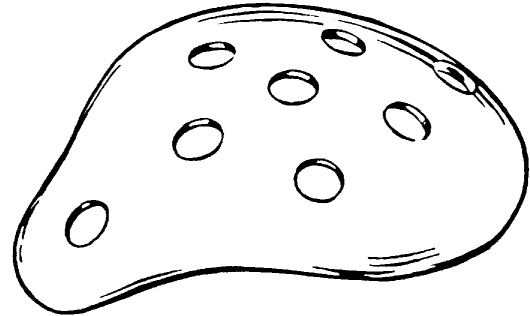

DISPOSABLE EYE SHIELDS

96-1070 Single Each
96-1071 Single Dozen

Lighter in weight, Sklar disposable eye shields offer an alternative to metal shields. Made of high impact plastic. Not Autoclavable.

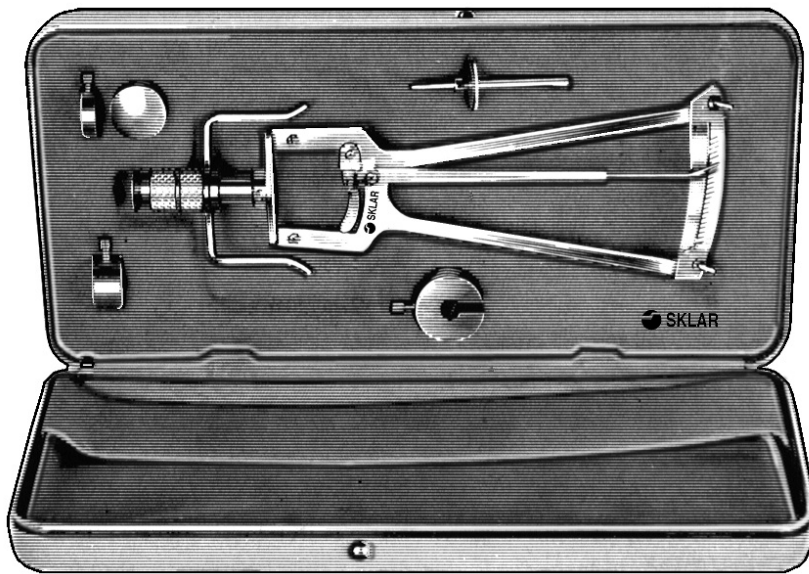

TEST BLOCK • FITTED TONOMETER CASE • CERTIFIED

SCHIOTZ TONOMETER

SKLAR ORIGINAL

65-7523 Screw-on 5.5GR Weight 2 Additional Weights 7.5 10GR
65-7516 Sklar Tonometer Case Case Only

Sklar is pleased to offer a full line of Care & Cleaning agents to disinfect, decontaminate, sterilize, lubricate and protect your stainless steel surgical instruments and equipment. For expert care of your surgical instruments at every stage of the cleaning process, trust Sklar to deliver the right cleaning agent for you.



3 EASY STEPS FOR THE CARE AND CLEANING OF YOUR INSTRUMENTS

1 DECONTAMINATION 2 MINUTE KILL TIME

Sklar Soak™ and Sklar Disinfectant are intended to safely decontaminate soiled surgical instruments immediately after use. These ready to use, non-toxic formulas are safe for use on medical and surgical equipment including stainless steel, chrome, glass, plastic, Plexiglas, baked on painted and other non-porous surfaces. Effective against H1N1, HIV-1, HSV, VRE, Rotaviruses, Hep B & C as well as TB, MRSA and A-Baumannii.



2 CLEANING DETERGENTS AND ENZYMATICS

Choose from a wide selection of Sklar cleaning agents. First use Sklar Spray-Zyme™ foam pre-cleaner to coat instruments until ready to process. Next use Sklar Enzymatic™, a fast acting detergent specifically formulated for soaking/cleaning instruments heavily laden with organic debris. Finally, use Sklar Kleen™ as your neutral pH, non-corrosive detergent for manual, ultrasonic or automatic cleaning. Sklar detergents and enzymatics are effective in the cleaning process of surgical instruments, as well as metals, glassware, plastics and rubber.



3 LUBRICATION SKLAR LUBRICANT

To guarantee the extended life and performance of your surgical instruments, lubrication should be a standard part of your instrument processing regimen. Easy to use Sklar Instru-Guard™ products improve function and contain rust inhibitors that protect against corrosion, rust and stains. Following lubrication, instruments should be sterilized according to the instrument manufacturer's instructions and/or stored until use.





SKLAR-CIDE™

HIGH LEVEL DISINFECTING/STERILIZING SOLUTION

Sporicidal • Bactericidal • Virucidal • Fungicidal

SKLAR-CIDE provides a safe and efficient means for the high-level disinfection or sterilization of medical, ophthalmic, dental and surgical devices including endoscopes and lensed instruments. It may be used as a standard soak solution or in automatic reprocessing units set at room temperature. Unlike other solutions, there is no need to increase the solution temperature to provide high-level disinfection in 25 minutes.

SKLAR-CIDE is a surfactant-free, activated glutaraldehyde liquid chemical sterilant and high-level disinfectant. Its active ingredient is glutaraldehyde in a nominal concentration of 3%. When the powdered activator is added, the solution turns green in color to indicate its readiness for use. In addition to the active ingredient, this ready-to-use solution contains buffers, wetting agents, and a rust and corrosion inhibitor and it has a pleasant fragrance.

SKLAR-CIDE produces high-level disinfection at room temperature in as little time as 25 minutes, and complete sterilization in 10 hours at room temperature.

- Reusable for 28 days instead of 14, safer and saves time and money.
- Surfactant free - Will not damage scope surfaces or cloud lenses.
- Can be used safely on reusable medical devices, including endoscopes, so there is no need to stock several products.

PRODUCT NUMBER	FEATURES	QUANTITIES
10-1623	28 HLD 1 Gallon Bottles	4/case
10-1624	28 HLD 1 Gallon Bottle	1 bottle

GLUTARALDEHYDE STRIPS

PRODUCT NUMBER	FEATURES	QUANTITIES
96-4105	2.1% MEC 60 Strips / Bottle	2/case



SKLAR SOAK™

**INSTRUMENT PRE-SOAK
& DECONTAMINANT
RUST INHIBITOR!**

2 MINUTE KILL TIME

Immersing soiled instruments in SKLAR SOAK effectively pre-cleans and decontaminates, without fear of rusting!

SKLAR SOAK is an effective pre-cleaner that breaks down bio-burden, lifting blood and other organic debris from the surface, while penetrating box locks and crevices. The rust inhibitor protects the delicate chromium oxide layer of the instrument, thus defending against rust and corrosion. Using SKLAR SOAK™ as a pre-cleaner is the first line of defense against rust, prolonging the life of the instrument and therefore protecting your investment!

Soaking for only two minutes thoroughly decontaminates instruments, reducing the risk of exposure to pathogens such as H1N1, HIV-1, HSV, VRE, Rotaviruses, Hep B & C as well as TB, MRSA and A. Baumannii. However, because SKLAR SOAK™ contains a rust inhibitor, instruments may soak for several hours without fear of rusting! Employees are protected from cross contamination, and at the same time can enjoy the freedom to reprocess instruments when it's convenient!

SKLAR SOAK is ideal for use on all stainless steel instruments:

- PH balanced, designed specifically for stainless steel instruments.
- Intermediate level disinfectant with broad spectrum of efficacy.
- Rust inhibitor.
- Non-caustic.
- Ready to use.
- No special handling or disposal.
- Fragrant, no offensive odors.
- Non-glutaraldehyde.

PRODUCT NUMBER	FEATURES	QUANTITIES
10-1603	1 Gallon Bottles	4/case
10-1609	1 Gallon Bottle	1 bottle





SKLAR DISINFECTANT

READY-TO-USE SPRAY SURFACE
DISINFECTANT

2 MINUTE KILL TIME

- Ready to use, non-toxic formula.
- Specially formulated for the control of cross contamination.
- Effective against a broad range of pathogens including H1N1, HIV-1, HSV, VRE, Rotaviruses, Hep B & C as well as TB, MRSA and A. Baumannii.
- Safe for all areas of the facility including the NICU, operating rooms, isolation rooms, patient care areas and laboratories.
- Satisfies OSHA's Bloodborne Pathogen Standards.
- Non-irritating formula.
- EPA registered.

PRODUCT NUMBER	FEATURES	QUANTITIES
10-1643	24 oz. Spray Bottles	6/case
10-1642	24 oz. Spray Bottle	1 bottle
10-1653	1 Gallon Bottles	4/case
10-1654	1 Gallon Bottle	1 bottle
10-1644	2.5 Gallon Bottles	2/case



SKLAR DISINFECTANT SURFACE WIPES

READY-TO-USE, PRE-MOISTENED
TOWELETTES

Same Effective Formula as Sklar Disinfectant!

2 MINUTE KILL TIME

- Ready to use, non-toxic formula.
- Specially formulated for the control of cross contamination.
- Effective against a broad range of pathogens including H1N1, HIV-1, HSV, VRE, Rotaviruses, Hep B & C as well as TB, MRSA and A. Baumannii.
- Safe for all areas of the facility.
- Satisfies OSHA's Bloodborne Pathogen Standards.
- Destroys odor-causing bacteria.
- 7" x 10" non-woven, tear resistant towelette.
- Clean scent.
- EPA registered.

PRODUCT NUMBER	FEATURES	QUANTITIES
10-1617	100 Towelettes / Canister	12/case
10-1616	100 Towelettes / Canister	6/case





SKLAR SPRAY-ZYME™

SURGICAL INSTRUMENT PRE-CLEANER

Multi-Enzyme Based Instrument Presoak & Foam Spray Cleaner

The newest product from Sklar for expert care of your surgical instruments. This multi-enzymatic spray creates a heavy foam which lasts for hours, keeping soiled instruments moist to avoid encrustation while also pre-cleaning prior to processing.

- Prevents Organic Encrustation - enzyme foam stops blood and bodily fluids from drying onto surgical instruments.
- No More Spills - messy spills between the operating room and central cleaning /processing department are eliminated.
- Neutral pH - safe and effective for stainless steel, tungsten carbide, aluminum and brass instruments.
- Bacteriostatic - microbiological control agents inhibit bacterial growth in the ready-to-use foam solution.
- Long Lasting Foam - saves time, foam lasts many hours after application.
- Multi-Enzymatic - contains a synergistic blend of lipase and protease enzymes.



PRODUCT NUMBER	FEATURES	QUANTITIES
10-2722	22 oz. Refillable Spray Bottles	12/case
10-2724	1 Gallon Bottles	4/case

SKLAR ENZYMATIC™ DETERGENT

MULTI-ENZYME DETERGENT CONCENTRATE

100% Biodegradable

SKLAR ENZYMATIC Detergent is a low-foaming, fast acting detergent specifically designed for use when instruments and/or cloth materials become soiled with organic debris.

SKLAR ENZYMATIC Detergent may be used as a manual instrument cleaner as well as a pre-soak for blood-contaminated linens, clothing and instruments. It is also used in ultrasonic units and in washer/sterilizer/decontaminators. When used as directed, the enzymatic pH neutral formula is safe for use on rubber, plastic, stainless steel instruments, and surgical equipment as well as cloth.

PRODUCT NUMBER	FEATURES	QUANTITIES
10-2777	1 Gallon Bottles	4/case
10-2775	1 Gallon Bottles	2/case
10-2776	1 Gallon Bottle	1 bottle





SKLAR KLEEN™

LOW FOAM LIQUID DETERGENT

The SKLAR KLEEN Low Foam Liquid is concentrated in order to provide a very cost-effective cleaning solution. It is especially formulated to maintain low foam, ensuring effective and trouble-free action in washer/ sterilizers and tunnel washers. Dispersing quickly, the SKLAR KLEEN Low Foam formula is also well suited for automatic injection systems.

- Safe for use in washer / sterilizers.
- Low, Non-Aggressive pH.
- Disperses quickly.
- Non-etching to glassware.
- Concentrated formula.
- Cost-effective solution.

PRODUCT NUMBER	FEATURES	QUANTITIES
10-2701	Low Foam 1 Gallon Bottles	4/case
10-2702	Low Foam 1 Gallon Bottle	1 bottle



SKLAR KLEEN™

LIQUID DETERGENT

The SKLAR KLEEN Liquid Detergent was specifically developed for manual and ultrasonic cleaning. It is an effective yet non-corrosive, non aggressive, phosphate free and biodegradable formula that will not etch glassware or attack instrument metals. It is moderately alkaline, affording optimum cleaning power.

- Ideal for manual and ultrasonic cleaning.
- Non-Corrosive.
- Phosphate free and biodegradable.
- Superior wetting power to penetrate locks, serrations, and crevices.
- Appropriate for stainless steel, metal, glassware, plastics, and rubber.

PRODUCT NUMBER	FEATURES	QUANTITIES
10-1612	Liquid 8 oz. Bottles	6/case
10-1614	Liquid 8 oz. Bottle	1 bottle
10-1613	Liquid 1 Gallon Bottles	4/case
10-1627	Liquid 1 Gallon Bottle	1 bottle





SKLAR KLEEN™ POWDERED DETERGENT

The SKLAR KLEEN Powdered Detergent is an excellent way to reduce or neutralize the effects of corrosive minerals that are inherent in virtually all community water supplies. SKLAR KLEEN Powdered Detergent is ideal for manual and ultrasonic cleaning. One economical 3½ lb. container can make in excess of 150 gallons of cleaning solution. It's non-corrosive, non aggressive, phosphate free formula will not etch glassware or attack your instrument metals.

- Economical and Convenient.
- For all manual and ultrasonic cleaning.
- Appropriate for stainless steel, metal glassware, plastics and rubber.
- Non-Corrosive.
- Easy to Handle.

PRODUCT NUMBER	FEATURES	QUANTITIES
10-1608	Powdered 5 lb. Containers	4/case
10-1607	Powdered 5 lb. Container	1 container
10-1630	Powdered 3.5 lb. Pitchers	6/case
10-2765	Powdered 3.5 lb. Pitcher	1 pitcher



SKLAR INSTRU-GUARD™ ONE STEP

A "ONE STEP" INSTRUMENT
CLEANER AND LUBRICANT

EXTENDS INSTRUMENT LIFE!

SKLAR INSTRU-GUARD ONE STEP contains a penetrating lubricant that protects and helps extend the life of surgical instruments. The detergents and emulsifiers quickly break up and remove organic soils from box locks and other hard to reach places.

SKLAR INSTRU-GUARD ONE STEP contains four rust inhibitors to control corrosion and eliminate scaling and spotting. It does not contain phosphates and can reduce the time and cost involved in processing your instruments by cleaning and lubricating in one easy step.

PRODUCT NUMBER	FEATURES	QUANTITIES
10-2800	1 Gallon Bottles	4/case
10-2801	1 Gallon Bottle	1 bottle





SKLAR INSTRU-GUARD™ LUBE

SURGICAL INSTRUMENT LUBRICANT / RUST INHIBITOR

- Increase the Life of Instruments.
- Protects all Surgical Instruments.

SKLAR INSTRU-GUARD LUBE is an easy-to-use, water soluble, lubricant and rust inhibitor. When used daily as a rinse after every cleaning it keeps moving parts operating smoothly and remains effective after autoclaving, but without leaving an oily residue.

Regular lubrication is essential no matter which cleaning method is employed (including ultrasonic). SKLAR INSTRU-GUARD LUBE is ideal for maintaining instrument quality and performance.

PRODUCT NUMBER	FEATURES	QUANTITIES
10-1635	1 Gallon Bottles Concentrated	4/case
10-1634	1 Gallon Bottle Concentrated	1 bottle
10-1636	8 oz. Spray Bottles	12/case
10-1637	8 oz. Spray Bottle	1 bottle



SKLAR POLISH™

INSTRUMENT STAIN REMOVER

SKLAR POLISH is an effective stain remover for use on all your metal surgical instruments, including stainless steel, chrome and brass. It will brighten instrument trays, basins and racks and may also be used on porcelain and tile surfaces.

- Non Abrasive.
- Removes corrosive stains.
- Extends instruments life.
- "Like New" results.

PRODUCT NUMBER	FEATURES	QUANTITIES
10-1626	8 oz. Jars	12/case
10-1629	8 oz. Jars	6/case
10-1927	8 oz. Jar	1 jar



INSTRUMENT CLEANING BRUSHES

PRODUCT NUMBER	FEATURES	QUANTITIES
10-1650	Stainless Steel Bristles	3/pk
10-1652	Stainless Steel Bristles	12/pk
10-1658	Stainless Steel Bristles	24/pk
10-1665	Stainless Steel Bristles	48/pk
10-1651	Nylon Bristles	3/pk
10-1657	Nylon Bristles	12/pk
10-1659	Nylon Bristles	24/pk
10-1656	Nylon Bristles	48/pk





SKLAR-SHEEN™

AUTOCLAVE AND INSTRUMENT CLEANER AND CONDITIONER

With an efficacious phosphoric acid cleaning agent, Sklar Sheen is a robust spot, stain and rust remover. Ideal for dissolving hard water deposits, use in autoclaves to quickly cut through steam sterilizer scale and deposits.

Versatile and effective, SKLAR SHEEN couples powerful stain and rust removers with corrosion inhibitors, making it tough on stains but gentle on instruments. Recondition instruments to restore them to nearly brand new working condition.

Use to clean and recondition any stainless steel surface to its original luster, including carts, trays and basins.

- Conditions for lasting protection.
- Dissolves hard water deposits.
- Frees up frozen box locks.
- Restores and extends surgical instrument life.



PRODUCT NUMBER	FEATURES	QUANTITIES
10-2738	22 oz. Spray Bottles	12/case
10-2736	22 oz. Spray Bottle	1 bottle

SKLAR INSTRUMENT CARE KIT

A TRIO OF SKLAR'S CARE AND CLEANING PRODUCTS
TO KEEP INSTRUMENTS IN TIPTOP SHAPE

PRODUCT NUMBER	FEATURES	QUANTITIES
10-1625		
	Sklar Lube™ 8 oz. Spray Bottle	1 each
	Sklar Kleen™ Liquid 8 oz. Bottle	1 each
	Sklar Polish™ 8 oz. Jar	1 each
	Stainless Steel Bristle Cleaning Brush	1 each
	Nylon Bristle Cleaning Brush	1 each



TROUBLE SHOOTING

NO STEEL IS TRULY “STAINLESS”

It's a fact that so-called stainless steel is subject to both water-spotting and staining. However what many people identify as a “rust” problem usually turns out to be something else.

Most often the problem is actually a stain caused by a surface deposit. These deposits appear in a variety of colors depending upon the type of deposit involved. Once the cause is found and eliminated, most of the time the problem quickly diminishes or disappears completely.

As you probably know, stainless steel is ideally suited for the surgical suite because it is rust resistant, it can

be honed to an extremely sharp edge or fine point, and it can be hardened to maintain the delicate, yet precise, requirements of the surgeon. But it is this hardenable quality – the amount of carbon used during the manufacturing process – coupled with harsh chemicals and improper care that renders stainless steel susceptible to corrosion.

To protect your instrument investment, Sklar cleaning agents have been specifically formulated to provide cost effective instrument care solutions that help maintain the life and performance of your valuable stainless steel instruments.



889 South Matlack Street

West Chester, PA 19382

610.430.3200

Fax: 610.696.9007

service@sklarcorp.com

www.sklarcorp.com

©Copyright 2016. Sklar Corporation, West Chester, PA. All rights reserved. No portion of this catalog may be reproduced without written permission of Sklar.